G

TRANSWAB®

Transwab® is a unique transport system for the safe and efficient transport of both aerobic and anaerobic bacteria. Each Transwab® unit consists of a sterile peel pack containing a swab stick with Dacron®/ Rayon® tip with a coloured- coded cap and a pre-labelled transport tube. There are numerous design features, which have made Transwab® a world leader. Each transtube contains a generous amount of transport medium (5ml) to totally immerse its swab. The versatile Transwab® is available in a variety of styles suitable for different applications.

INSTRUCTIONS FOR USE

1. Peel back pouch until applicator and Transtube caps are exposed.
2. Remove Transtube cap between forefinger and thumb, and discard.
3. Remove applicator and take sample.
4. Place applicator into Transtube and push down firmly until the Bell cap reaches line.
5. Fill in patient's details.
6. Transport to laboratory immediately.

Formula

Deionised water
Calcium chloride activated
Sodium Chloride
Potassium chloride
Magnesium chloride
Agar
Potassium di-hydrogen phosphate
Di-sodium hydrogen phosphate
Charcoal decolourising powder (Amies charcoal media only)
PH 7.3 +/- 0.2

Material safety information

Transwabs plastic components do not contain latex or PVC.
DO NOT USE IF DAMAGED

QC PERFORMANCE TESTS

Samples are tested to ensure that the minimal inocula of a spectrum of organisms are recovered after a specified duration of time. Suspensions of the following organisms are absorbed on to the swab of each product sample and placed in its respective medium.

<i>Neisseria gonorrhoea</i>	N.C.T.C. 8375/A.T.C.C. 19424
<i>Haemophilus influenzae</i>	N.C.T.C. 8143
<i>Bacteroides fragilis</i>	N.C.T.C. 9343/A.T.C.C. 25285
<i>Bordetella pertussis</i>	N.C.T.C. 0739/A.T.C.C. 9797

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