Σ-Transwab®

Product Reference: MW167S, MW176S, MW176S3, MW177S, MW176HF/PF, MW177HF/PF, MW178HF/PF

Σ-Transwab® is a unique transport system for the safe and efficient transport of both aerobic and anaerobic bacteria. Each Σ-Transwab® unit consists of a sterile peel pack containing one, two or three Σ-swabs with cellular foam, Purflock® or Hydraflock® tips and a pre-labelled transport tube containing liquid Amies medium.

INSTRUCTIONS FOR USE

1. Before use always check that immediate packaging (peel pouch) is intact, that tube contains medium and there is no sign of leakage. In case of defect please do not use the device.
2. Peel back pouch, remove vial and place on a flat surface. Loosen cap.
3. Withdraw one Σ-swab, and use to take specimen.
4. Remove cap from vial, insert Σ-swab into vial and snap off the non-bud end so that the remaining shaft fits within the vial. The shaft has a scored breakpoint to assist this process.
5. If there are more Σ-swabs, repeat steps 2 and 3 for each swab. Each swab can be used to sample a different body site from the same patient.
6. Replace cap, and turn until secure.
7. Fill in patients details.
8. Transport to the laboratory immediately

Formula

Deionised water
Calcium chloride activated
Sodium Chloride
Potassium chloride
Magnesium chloride
Potassium di-hydrogen phosphate
Di-sodium hydrogen phosphate
Sodium thioglycollate
PH 7.3 +/- 0.3

Material safety information

Transwabs plastic components do not contain latex or PVC.

QC PERFORMANCE TESTS

Samples are tested to ensure that the minimal inocula of a spectrum of organisms are recovered after a specified duration of time. Suspensions of the following organisms are absorbed on to the swab of each product sample and placed in its respective medium.

<i>Neisseria gonorrhoea</i>	N.C.T.C. 8375/A.T.C.C. 19424
<i>Haemophilus influenzae</i>	N.C.T.C. 8143
<i>Bacteroides fragilis</i>	N.C.T.C. 9343/A.T.C.C. 25285
<i>Bordetella pertussis</i>	N.C.T.C. 0739/A.T.C.C 9797

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