BOITACOUCOU / COUCOUBOX

CAISSON DE FORMATION À LA TECHNIQUE DE FRICTION / RUBBING TECHNIQUE TRAINING BOX

NOTICE DE MONTAGE / ASSEMBLY INSTRUCTIONS

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Unpack the cardboard.

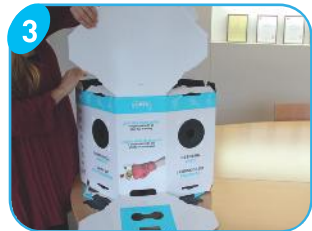
Content: 1 training box, 2 lamps, 6 batteries, 4 feet.

Assembly instructions are available below the box.

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For lower box: Fold face 4 then put strips into the notches (Face numbers are printed on each of them).

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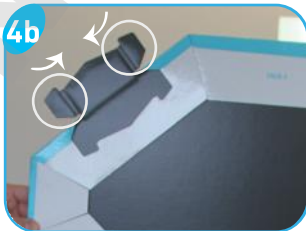
a) Put face 3 above face 4.

b) Fold the ends of strips then put them into the notches.

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For upper box: Repeat the same operation - fold face 2 then put strips into the notches. Fold the handle.

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Put face 1 above face 2, put the handle through the hole in the center. Fold the ends of strips then put them into the notches.

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Turn the box, and stick the plastic feet below using the guides.

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a) Insert batteries into lamps. Place lamps heads down while using the box.

b) After use, place lamps heads up in order not to fall during transport. Put ANIOSGEL 85 NPC FLUORESCENT bottle in the space above the box. COUCOUBOX is ready to use.

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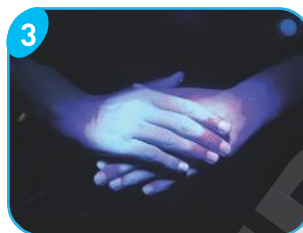
Take a sufficient amount of ANIOSGEL 85 NPC FLUORESCENT to keep your hands damp for 30 seconds.

Rub using the hygienic rubbing technique (protocol is printed on each side of the box).

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Analyse result with trainer in the double-viewing slot. LED lamps reveal areas where gel has been applied.

Do not hesitate to adjust hands position inside the box in order to find the better angle for viewing (up and down, right and left, nearest or farthest to you).

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Store in a clean and dry place. Do not spray any product directly on the cardboard. Use a slightly wet mop.

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- *In order to ensure an optimal viewing, we recommend to replace batteries every 2 hours by new, high quality batteries.*
- *Turn off lamps after each use.*