

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
B25848	CISEAUX KELLY POUR FISTULE COURBES L:180MM	3661219005186
	KELLY SCISSORS FOR FUSTULA CURVED L:180MM	 3 661219 005186

Catalogues / *catalogs*

Landanger 2015

Chapitre-page

03-048

DESCRIPTIF DU DISPOSITIF/ *PRODUCT DESCRIPTION*

LANDANGER

Ciseaux longs et fins - Long and fine scissors - Tijeras largas y finas

KELLY <table border="0"> <tr> <td></td> <td>Droit Straight Recta</td> <td>Courbe Curved Curva</td> </tr> <tr> <td>16 cm - 6 1/8"</td> <td>B258468</td> <td>B258478</td> </tr> <tr> <td>18 cm - 7"</td> <td>B25847</td> <td>B25848</td> </tr> <tr> <td colspan="3">Dentelées - Serrated - Serradas</td> </tr> <tr> <td>16 cm - 6 1/8"</td> <td>B258469</td> <td>B258479</td> </tr> </table>		Droit Straight Recta	Courbe Curved Curva	16 cm - 6 1/8"	B258468	B258478	18 cm - 7"	B25847	B25848	Dentelées - Serrated - Serradas			16 cm - 6 1/8"	B258469	B258479	STRULLY Courbe Curved Curva B258538 22,5 cm - 9"
	Droit Straight Recta	Courbe Curved Curva														
16 cm - 6 1/8"	B258468	B258478														
18 cm - 7"	B25847	B25848														
Dentelées - Serrated - Serradas																
16 cm - 6 1/8"	B258469	B258479														

Pour fistule
For fistula
Para fistula



B25847
18 cm - 7"



B25848



B258537
22,5 cm - 9"

III - 48

Copyright © LANDANGER - 2009 - 2015

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
B25848	CISEAUX KELLY POUR FISTULE COURBES L:180MM	3661219005186
	KELLY SCISSORS FOR FUSTULA CURVED L:180MM	 3 661219 005186

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

38727

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

ACIER INOXYDABLE / STAINLESS STEEL

Material type :

Nuance matière :

X39Cr13(1.4031)

Material grade :

Dimensions :

Voir désignations produits

Landanger 2015

Outer dimensions :

See products description

Chapitre-pages

03-048

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : *Standard conditions for reusable surgical instruments*

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).