

LANDANGER	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	--	---

Éditée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
B35040	CISEAUX MAYO STILLE LAMES RONDES COURBES L:170MM TUNGSTENE	3661219005599
	MAYO STILLE SCISSORS CURVED ROUND BLADE L:170MM TUNGSTEN	 3 661219 005599

Catalogues / catalogs

Landanger 2015

Chapitre-page

03-061

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

LANDANGER

Ciseaux tungstène - Tungsten scissors - Tijeras de tungsteno

MAYO-LEXER		LEXER-FINO		MAYO-STILLE	
<i>Courbe - Curved - Curva</i>					
 TC Insert Standard Standard Estándar	 Microcut	 SupercutBlack	 Standard Standard Estándar	 SupercutBlack	 Microcut
 B35008	 B35008M	 B35008SB	 B350095	 B350095SB	15 cm - 6" 17 cm - 6 3/4"
					 B35020  B35040  B35040M


B35008
16 cm - 6 1/4"


B350095
16 cm - 6 1/4"


B35020
15 cm - 6"

Lames rondes
Round blades
Hojas redondas

III - 61

Copyright © LANDANGER - 2009 - 2015

III - Ciseaux
III - Scissors
III - Tijeras

1/1

*Microcut : 1 lame TC standard + 1 lame TC microdentée - 1 TC standard blade + 1 TC serrated blade - 1 hoja TC estándar + 1 hoja TC micro-serrada
*SupercutBlack : 1 lame TC microdentée + 1 lame TC noire + Nitride de titane noir - 1 TC serrated blade + 1 TC black blade + Black coating - 1 hoja TC micro-serrada + 1 hoja TC negra + recubrimiento negro

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
B35040	CISEAUX MAYO STILLE LAMES RONDES COURBES L:170MM TUNGSTENE	3661219005599
	MAYO STILLE SCISSORS CURVED ROUND BLADE L:170MM TUNGSTEN	 3 661219 005599

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

38727

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

ACIER INOXYDABLE + TUNGSTENE / STAINLESS STEEL + TUNGSTEN

Material type :

Nuance matière :

X39Cr13(1.4031) + TUNGSTENE

Material grade :

Dimensions :

Voir désignations produits

Landanger 2015

Outer dimensions :

See products description

Chapitre-pages

03-061

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : *Standard conditions for reusable surgical instruments*

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).