

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
B406010	PINCE DISSECTION ADSON L:150MM DROITE SANS GRIFFES MORS TUNGSTENE	3661219334071
	ADSON DISSECTING FORCEPS L:150MM STRAIGHT WITHOUT TEETH, TUNGSTEN	 3 661219 334071

Catalogues / catalogs

Landanger 2015

Chapitre-page

02-047

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

L A N D A N G E R

Pinces à dissection mors tungstène - Dissecting forceps with tungsten jaws - Pinzas para disección boca de tungsteno

 TC Inserts		ADSON	MICRO ADSON		
 B40600 12 cm - 4 3/4"	 B406001 12 cm - 4 3/4"	 B406020 12 cm - 4 3/4"			
 B406010 15 cm - 6"	 B406011 15 cm - 6"	 B406021 15 cm - 6"			
Sans griffes Without teeth Sin dientes	1 x 2 griffes 1 x 2 teeth 1 x 2 dientes	1 x 2 griffes 1 x 2 teeth 1 x 2 dientes	SEMKEN	MC INDOE	
					
					
B40600 12 cm - 4 3/4"	B406001 12 cm - 4 3/4"	B406020 12 cm - 4 3/4"	B406950 15 cm - 6"	B406960 15 cm - 6"	
 1/1					

II - Dissection
II - Dissection
II - Dissection

II - 47

Copyright © LANDANGER - 2009 - 2015

LANDANGER	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	--	---

Éditée le / Publishing date

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
B406010	PINCE DISSECTION ADSON L:150MM DROITE SANS GRIFFES MORS TUNGSTENE	3661219334071
	ADSON DISSECTING FORCEPS L:150MM STRAIGHT WITHOUT TEETH, TUNGSTEN	 3 661219 334071

INFORMATIONS REGLEMENTAIRES / REGULATORY INFORMATION

Classe CE :
CE Class :

I

Code GMDN :
GMDN code :

62466

PERFORMANCES REVENDIQUEES / INTENDED PERFORMANCE

Dispositif médical de bloc opératoire :
Operating room medical device :

REUTILISABLE/REUSABLE

CARACTERISTIQUES SPECIFIQUE / SPECIFIC FEATURES

Type matière :
Material type :

ACIER INOXYDABLE + TUNGSTENE / STAINLESS STEEL + TUNGSTEN

Nuance matière :
Material grade :

X20Cr13(1.4021) + TUNGSTENE

Dimensions : Voir désignations produits
Outer dimensions : See products description
Chapitre-pages

Landanger 2015

02-047

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.
Cleaning / Pre-disinfection : Standard conditions for reusable surgical instruments

Sterilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / PACKAGING

Conditionnement : Pochette plastique en polyéthylène (PE).
Packaging : Polyethylene plastic pouch (PE).