

LANDANGER	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	--	---

Éditée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
B504584	PINCE HALSTEAD-MOSQUITO L:120MM COURBE MDLE ALLEMAND	3661219160694
	HALSTEAD-MOSQUITO FORCEPS L:120MM CURVED GERMAN MODEL	 3 661219 160694

Catalogues / catalogs

Landanger 2015

Chapitre-page

04-023

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

LANDANGER

Pinces hémostatiques - Haemostatic forceps - Pinzas hemostáticas

Droite Straight Recta		HALSTEAD-MOSQUITO	Courbe Curved Curva	
Sans griffes Without teeth Sin dientes	A griffes With teeth Con dientes		Sans griffes Without teeth Sin dientes	A griffes With teeth Con dientes
B504580	B504582	12 cm - 4 3/4"	B504584	B504586
B504581	B504583	14 cm - 5 1/2"	B504585	B504587
B504560	B504564	18 cm - 7"	B504562	B504566
 B504561	B504565	21 cm - 8 1/4"	 B504563	B504567

Modèles allemands
German models
Modelos alemanes



B504580
12 cm - 4 3/4"



B504581
14 cm - 5 1/2"



B504560
18 cm - 7"



B504565
21 cm - 8 1/4"



B504561
21 cm - 8 1/4"



B504584
12 cm - 4 3/4"



B504585
14 cm - 5 1/2"



B504562
18 cm - 7"



B504567
21 cm - 8 1/4"



B504563
21 cm - 8 1/4"

IV. Pinces
IV. Forceps
IV. Pinzas

IV - 23

4/5

Copyright © LANDANGER - 2009 - 2015

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
B504584	PINCE HALSTEAD-MOSQUITO L:120MM COURBE MDLE ALLEMAND	3661219160694
	HALSTEAD-MOSQUITO FORCEPS L:120MM CURVED GERMAN MODEL	 3 661219 160694

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

62468

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

ACIER INOXYDABLE / STAINLESS STEEL

Material type :

Nuance matière :

X20Cr13(1.4021)

Material grade :

Dimensions :

Voir désignations produits

Landanger 2015

Outer dimensions :

See products description

Chapitre-pages

04-023

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : *Standard conditions for reusable surgical instruments*

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).