

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
B80544	ECARTEUR BECKMANN L:140MM 4X4 GRIFFES MOUSSES P:8MM	3661219012672
	BECKMANN RETRACTOR L:140MM WITH 4 AND 4 BLUNT TEETH DEPTH 8MM	 3 661219 012672

Catalogues / catalogs

Landanger 2015

Chapitre-page

07-118

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

L A N D A N G E R

Écarteurs - Retractors - Separadores

BECKMANN CONE

Pointu Sharp Agudo	Mousse Blunt Romo	Dents Teeth Dientes
		
		P
14 cm - 5 ^{1/2} "	B805425 B80543	14 mm 3 x 4
17 cm - 6 ^{3/4} "	B805445 B80545	14 mm 3 x 4
21 cm - 8 ^{1/4} "	B805495 B80550	16 mm 3 x 4
26 cm - 10 ^{1/4} "	B805530 B805535	25 mm 3 x 4

BECKMANN

Mousse Blunt Romo	Dents Teeth Dientes
	
	P
14 cm - 5 ^{1/2} "	B80544 8 mm 4 x 4
17 cm - 6 ^{3/4} "	B80546 10 mm 4 x 4
21 cm - 8 ^{1/4} "	B80551 14 mm 4 x 4
21 cm - 8 ^{1/4} "	B80548 30 mm 4 x 4



B80550
21 cm - 8^{1/4}"



B80551
21 cm - 8^{1/4}"

P : Profondeur - Depth - Profundidad

VII - 118

 VII - Écarteurs
VII - Retractors
VII - Separadores

Copyright © LANDANGER - 2009 - 2015



LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
B80544	E CARTEUR BECKMANN L:140MM 4X4 GRIFFES MOUSSES P:8MM	3661219012672
	BECKMANN RETRACTOR L:140MM WITH 4 AND 4 BLUNT TEETH DEPTH 8MM	 3 661219 012672

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

45182

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

ACIER INOXYDABLE / STAINLESS STEEL

Material type :

Nuance matière :

X20Cr13(1.4021)

Material grade :

Dimensions :

Voir désignations produits

Landanger 2015

Outer dimensions :

See products description

Chapitre-pages

07-118

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : *Standard conditions for reusable surgical instruments*

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).