

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
B80670	VALVE MEDIANE RICARD P:65MM I:80MM	3661219013099
	RICARD CENTRAL BLADE WIDTH 80MM DEPTH 65MM	 3 661219 013099

Catalogues / catalogs

Landanger 2015

Chapitre-page

07-073

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

L A N D A N G E R

Écarteurs - Retractors - Separadores

RICARD

Valves latérales (à l'unité)
Lateral blades (per unit)
Valvas laterales (la unidad)



	P	L
 B80645	50 mm	60 mm
B80650	65 mm	80 mm
B80655	80 mm	100 mm
B80660	100 mm	120 mm

Compas seul
Compass only
Compas sólo



Ouverture maxi 300mm
Blade opening 11 3/4"
Abertura total 300 mm

B80640
18 cm - 7"

TOUPET

Valve médiane
Center blade
Valva mediana



	P	L
 B80665	50 mm	60 mm
B80670	65 mm	80 mm
B80675	80 mm	100 mm
B80680	100 mm	120 mm

Valves latérales (paire)
Lateral blades (pair)
Valvas laterales (par)



	P	L
 B80806	70 mm	40 mm
B80807	50 mm	40 mm

O : Ouverture - Blades opening - Abertura de las hojas
P : Profondeur - Depth - Profundidad
L : Largeur - Width - Anchura

Copyright © LANDANGER - 2009 - 2015

VII - Écarteurs
VII - Retractors
VII - Separadores

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
B80670	VALVE MEDIANE RICARD P:65MM I:80MM	3661219013099
	RICARD CENTRAL BLADE WIDTH 80MM DEPTH 65MM	 3 661219 013099

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

46774

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

ACIER INOXYDABLE / STAINLESS STEEL

Material type :

Nuance matière :

X2CrNi19-11(1.4306)

Material grade :

Dimensions :

Voir désignations produits

Landanger 2015

Outer dimensions :

See products description

Chapitre-pages

07-073

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : *Standard conditions for reusable surgical instruments*

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).