

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
DC12001-15	MICRO PINCE DE BAKLEY L:150MM COUDEE MANCHE ROND DIA. 8MM MORS 1,3MM 180GR	3661219125006
	DE BAKLEY MICRO TISSUE FORCEPS L:150MM ANG ROUND HANDLE DIA. 8MM JAWS 1,3MM 180GR	 3 661219 125006

Catalogues / catalogs

Delacroix 2020

Chapitre-page

01-003

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION



DeBailey



Delacroix-Chevalier
Paris

DeBailey Micro Tissue Forceps / Micro Pinces DeBailey / Micro Pinzas DeBailey

1.3 mm Jaws - 180 g Pressure
Mors 1,3 mm - Pression 180 g
Bocas 1,3 mm - Presión 180 g

DeBailey

1 - Dissection

Straight
Droite
Recta



- DC12000-12
L : 120 mm - 4 3/4"
- DC12000-15 
L : 150 mm - 6"
-  DC12000-18 
L : 180 mm - 7"
- DC12000-21 
L : 210 mm - 8"
- DC12000-23 
L : 230 mm - 9"



Angled
Coudée
Angulada



- DC12001-12
L : 120 mm - 4 3/4"
- DC12001-15 
L : 150 mm - 6"
-  DC12001-18 
L : 180 mm - 7"
- DC12001-21 
L : 210 mm - 8"
- DC12001-23 
L : 230 mm - 9"

 1/1

 Add TI for Titanium Model / Ajoutez TI pour le Modèle Titane / Añadir TI para Modelo Titánico

3

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
DC12001-15	MICRO PINCE DE BAKEY L:150MM COUDEE MANCHE ROND DIA. 8MM MORS 1,3MM 180GR	3661219125006
	DE BAKEY MICRO TISSUE FORCEPS L:150MM ANG ROUND HANDLE DIA. 8MM JAWS 1,3MM 180GR	 3 661219 125006

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :
CE Class :

I

Code GMDN :
GMDN code :

62466

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :
Operating room medical device :

REUTILISABLE/REUSABLE

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :
Material type :

ACIER INOXYDABLE / STAINLESS STEEL

Nuance matière :
Material grade :

X20Cr13(1.4021)

Dimensions : Voir désignations produits
Outer dimensions : See products description
Chapitre-pages

Delacroix 2020

01-003

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.
Cleaning / Pre-disinfection : Standard conditions for reusable surgical instruments

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement : Pochette plastique en polyéthylène (PE).
Packaging : Polyethylene plastic pouch (PE).