

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-29

Page

1

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
DC13040-21S	PINCE A DISSECTION SUPERGRIP MODELE FORT MORS DEBAKEY 3.5MM MANCHES ROUNDS DIA.10MM L:210MM 250GR	3661219400349
	<i>SUPERGRIP HEAVY STYLE FORCEPS 3.5MM DEBAKEY JAWS 10MM DIAMETER ROUND HANDLES L:210MM 250GR</i>	 3 661219 400349

Catalogues / catalogs

Delacroix 2020

Chapitre-page

01-016

DESCRIPTIF DU DISPOSITIF/ *PRODUCT DESCRIPTION*

1 - Dissection



SuperGrip Tying Micro Tissue Forceps / *Micro Pinces à Dissection SuperGrip* / *Micro Pinzas de disección SuperGrip*

Heavy Model
Modèle Fort
Modelo Fuerte


Resano


DeBakey


DeBakey


3.5 mm

DC13030-21S
L : 210 mm - 8"
DC13030-24S
L : 240 mm - 9 1/2"


3.5 mm

DC13040-21S
L : 210 mm - 8"
DC13040-24S
L : 240 mm - 9 1/2"





16

1/1

Copyright © LANDANGER - 2009 - 2020

LANDANGER SA – Rue Renée et Louis Landanger - Z.I. La Vendue - CS 52152 - 52906 CHAUMONT Cedex 9
Téléphone / Phone +33 (0)3 25 02 10 10 Télécopie / Fax +33 (0)3 25 02 10 20

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-29

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
DC13040-21S	PINCE A DISSECTION SUPERGRIP MODELE FORT MORS DEBAKEY 3.5MM MANCHES RONDS DIA.10MM L:210MM 250GR	3661219400349
	SUPERGRIP HEAVY STYLE FORCEPS 3.5MM DEBAKEY JAWS 10MM DIAMETER ROUND HANDLES L:210MM 250GR	 3 661219 400349

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

62466

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

ACIER INOXYDABLE + POLYMERE / STAINLESS STEEL + POLYMER

Material type :

Nuance matière :

X20Cr13(1.4021) + POLYPROPYLENE

Material grade :

Dimensions :

Voir désignations produits

Delacroix 2020

Outer dimensions :

See products description

Chapitre-pages

01-016

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : *Standard conditions for reusable surgical instruments*

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).