

|           |                                                                  |                                                        |
|-----------|------------------------------------------------------------------|--------------------------------------------------------|
| LANDANGER | <b>FICHE TECHNIQUE PRODUIT</b><br><b>PRODUCT TECHNICAL SHEET</b> | <b>FAQ 026 INFO</b><br>Indice : 2<br>Date : 01/06/2018 |
|-----------|------------------------------------------------------------------|--------------------------------------------------------|

Editée le / Publishing date

2022-04-28

Page

1

| REFERENCE  | DESIGNATIONS - DESCRIPTIONS                                   | CODE EAN                                                                                            |
|------------|---------------------------------------------------------------|-----------------------------------------------------------------------------------------------------|
| DC20145-18 | MICRO CISEAUX STREAMLINE LAMES 10MM L:180MM 90° MANCHE ROND   | 3661219127673                                                                                       |
|            | STREAMLINE MICRO SCISSORS L:180MM 90° ROUND HANDLE BLADE 10MM |  3 661219 127673 |

Catalogues / catalogs

Delacroix 2020

Chapitre-page

02-009

### DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION



Delacroix-Chevalier  
Paris


Jacobson Micro Scissors / Micro Ciseaux Jacobson / Micro Tijeras Jacobson

10 mm Blades - Streamline - Round Handle  
Lames 10 mm - Streamline - Manche Rond  
Lamas 10 mm - Streamline - Mango Redondo



Streamline

Jacobson

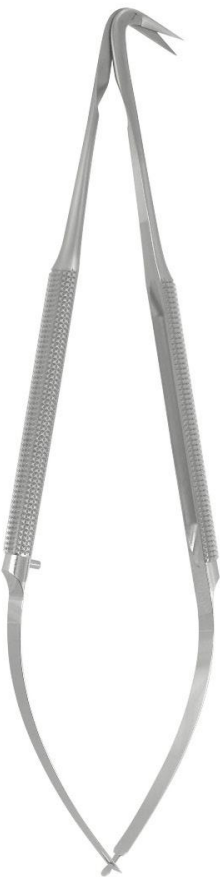

2-Scissors

25°



**DC20142-18**  
L : 180 mm - 7"

90°



**DC20145-18**  
L : 180 mm - 7"

45°



**DC20143-18**  
L : 180 mm - 7"

125°



**DC20146-18**  
L : 180 mm - 7"

60°



**DC20144-18**  
L : 180 mm - 7"

 1/1

9

|           |                                                    |                                                 |
|-----------|----------------------------------------------------|-------------------------------------------------|
| LANDANGER | FICHE TECHNIQUE PRODUIT<br>PRODUCT TECHNICAL SHEET | FAQ 026 INFO<br>Indice : 2<br>Date : 01/06/2018 |
|-----------|----------------------------------------------------|-------------------------------------------------|

Éditée le / Publishing date

2022-04-28

Page

2

| REFERENCE  | DESIGNATIONS - DESCRIPTIONS                                   | CODE EAN                                                                                               |
|------------|---------------------------------------------------------------|--------------------------------------------------------------------------------------------------------|
| DC20145-18 | MICRO CISEAUX STREAMLINE LAMES 10MM L:180MM 90° MANCHE ROND   | 3661219127673                                                                                          |
|            | STREAMLINE MICRO SCISSORS L:180MM 90° ROUND HANDLE BLADE 10MM | <br>3 661219 127673 |

### INFORMATIONS REGLEMENTAIRES / REGULATORY INFORMATION

Classe CE :  
CE Class :

I

Code GMDN :  
GMDN code :

38727

### PERFORMANCES REVENDIQUEES / INTENDED PERFORMANCE

Dispositif médical de bloc opératoire :  
Operating room medical device :

REUTILISABLE/REUSABLE

### CARACTERISTIQUES SPECIFIQUE / SPECIFIC FEATURES

Type matière :  
Material type :

ACIER INOXYDABLE / STAINLESS STEEL

Nuance matière :  
Material grade :

X46Cr13(1.4034)

Dimensions : Voir désignations produits  
Outer dimensions : See products description  
Chapitre-pages

Delacroix 2020

02-009

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.  
Cleaning / Pre-disinfection : Standard conditions for reusable surgical instruments

Sterilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

| Méthode de stérilisation à la vapeur saturée | Température     | Durée      | Temps de séchage | Pression absolue            |
|----------------------------------------------|-----------------|------------|------------------|-----------------------------|
| Pré-vide                                     | 132 °C (270 °F) | 4 minutes  | 30 minutes       | 2,9 bars, 2,8 atm, 41,6 psi |
| Pré-vide                                     | 134 °C (273 °F) | 3 minutes  | 35 minutes       | 3,0 bars, 3,0 atm, 44,1 psi |
| Pré-vide                                     | 134 °C (273 °F) | 5 minutes  | 35 minutes       | 3,0 bars, 3,0 atm, 44,1 psi |
| Pré-vide                                     | 134 °C (273 °F) | 18 minutes | 35 minutes       | 3,0 bars, 3,0 atm, 44,1 psi |

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

| Saturated steam sterilization method | Temperature     | Time       | Drying time | Absolute pressure          |
|--------------------------------------|-----------------|------------|-------------|----------------------------|
| Pre-vacuum                           | 132 °C (270 °F) | 4 minutes  | 30 minutes  | 2.9 bar, 2.8 atm, 41.6 psi |
| Pre-vacuum                           | 134 °C (273 °F) | 3 minutes  | 35 minutes  | 3.0 bar, 3.0 atm, 44.1 psi |
| Pre-vacuum                           | 134 °C (273 °F) | 5 minutes  | 35 minutes  | 3.0 bar, 3.0 atm, 44.1 psi |
| Pre-vacuum                           | 134 °C (273 °F) | 18 minutes | 35 minutes  | 3.0 bar, 3.0 atm, 44.1 psi |

### EMBALLAGE / PACKAGING

Conditionnement : Pochette plastique en polyéthylène (PE).  
Packaging : Polyethylene plastic pouch (PE).