

LANDANGER	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	--	--

Editée le / Publishing date

2022-04-29

Page

1

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
DC20163-15	MICRO CISEAUX BALL TYPE 1MM LAMES 7MM L:150MM 90° MANCHE ROND	3661219393603
	BALL TYPE 1MM MICRO SCISSORS 7MM BLADES L:150MM 90° ROUND HANDLE	 3 661219 393603

Catalogues / catalogs

Delacroix 2020

Chapitre-page

02-014

DESCRIPTIF DU DISPOSITIF/ *PRODUCT DESCRIPTION*

2 - Scissors



Ball Type Micro Scissors / *Micro Ciseaux à Extrémité Sphérique* / *Micro Tijeras con Extremidad Esférica*

7 mm Blades - Ball 1 mm - Round Handle
Lames 7 mm - Sphère 1 mm - Manche Rond
Lamas 7 mm - Esfera 1 mm - Mango Redondo

25°

DC20160-12
L : 120 mm - 4 ^{3/4}"

DC20160-15 NEW
NOUVEAU
NUEVO
L : 150 mm - 6"

DC20160-18
L : 180 mm - 7"

DC20160-20
L : 210 mm - 8"

90°

DC20163-12
L : 120 mm - 4 ^{3/4}"

DC20163-15 NEW
NOUVEAU
NUEVO
L : 150 mm - 6"

DC20163-18
L : 180 mm - 7"

DC20163-20
L : 210 mm - 8"

45°

DC20161-12
L : 120 mm - 4 ^{3/4}"

DC20161-15 NEW
NOUVEAU
NUEVO
L : 150 mm - 6"

DC20161-18
L : 180 mm - 7"

DC20161-20
L : 210 mm - 8"

125°

DC20164-12
L : 120 mm - 4 ^{3/4}"

DC20164-15 NEW
NOUVEAU
NUEVO
L : 150 mm - 6"

DC20164-18
L : 180 mm - 7"

DC20164-20
L : 210 mm - 8"

60°

DC20162-12
L : 120 mm - 4 ^{3/4}"

DC20162-15 NEW
NOUVEAU
NUEVO
L : 150 mm - 6"

NEW
NOUVEAU
NUEVO DC20162-18
L : 180 mm - 7"

DC20162-20
L : 210 mm - 8"



14

 1/1
Copyright © LANDANGER - 2009 - 2020

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-29

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
DC20163-15	MICRO CISEAUX BALL TYPE 1MM LAMES 7MM L:150MM 90° MANCHE ROND	3661219393603
	BALL TYPE 1MM MICRO SCISSORS 7MM BLADES L:150MM 90° ROUND HANDLE	 3 661219 393603

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

38727

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

ACIER INOXYDABLE / STAINLESS STEEL

Material type :

Nuance matière :

X46Cr13(1.4034)

Material grade :

Dimensions :

Voir désignations produits

Delacroix 2020

Outer dimensions :

See products description

Chapitre-pages

02-014

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : *Standard conditions for reusable surgical instruments*

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).