

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
DC21275-00	CISEAUX CARDIOVASC DE MARTEL L:180MM COUDE POINTU	3661219128649
	DE MARTEL CARDIOVASCULAR SCISSORS L:180MM ANG SHARP	 3 661219 128649

Catalogues / catalogs

Delacroix 2020

Chapitre-page

02-031

DESCRIPTIF DU DISPOSITIF/ *PRODUCT DESCRIPTION*



De Martel Scissors / Ciseaux De Martel / Tijeras De Martel

Straight - Blunt
Droit - Mousse
Recta - Desafilada



DC21273-14
L: 140 mm - 5 1/2"

DC21273-17
L: 175 mm - 6 3/4"

Curved - Sharp
Courbe - Pointu
Curva - Aguda



DC21276-14
L: 140 mm - 5 1/2"

DC21276-17
L: 170 mm - 6 3/4"

Curved - Blunt
Courbe - Mousse
Curva - Desafilada



DC21272-14
L: 140 mm - 5 1/2"

 **DC21272-17**
L: 170 mm - 6 3/4"

De Martel



Angled - Sharp
Coudé - Pointu
Angulada - Aguda



DC21275-14
L: 140 mm - 5 1/2"

DC21275-00
L: 180 mm - 7"

Angled - Sharp
Coudé - Pointu
Angulada - Aguda



DC21270-14
L: 140 mm - 5 1/2"

DC21270-00
L: 180 mm - 7"

Angled - Blunt
Coudé - Mousse
Angulada - Desafilada



DC21274-14
L: 140 mm - 5 1/2"

DC21274-00
L: 180 mm - 7"

2 - Scissors

1/1

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
DC21275-00	CISEAUX CARDIOVASC DE MARTEL L:180MM COUDE POINTU	3661219128649
	DE MARTEL CARDIOVASCULAR SCISSORS L:180MM ANG SHARP	 3 661219 128649

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :
CE Class :

I

Code GMDN :
GMDN code :

38727

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :
Operating room medical device :

REUTILISABLE/REUSABLE

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :
Material type :

ACIER INOXYDABLE / STAINLESS STEEL

Nuance matière :
Material grade :

X38CrMoV15(1.4117)

Dimensions : Voir désignations produits
Outer dimensions : See products description
Chapitre-pages

Delacroix 2020

02-031

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.
Cleaning / Pre-disinfection : Standard conditions for reusable surgical instruments

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement : Pochette plastique en polyéthylène (PE).
Packaging : Polyethylene plastic pouch (PE).