

LANDANGER	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	--	---

Éditée le / Publishing date

2022-04-29

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
DC41106-23	CLAMP BULLDOG DE BAKEY MORS 23MM L:80MM COURBE	3661219265641
	DE BAKEY BULLDOG CLAMP JAWS 23MM L:80MM CURVED	 3 661219 265641

Catalogues / catalogs

Delacroix 2020

Chapitre-page

04.1-012

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

4.1 - Small Clamps



DeBakey Bulldog Clamps / Clamps Bulldog DeBakey / Clamps Bulldog DeBakey

Pressure / Pression / Presión : 250 g

DeBakey



Straight
Droit
Recta



Curved
Courbe
Curva

DC41100-23 

L : 80 mm - 3 1/8"

Jaws / Mors / Bocas : 23 mm

DC41101-29 

L : 88 mm - 3 1/2"

Jaws / Mors / Bocas : 29 mm

DC41102-50 

L : 110 mm - 4 1/4"

Jaws / Mors / Bocas : 50 mm

 **DC41103-66** 

L : 125 mm - 4 7/8"

Jaws / Mors / Bocas : 66 mm

DC41104-70 

L : 130 mm - 5 1/8"

Jaws / Mors / Bocas : 70 mm

 Add TI for Titanium Model / Ajoutez TI pour le Modèle Titane / Añadir TI para Modelo Titánico

DC41105-21

L : 77 mm - 3"

Jaws / Mors / Bocas : 21 mm

DC41106-23 

L : 80 mm - 3 1/8"

Jaws / Mors / Bocas : 23 mm

DC41107-32 

L : 90 mm - 3 1/2"

Jaws / Mors / Bocas : 32 mm

DC41108-49 

L : 105 mm - 4 1/8"

Jaws / Mors / Bocas : 49 mm

 **DC41109-64** 

L : 123 mm - 4 7/8"

Jaws / Mors / Bocas : 64 mm

12

Copyright LANDANGER - 2009 - 2020

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-29

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
DC41106-23	CLAMP BULLDOG DE BAKEY MORS 23MM L:80MM COURBE	3661219265641
	DE BAKEY BULLDOG CLAMP JAWS 23MM L:80MM CURVED	 3 661219 265641

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :
CE Class :

I

Code GMDN :
GMDN code :

62470

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :
Operating room medical device :

REUTILISABLE/REUSABLE

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :
Material type :

ACIER INOXYDABLE / STAINLESS STEEL

Nuance matière :
Material grade :

X20Cr13(1.4021)

Dimensions : Voir désignations produits
Outer dimensions : See products description
Chapitre-pages

Delacroix 2020

04.1-012

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.
Cleaning / Pre-disinfection : Standard conditions for reusable surgical instruments

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement : Pochette plastique en polyéthylène (PE).
Packaging : Polyethylene plastic pouch (PE).