

LANDANGER	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	--	---

Éditée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
DC41500-03TI	CLAMP BULLDOG ATRAUMA L:30MM DROIT POUR ARTERE TITANE	3661219135838
	ATRAUMATIC BULLDOG CLAMP L:30MM STRAIGHT FOR ARTERY TI	

Catalogues / catalogs

Delacroix 2020

Chapitre-page

04.1-018

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

4.1 - Small Clamps



Atraumatic Bulldog Clamps / Clamps Bulldog Atraumatiques / Clamps Bulldog Atraumáticos

Titanium Mini Glover / Mini Glover Titane / Mini Glover Titanio

Pressure / Pression / Presión : 400 g



Artery
Artères
Arterias

DC41500-03TI
L : 30 mm - 1 1/8"
DC41502-03TI
L : 35 mm - 1 1/4"
DC41504-04TI
L : 40 mm - 1 3/5"
DC41506-05TI
L : 50 mm - 2"



Artery
Artères
Arterias

DC41510-03TI
L : 30 mm - 1 1/8"
DC41512-03TI
L : 35 mm - 1 1/4"
DC41514-04TI
L : 40 mm - 1 3/5"
DC41516-05TI
L : 50 mm - 2"

Pressure / Pression / Presión : 200 g



Vein
Veines
Venas

DC41501-03TI
L : 30 mm - 1 1/8"
DC41503-03TI
L : 35 mm - 1 1/4"
DC41505-04TI
L : 40 mm - 1 3/5"
DC41507-05TI
L : 50 mm - 2"



Vein
Veines
Venas

DC41511-03TI
L : 30 mm - 1 1/8"
DC41513-03TI
L : 35 mm - 1 1/4"
DC41515-04TI
L : 40 mm - 1 3/5"
DC41517-05TI
L : 50 mm - 2"

Copyright LANDANGER - 2009 - 2020

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
DC41500-03TI	CLAMP BULLDOG ATRAUMA L:30MM DROIT POUR ARTERE TITANE	3661219135838
	ATRAUMATIC BULLDOG CLAMP L:30MM STRAIGHT FOR ARTERY TI	 3 661219 135838

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

62470

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

TITANE / TITANIUM

Material type :

Nuance matière :

TA6V(TITANE)

Material grade :

Dimensions :

Voir désignations produits

Delacroix 2020

Outer dimensions :

See products description

Chapitre-pages

04.1-018

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : *Standard conditions for reusable surgical instruments*

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).