

LANDANGER	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	--	---

Éditée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
DC42011-00	MICRO CLAMP VASC NEONATAL DE BAKEY CASTANEDA L:120MM BRANCHES DROITES MORS DOUBLEMENT COUDES MORS:1,4MM	3661219136422
	DE BAKEY CASTANEDA NEONATAL MICRO VASCULAR CLAMP L:120MM STRAIGHT SHANKS DBLE ANG JAWS JAWS:1,4MM	 3 661219 136422

Catalogues / catalogs

Delacroix 2020

Chapitre-page

04.1-031

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION



Castaneda Clamps / Clamps Castaneda / Clamps Castaneda

Straight Shanks / Branches Droites / Bocas Rectas

No Scale / Pas d'Echelle / Sin Escala




DeBakey

Castaneda


DC42000-00
 L : 120 mm - 4 4/5"


DC42001-30
 L : 120 mm - 4 4/5"


DC42002-45
 L : 125 mm - 5"


DC42003-60
 L : 110 mm - 4 2/5"


DC42004-90
 L : 105 mm - 4 1/5"


DC42005-00
 L : 120 mm - 4 4/5"


DC42006-00
 L : 120 mm - 4 4/5"


DC42007-00
 L : 120 mm - 4 4/5"


DC42008-00
 L : 115 mm - 4 3/5"


DC42009-00
 L : 110 mm - 4 2/5"


DC42010-00
 L : 115 mm - 4 3/5"


DC42011-00
 L : 120 mm - 4 4/5"


DC42012-00
 L : 130 mm - 5 1/5"


DC42013-00
 L : 115 mm - 4 3/5"


DC42014-00
 L : 110 mm - 4 2/5"


DC42015-00
 L : 120 mm - 4 4/5"


DC42016-00
 L : 120 mm - 4 4/5"


DC42017-00
 L : 120 mm - 4 4/5"



31

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
DC42011-00	MICRO CLAMP VASC NEONATAL DE BAKEY CASTANEDA L:120MM BRANCHES DROITES MORS DOUBLEMENT COUDES MORS:1,4MM	3661219136422
	DE BAKEY CASTANEDA NEONATAL MICRO VASCULAR CLAMP L:120MM STRAIGHT SHANKS DBLE ANG JAWS JAWS:1,4MM	 3 661219 136422

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

62468

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

ACIER INOXYDABLE / STAINLESS STEEL

Material type :

Nuance matière :

X20Cr13(1.4021)

Material grade :

Dimensions :

Voir désignations produits

Delacroix 2020

Outer dimensions :

See products description

Chapitre-pages

04.1-031

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : *Standard conditions for reusable surgical instruments*

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).