

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
DC50715-25	MICRO PORTE AIGUILLE STREAMLINE DESIGN L:250MM MANCHE ROND D:9MM AVEC CREMAILLERE MORS COURBES 1:1,2MM DIAMONITE	3661219390435
	MICRO N H STREAMLINE DESIGN L:250MM ROUND HANDLE D:9MM W/ RATCHET CURVED JAWS 1:1,2MM DIAMONITE	

Catalogues / catalogs

Delacroix 2020

Chapitre-page

05-021

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION



Micro Needle Holders / Micro Porte-Aiguilles / Micro Porta Agujas



Diamonite



Streamline

Width: 1.2 mm - For Suture: 7/0 -> 5/0 - Round Handle
Largeur: 1.2 mm - Pour Suture: 7/0 -> 5/0 - Manche Rond
Anchura: 1.2 mm - Para Sutura: 7/0 -> 5/0 - Mango Redondo





Straight - Without Ratchet
Droit - Sans Verrou
Recta - Sin Cerrojo

- DC50704-18 
L: 180 mm - 7"
- DC50704-21 
L: 210 mm - 8"
- DC50704-23 
L: 230 mm - 9"
- DC50704-25 
L: 250 mm - 10"

Curved - Without Ratchet
Courbe - Sans Verrou
Curva - Sin Cerrojo

- DC50714-18 
L: 180 mm - 7"
- DC50714-21 
L: 210 mm - 8"
- DC50714-23 
L: 230 mm - 9"
- DC50714-25 
L: 250 mm - 10"

Straight - With Ratchet
Droit - Avec Verrou
Recta - Con Cerrojo

- DC50705-18 
L: 180 mm - 7"
-  DC50705-21 
L: 210 mm - 8"
- DC50705-23 
L: 230 mm - 9"
- DC50705-25 
L: 250 mm - 10"

Curved - With Ratchet
Courbe - Avec Verrou
Curva - Con Cerrojo

- DC50715-18 
L: 180 mm - 7"
- DC50715-21 
L: 210 mm - 8"
- DC50715-23 
L: 230 mm - 9"
- DC50715-25 
L: 250 mm - 10"

 Add TI for Titanium Model / Ajoutez TI pour le Modèle Titane /
Añadir TI para Modelo Titánico



21

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
DC50715-25	MICRO PORTE AIGUILLE STREAMLINE DESIGN L:250MM MANCHE ROND D:9MM AVEC CREMAILLE MORS COURBES l:1,2MM DIAMONITE	3661219390435
	MICRO N H STREAMLINE DESIGN L:250MM ROUND HANDLE D:9MM W/ RATCHET CURVED JAWS l:1,2MM DIAMONITE	 3 661219 390435

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :
CE Class :

I

Code GMDN :
GMDN code :

12726

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :
Operating room medical device :

REUTILISABLE/REUSABLE

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :
Material type :

ACIER INOXYDABLE + TUNGSTENE / STAINLESS STEEL + TUNGSTEN

Nuance matière :
Material grade :

X12Cr13(1.4006) + TUNGSTENE

Dimensions : Voir désignations produits
Outer dimensions : See products description
Chapitre-pages

Delacroix 2020

05-021

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.
Cleaning / Pre-disinfection : Standard conditions for reusable surgical instruments

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement : Pochette plastique en polyéthylène (PE).
Packaging : Polyethylene plastic pouch (PE).