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REFERENCES / <i>REFERENCES</i>	DESIGNATIONS / DESIGNATIONS	
HF5718110	CISEAU BIPOLAIRE BITECH COURBE LAME STANDARD L:180MM	BITECH BIPOLAR SCISSOR 180 MM CURVED, ROUND PLUG
HF5718120	CISEAU BIPOLAIRE BITECH COURBE LAME EFFILEE L:180MM	BITECH BIPOLAR SCISSOR 180 MM FINE DELICATE BLADE, CURVED, ROUND PLUG
HF5721110	CISEAU BIPOLAIRE BITECH COURBE LAME STANDARD L:210MM	BITECH BIPOLAR SCISSOR 210 MM CURVED, ROUND PLUG
HF5721120	CISEAU BIPOLAIRE BITECH COURBE LAME EFFILEE L:210MM	BITECH BIPOLAR SCISSOR 210 MM FINE DELICATE BLADE, CURVED, ROUND PLUG
HF5723110	CISEAU BIPOLAIRE BITECH COURBE LAME STANDARD L:230MM	BITECH BIPOLAR SCISSOR 230 MM CURVED, ROUND PLUG
HF5723120	CISEAU BIPOLAIRE BITECH COURBE LAME EFFILEE L:230MM	BITECH BIPOLAR SCISSOR 230 MM FINE DELICATE BLADE, CURVED, ROUND PLUG
HF5728110	CISEAU BIPOLAIRE BITECH COURBE LAME STANDARD L:280MM	BITECH BIPOLAR SCISSOR 280 MM CURVED, ROUND PLUG
HF5728120	CISEAU BIPOLAIRE BITECH COURBE LAME EFFILEE L:280MM	BITECH BIPOLAR SCISSOR 280 MM FINE DELICATE BLADE, CURVED, ROUND PLUG

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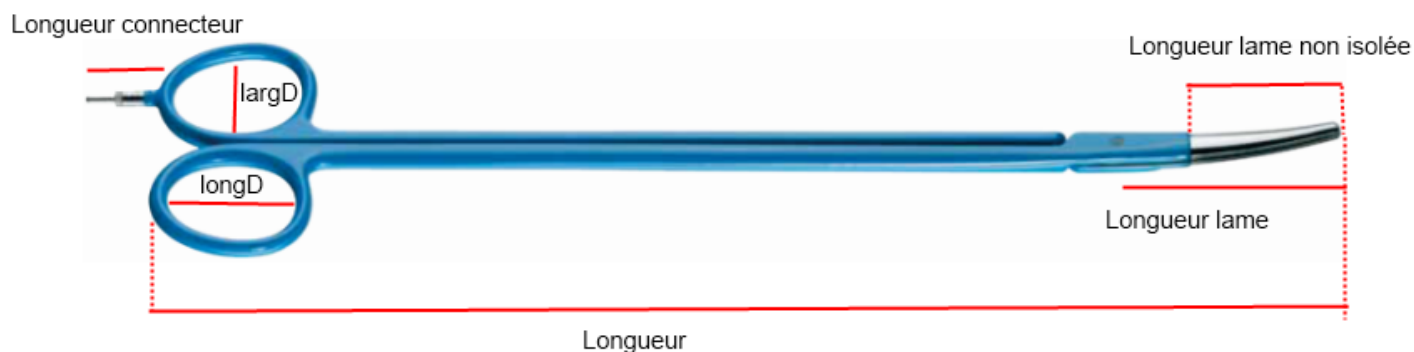
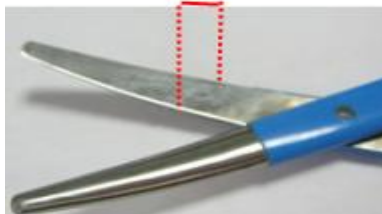
➤ Nettoyage, pré-désinfection / Cleaning, pre-disinfection

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The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

<i>Saturated steam sterilization method</i>	<i>Temperature</i>	<i>Time</i>	<i>Drying time</i>	<i>Absolute pressure</i>
<i>Pre-vacuum</i>	<i>132 °C (270 °F)</i>	<i>4 minutes</i>	<i>30 minutes</i>	<i>2.9 bar, 2.8 atm, 41.6 psi</i>
<i>Pre-vacuum</i>	<i>134 °C (273 °F)</i>	<i>3 minutes</i>	<i>35 minutes</i>	<i>3.0 bar, 3.0 atm, 44.1 psi</i>
<i>Pre-vacuum</i>	<i>134 °C (273 °F)</i>	<i>5 minutes</i>	<i>35 minutes</i>	<i>3.0 bar, 3.0 atm, 44.1 psi</i>
<i>Pre-vacuum</i>	<i>134 °C (273 °F)</i>	<i>18 minutes</i>	<i>35 minutes</i>	<i>3.0 bar, 3.0 atm, 44.1 psi</i>

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