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surgical instruments dedicated to conduct fluids in or out of the body during surgical procedures

Reusable instrumentation for open surgery. The suction cannula is usable to suck up blood, physiological serum, sterile water and tissue debris depending on the diameter of the cannula used. It provides a good visibility to the surgeon. The cannula can be used to inject liquids into the body

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The estimated life expectancy for surgical instruments made entirely of metal is 10 years.

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Store in a clean and dry place.

➤ Nettoyage, pré-désinfection / Cleaning, pre-disinfection

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Standard conditions for reusable surgical instruments.

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The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

<i>Saturated steam sterilization method</i>	<i>Temperature</i>	<i>Time</i>	<i>Drying time</i>	<i>Absolute pressure</i>
<i>Pre-vacuum</i>	<i>132 °C (270 °F)</i>	<i>4 minutes</i>	<i>30 minutes</i>	<i>2.9 bar, 2.8 atm, 41.6 psi</i>
<i>Pre-vacuum</i>	<i>134 °C (273 °F)</i>	<i>3 minutes</i>	<i>35 minutes</i>	<i>3.0 bar, 3.0 atm, 44.1 psi</i>
<i>Pre-vacuum</i>	<i>134 °C (273 °F)</i>	<i>5 minutes</i>	<i>35 minutes</i>	<i>3.0 bar, 3.0 atm, 44.1 psi</i>
<i>Pre-vacuum</i>	<i>134 °C (273 °F)</i>	<i>18 minutes</i>	<i>35 minutes</i>	<i>3.0 bar, 3.0 atm, 44.1 psi</i>

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