

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
I150886	PINCE DISSECTION OPHTALMO L:105MM COURBE 1X2 GRIFFES FINE	3661219344179
	OPHTHALMOLOGY DISSECTING FORCEPS L:105MM CURVED 1X2 TEETH, FINE JAWS	 3 661219 344179

Catalogues / catalogs

Landanger 2015

Chapitre-page

02-036

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

II - Dissection
II - Dissection
II - Dissection

L A N D A N G E R

Pinces à dissection ophtalmologiques - Ophthalmologic dissecting forceps - Pinzas para disección oftalmológicas

Copyright © LANDANGER - 2009 - 2015



Sans griffes
Without teeth
Sin dientes

I150880
10,5 cm - 4 1/4"



Sans griffes
Without teeth
Sin dientes

I150881
10,5 cm - 4 1/4"



Sans griffes
Without teeth
Sin dientes

I150882
10,5 cm - 4 1/4"



1 x 2 griffes
1 x 2 teeth
1 x 2 dientes

I150885
10,5 cm - 4 1/4"



1 x 2 griffes
1 x 2 teeth
1 x 2 dientes

I150886
10,5 cm - 4 1/4"



1 x 2 griffes
1 x 2 teeth
1 x 2 dientes

I150887
10,5 cm - 4 1/4"



Sans griffes
Mors 8/10
Without teeth
8/10 jaws
Sin dientes
Bocas 8/10

I150890
10 cm - 4"



Sans griffes
Without teeth
Sin dientes

I150895
10 cm - 4"



Sans griffes
Without teeth
Sin dientes

B40840
11 cm - 4 1/4"



Sans griffes
Without teeth
Sin dientes

I15120
11 cm - 4 1/4"



1 x 2 griffes
1 x 2 teeth
1 x 2 dientes

B40830
11 cm - 4 1/4"



1 x 2 griffes
1 x 2 teeth
1 x 2 dientes

I15110
11 cm - 4 1/4"

II - 36

 1/1

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
I150886	PINCE DISSECTION OPHTALMO L:105MM COURBE 1X2 GRIFFES FINE	3661219344179
	OPHTHALMOLOGY DISSECTING FORCEPS L:105MM CURVED 1X2 TEETH, FINE JAWS	 3 661219 344179

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

62466

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

ACIER INOXYDABLE / STAINLESS STEEL

Material type :

Nuance matière :

X20Cr13(1.4021)

Material grade :

Dimensions :

Voir désignations produits

Landanger 2015

Outer dimensions :

See products description

Chapitre-pages

02-036

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : *Standard conditions for reusable surgical instruments*

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).