

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
J17001	PINCE RONGEUR CUSHING L:130MM MORS DROITS 2X10MM	3661219075967
	CUSHING LAMINECTOMY RONGEUR L:130MM STRAIGHT JAWS 2X10MM	

Catalogues / catalogs

Landanger 2015

Chapitre-page

11-011

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

L A N D A N G E R

Neurochirurgie : pinces rongeurs - Neurosurgery : laminectomy rongeurs - Neurocirugía : pinzas para laminectomía



J170160



Droite
Straight
Recta



Vers le haut
Upward
Hacia arriba



Vers le bas
Downward
Hacia abajo

	Largeur de mors Width of jaw Anchura de boca	Longueur tige Length of shaft Largo del tallo			
CUSHING					
	2 x 10 mm	13 cm - 5 1/8"	J17001	J17002	J17003
		15 cm - 6"	J170040	J170041	J170042
		18 cm - 7"	J17005	J17006	J17007
		23 cm - 9"	J170043	J170044	-
		28 cm - 11"	J170046	J170047	-
LOVE-GRUENWALD					
		13 cm - 5 1/8"	J17010	J17012	J17015
	3 x 10 mm	15 cm - 6"	J170160	J170161	J170162
		18 cm - 7"	J17020	J17025	J17035
		23 cm - 9"	J170163	J170164	-
		28 cm - 11"	J170166	J170167	-
SPURLING					
		13 cm - 5 1/8"	J17045	J17050	J17055
		15 cm - 6"	J170560	J170561	J170562
		18 cm - 7"	J17060	J17065	J17070
		23 cm - 9"	J170563	J170564	-
		28 cm - 11"	J170566	J170567	-

Copyright © LANDANGER - 2009 - 2015

 2/3

XI - 11

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
J17001	PINCE RONGEUR CUSHING L:130MM MORS DROITS 2X10MM	3661219075967
	CUSHING LAMINECTOMY RONGEUR L:130MM STRAIGHT JAWS 2X10MM	 3 661219 075967

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :
CE Class :

I

Code GMDN :
GMDN code :

35541

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :
Operating room medical device :

REUTILISABLE/REUSABLE

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :
Material type :

ACIER INOXYDABLE / STAINLESS STEEL

Nuance matière :
Material grade :

X20Cr13(1.4021) + X8CrNiS18-9(1.4305)

Dimensions :
Outer dimensions :

Voir désignations produits
See products description
Chapitre-pages

Landanger 2015

11-011

Nettoyage/Pré-désinfection :
Cleaning / Pre-disinfection :

Standard pour instrumentation de bloc opératoire réutilisable.
Standard conditions for reusable surgical instruments

Sterilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :
Packaging :

Pochette plastique en polyéthylène (PE).
Polyethylene plastic pouch (PE).