

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
J17007	PINCE RONGEUR CUSHING L:180MM MORS COUDES 30° BAS 2X10MM	3661219076018
	CUSHING LAMINECTOMY RONGEUR L:180MM ANGLED JAWS DOWNWARD 30° 2X10MM	

Catalogues / catalogs

Landanger 2015

Chapitre-page

11-011

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

L A N D A N G E R

Neurochirurgie : pinces rongeurs - Neurosurgery : laminectomy rongeurs - Neurocirugía : pinzas para laminectomía



J170160

Largueur de mors Width of jaw Anchura de boca	Longueur tige Length of shaft Largo del tallo	Droite Straight Recta	Vers le haut Upward Hacia arriba	Vers le bas Downward Hacia abajo
---	---	-----------------------------	--	--

CUSHING



2 x 10 mm

13 cm - 5 1/8"	J17001	J17002	J17003
15 cm - 6"	J170040	J170041	J170042
18 cm - 7"	J17005	J17006	J17007
23 cm - 9"	J170043	J170044	-
28 cm - 11"	J170046	J170047	-

LOVE-GRUENWALD



3 x 10 mm

13 cm - 5 1/8"	J17010	J17012	J17015
15 cm - 6"	J170160	J170161	J170162
18 cm - 7"	J17020	J17025	J17035
23 cm - 9"	J170163	J170164	-
28 cm - 11"	J170166	J170167	-

SPURLING



4 x 10 mm

13 cm - 5 1/8"	J17045	J17050	J17055
15 cm - 6"	J170560	J170561	J170562
18 cm - 7"	J17060	J17065	J17070
23 cm - 9"	J170563	J170564	-
28 cm - 11"	J170566	J170567	-

XI - Neurochirurgie
XI - Neurosurgery
XI - Neurocirugía

Copyright © LANDANGER - 2009 - 2015



XI - 11

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
J17007	PINCE RONGEUR CUSHING L:180MM MORS COUDES 30° BAS 2X10MM	3661219076018
	CUSHING LAMINECTOMY RONGEUR L:180MM ANGLED JAWS DOWNWARD 30° 2X10MM	 3 661219 076018

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

35541

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

ACIER INOXYDABLE / STAINLESS STEEL

Material type :

Nuance matière :

X20Cr13(1.4021) + X8CrNiS18-9(1.4305)

Material grade :

Dimensions :

Voir désignations produits

Landanger 2015

Outer dimensions :

See products description

Chapitre-pages

11-011

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : *Standard conditions for reusable surgical instruments*

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).