

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-29

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
J211852EL	PINCE EMPORTE PIECE FERRIS SMITH KERRISON OUV 16MM A BAIONNETTE A GAUCHE L:180MM MORS 2MM 40° HAUT, DEMONTABLE, EJECTEUR	3661219425649
	LEFT BAYONET FERRIS SMITH KERRISON PUNCH FORCEPS 16MM OPENING L:180MM JAWS 2MM 40° UPWARD, DETACHABLE, EJECTOR	

Catalogues / catalogs

OrthoVertebre 2019

Chapitre-page

300


DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

L A N D A N G E R

Pinces emporte pièces - Punches forceps - Pinzas sacabocados

Ferris-Smith-Kerrison

Démontable - Avec éjecteur - Detachable - With ejector - Desmontable - Con Ejector
 Grande ouverture: 16 mm - Large opening: 16 mm - Gran apertura: 16 mm
 Baïonnette à gauche - Bayonet to the left - Bayoneta a la izquierda

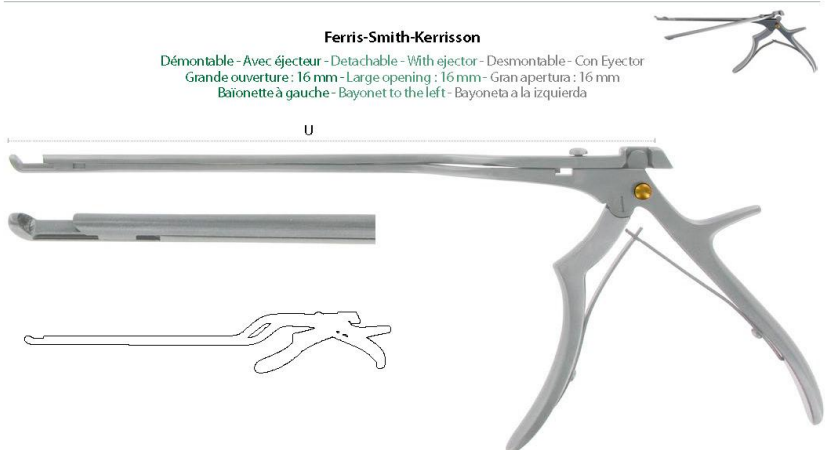

J211155EL
U: 110 mm - 4^{1/2}"

40° Haut / 40° Upward / 40° Arriba

Largeur de mors Longueur tige - Shaft length - Largo del tallo
 Width of jaw Anchura de boca U: 110 mm - 4^{1/2}" U: 150 mm - 6" U: 180 mm - 7"

1 mm	J211151EL	J211551EL	J211851EL
2 mm	J211152EL	J211552EL	J211852EL
3 mm	J211153EL	J211553EL	J211853EL
4 mm	J211154EL	J211554EL	J211854EL
5 mm	J211155EL	J211555EL	J211855EL
6 mm	J211156EL	J211556EL	J211856EL

U: Longueur utile
U: Working length
U: Longitud útil

VIII - Pinces
VIII - Forceps
VIII - Pinzas

Copyright © LANDANGER - 2019

300

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-29

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
J211852EL	PINCE EMPORTE PIECE FERRIS SMITH KERRISON OUV 16MM A BAIONNETTE A GAUCHE L:180MM MORS 2MM 40° HAUT, DEMONTABLE, EJECTEUR	3661219425649
	LEFT BAYONET FERRIS SMITH KERRISON PUNCH FORCEPS 16MM OPENING L:180MM JAWS 2MM 40° UPWARD, DETACHABLE, EJECTOR	 3 661219 425649

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :
CE Class :

I

Code GMDN :
GMDN code :

32853

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :
Operating room medical device :

REUTILISABLE/REUSABLE

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :
Material type :

ACIER INOXYDABLE / STAINLESS STEEL

Nuance matière :
Material grade :

X20Cr13(1.4021)

Dimensions : Voir désignations produits
Outer dimensions : See products description
Chapitre-pages

OrthoVertebre 2019

300

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.
Cleaning / Pre-disinfection : Standard conditions for reusable surgical instruments

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement : Pochette plastique en polyéthylène (PE).
Packaging : Polyethylene plastic pouch (PE).