

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-29

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
J2152013	RONGEUR PRESTIGE CASPAR L:200MM MORS DROITS 3X12MM, DEMONTABLE	3661219426974
	PRESTIGE CASPAR LAMINECTOMY RONGEUR L:200MM STRAIGHT JAWS 3X12MM, DETACHABLE	

Catalogues / catalogs

OrthoVertebre 2019

Chapitre-page

257

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

L A N D A N G E R *Prestige*

Pinces rongeurs - Rongeurs forceps - Pinzas para laminectomía

Caspar

Démontable - Detachable - Desmontable



J2151812
L : 180 mm - 7"

			Longueur tige Shaft length Largo del tallo		
			U : 180 mm - 7"	U : 200 mm - 7 ⁵ / ₈ "	U : 230 mm - 9"
 Droite Straight Recto	Largueur de mors Width of jaw Anchura de boca	1,5 x 12 mm	J2151811	J2152011	J2152311
	2 x 12 mm	J2151812	J2152012	J2152312	
	3 x 12 mm	J2151813	J2152013	J2152313	
	4 x 12 mm	J2151814	J2152014	J2152314	
	5 x 12 mm	J2151815	J2152015	J2152315	
 30° Vers le haut Upward Hacia arriba	Largueur de mors Width of jaw Anchura de boca	1,5 x 12 mm	J2151821	J2152021	J2152321
	2 x 12 mm	J2151822	J2152022	J2152322	
	3 x 12 mm	J2151823	J2152023	J2152323	
	4 x 12 mm	J2151824	J2152024	J2152324	
	5 x 12 mm	J2151825	J2152025	J2152325	

U : Longueur utile
U : Working length
U : Longitud útil

LANDANGER	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	--	---

Éditée le / Publishing date

2022-04-29

Page

2

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
J2152013	RONGEUR PRESTIGE CASPAR L:200MM MORS DROITS 3X12MM, DEMONTABLE	3661219426974
	PRESTIGE CASPAR LAMINECTOMY RONGEUR L:200MM STRAIGHT JAWS 3X12MM, DETACHABLE	

INFORMATIONS REGLEMENTAIRES / REGULATORY INFORMATION

Classe CE :
CE Class :

I

Code GMDN :
GMDN code :

35541

PERFORMANCES REVENDIQUEES / INTENDED PERFORMANCE

Dispositif médical de bloc opératoire :
Operating room medical device :

REUTILISABLE/REUSABLE

CARACTERISTIQUES SPECIFIQUE / SPECIFIC FEATURES

Type matière :
Material type :

ACIER INOXYDABLE / STAINLESS STEEL

Nuance matière :
Material grade :

X20Cr13(1.4021)

Dimensions : Voir désignations produits
Outer dimensions : See products description
Chapitre-pages

OrthoVertebre 2019

257

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.
Cleaning / Pre-disinfection : Standard conditions for reusable surgical instruments

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / PACKAGING

Conditionnement : Pochette plastique en polyéthylène (PE).
Packaging : Polyethylene plastic pouch (PE).