

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-29

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
J251832EB	PINCE EMPORTE PIECE PRESTIGE FERRIS SMITH KERRISON L:180MM MORS 2MM 90° HAUT, DEMONTABLE, EJECTEUR, REVETEMENT CERAMIQUE NOIR	3661219428268
	PRESTIGE FERRIS SMITH KERRISON PUNCH FORCEPS L:180MM JAWS 2MM 90° UPWARD, DETACHABLE, EJECTOR, BLACK CERAMIC COATING	

Catalogues / catalogs

OrthoVertebre 2019

Chapitre-page

278

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

Prestige

L A N D A N G E R

Pincas emporte pièces - PUNCHES forceps - Pinzas sacabocados

Ferris-Smith-Kerrison

Démontable - Avec éjecteur - Detachable - With ejector - Desmontable - Con Ejector
Revetement céramique noir - Ceramic black coating - Revestimiento de cerámico negro

J252035EB
U : 200 mm - 8"

90° Haut / 90° Upward / 90° Arriba

Largeur de mors

Width of jaw

Anchura de boca

Longueur tige - Shaft length - Largo del tallo

U : 180 mm - 7" U : 200 mm - 8" U : 230 mm - 9" U : 280 mm - 11"

1 mm	J251831EB	J252031EB	J252331EB	J252831EB
2 mm	J251832EB	J252032EB	J252332EB	J252832EB
3 mm	J251833EB	J252033EB	J252333EB	J252833EB
4 mm	J251834EB	J252034EB	J252334EB	J252834EB
5 mm	J251835EB	J252035EB	J252335EB	J252835EB



90° Haut talon fin / 90° Upward thin foot plate / 90° Arriba talón fino

Largeur de mors

Width of jaw

Anchura de boca

Longueur tige - Shaft length - Largo del tallo

U : 180 mm - 7" U : 200 mm - 8" U : 230 mm - 9" U : 280 mm - 11"

1 mm	J251831ETB	J252031ETB	J252331ETB	J252831ETB
2 mm	J251832ETB	J252032ETB	J252332ETB	J252832ETB
3 mm	J251833ETB	J252033ETB	J252333ETB	J252833ETB
4 mm	J251834ETB	J252034ETB	J252334ETB	J252834ETB
5 mm	J251835ETB	J252035ETB	J252335ETB	J252835ETB

T: Talon fin : seulement pour usage délicat - petites portions d'os ou tissus mous

T: Thin footplate: for delicate use only - small portion of bones or soft tissues

T: Talón fino: solo para uso delicado - pequeña porción de huesos o tejidos blandos

U: Longueur utile

U: Working length

U: Longitud útil

278

VIII - Pincas
VIII - Forceps
VIII - Pinzas

Copyright © LANDANGER - 2019

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-29

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
J251832EB	PINCE EMPORTE PIECE PRESTIGE FERRIS SMITH KERRISON L:180MM MORS 2MM 90° HAUT, DEMONTABLE, EJECTEUR, REVETEMENT CERAMIQUE NOIR	3661219428268
	PRESTIGE FERRIS SMITH KERRISON PUNCH FORCEPS L:180MM JAWS 2MM 90° UPWARD, DETACHABLE, EJECTOR, BLACK CERAMIC COATING	 3 661219 428268

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

32853

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

ACIER INOXYDABLE + REVETEMENT / STAINLESS STEEL + COATING

Material type :

Nuance matière :

X20Cr13(1.4021) + AlTiN

Material grade :

Dimensions :

Voir désignations produits

OrthoVertebre 2019

Outer dimensions :

See products description

Chapitre-pages

278

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : Standard conditions for reusable surgical instruments

Sterilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).