

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
J329015	CISEAU OSTEOTOME LEXER L:220MM 15MM	3661219210610
	LEXER OSTEOTOMY SCISSOR L:220MM 15MM	 3 661219 210610

Catalogues / catalogs

Landanger 2015

Chapitre-page

13-080

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

L A N D A N G E R

Ciseaux ostéotomes, gouges - Ostéotomes, gouges - Osteotomos, gubias

MINI LEXER



10 mm

Osteotome
Osteotome
Osteotomo

18 cm - 7"	12 mm	18 cm - 7"
J328912		J368912
J328910		J368910
J328908		J368908
J328906		J368906
J328904		J368904



10 mm

Gouge
Gouge
Gubia


18 cm - 7"	12 mm	18 cm - 7"
J328912		J368912
J328910		J368910
J328908		J368908
J328906		J368906
J328904		J368904

LEXER



J329010
22 cm - 8 3/4"

22 cm - 8 3/4"	30 mm	22 cm - 8 3/4"
J329030		J369030
J329025		J369025
J329020		J369020
J329015		J369015
J329010		J369010
J329007		J369007
J329005		J369005

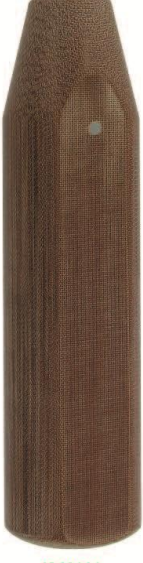

J369010
22 cm - 8 3/4"

22 cm - 8 3/4"	30 mm	22 cm - 8 3/4"
J329030		J369030
J329025		J369025
J329020		J369020
J329015		J369015
J329010		J369010
J329007		J369007
J329005		J369005

XIII - Osseux
XIII - Bones surgery
XIII - Craniósea



Copyright © LANDANGER - 2009 - 2015

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
J329015	CISEAU OSTEOTOME LEXER L:220MM 15MM	3661219210610
	LEXER OSTEOTOMY SCISSOR L:220MM 15MM	 3 661219 210610

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :

I

Code GMDN :

12844

CE Class :

GMDN code :

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :

REUTILISABLE/REUSABLE

Operating room medical device :

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :

ACIER INOXYDABLE + CELERON / STAINLESS STEEL + CELERON

Material type :

Nuance matière :

X46Cr13(1.4034) + CELERON

Material grade :

Dimensions :

Voir désignations produits

Landanger 2015

Outer dimensions :

See products description

Chapitre-pages

13-080

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.

Cleaning / Pre-disinfection : *Standard conditions for reusable surgical instruments*

Stérilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement :

Pochette plastique en polyéthylène (PE).

Packaging :

Polyethylene plastic pouch (PE).