

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-29

Page

1

REFERENCE	DESIGNATIONS - DESCRIPTIONS	CODE EAN
O123020	PLATEAU ACIER INOXYDABLE 300X200X50MM	3661219322542
	STAINLESS STEEL TRAY 300X200X50MM	

Catalogues / catalogs

Sterilisation 2016

Chapitre-page

50

DESCRIPTIF DU DISPOSITIF/ PRODUCT DESCRIPTION

L A N D A N G E R

Plateaux inox / Stainless steel trays



	Dimensions Dimensions	
O122010	200 x 100 x 25 mm / 7.9 x 3.9 x 1"	
O122015	200 x 150 x 40 mm / 7.9 x 5.9 x 1.6"	
O122315	230 x 150 x 50 mm / 9 x 5.9 x 1.9"	
O122514	250 x 140 x 50 mm / 9.8 x 5.5 x 1.9"	
O122520	250 x 200 x 50 mm / 9.8 x 7.9 x 1.9"	
O123020	300 x 200 x 50 mm / 11.8 x 7.9 x 1.9"	1
O123025	300 x 250 x 50 mm / 11.8 x 9.8 x 1.9"	
O123525	350 x 250 x 50 mm / 13.7 x 9.8 x 1.9"	
O123530	350 x 300 x 50 mm / 13.7 x 11.8 x 1.9"	
O124030	400 x 300 x 50 mm / 15.7 x 11.8 x 1.9"	
O124530	450 x 300 x 50 mm / 17.7 x 11.8 x 1.9"	

Boîtes de stérilisation inox / Stainless steel sterilization boxes



	Dimensions Dimensions	
O111770	170 x 70 x 30 mm / 6.7 x 2.7 x 1.2"	
O111880	180 x 80 x 40 mm / 7 x 3.1 x 1.6"	
O112010	200 x 100 x 40 mm / 7.8 x 3.9 x 1.6"	
O112512	250 x 125 x 50 mm / 9.8 x 4.9 x 1.9"	
O113015	300 x 150 x 50 mm / 11.8 x 5.9 x 1.9"	1
O113518	350 x 180 x 60 mm / 13.7 x 7 x 2.4"	
O114020	400 x 200 x 90 mm / 15.7 x 7.9 x 3.5"	
O115020	500 x 200 x 120 mm / 19.7 x 7.9 x 4.7"	

Haricots inox / Stainless steel kidney trays



	Longueur / Length	Hauteur / Height	
O130015	150 mm / 5.9"	30 mm / 1.2"	
O130018	180 mm / 7"		
O130020	200 mm / 7.9"	40 mm / 1.6"	1
O130025	250 mm / 9.8"		

 4 Boîtes, plateau et cuvettes
Surgical Trolleyware

50

Copyright © LANDANGER - 2009 - 2016

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-29

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
O123020	PLATEAU ACIER INOXYDABLE 300X200X50MM	3661219322542
	STAINLESS STEEL TRAY 300X200X50MM	

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :
CE Class :

I-MDR

Code GMDN :
GMDN code :

12143

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :
Operating room medical device :

REUTILISABLE/REUSABLE

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :
Material type :

ACIER INOXYDABLE / STAINLESS STEEL

Nuance matière :
Material grade :

X2CrNi19-11(1.4306)

Dimensions : Voir désignations produits
Outer dimensions : See products description
Chapitre-pages

Sterilisation 2016

50

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.
Cleaning / Pre-disinfection : Standard conditions for reusable surgical instruments

Sterilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement : Pochette plastique en polyéthylène (PE).
Packaging : Polyethylene plastic pouch (PE).