

L A N D A N G E R	FICHE TECHNIQUE PRODUIT PRODUCT TECHNICAL SHEET	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-------------------	--	--

Editée le / Publishing date

2022-04-28

Page

1

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
P35040	CUPULE A BEC FOND PLAT D:70MM H:35MM 85 ML	3661219117636
	LOTION BOWL WITH FLAT BOTTOM D:70MM HEIGHT 35MM CAPACITY 85ML	

Catalogues / catalogs

Sterilisation 2016

Chapitre-page

51

DESCRIPTIF DU DISPOSITIF/ *PRODUCT DESCRIPTION*

L A N D A N G E R

Cupules à bec, fond plat, inox / *Stainless steel lotion bowls with flat bottom*


	Diamètre / Diameter	Hauteur / Height	Volume / Volume	
P35010	Ø 40 mm / 1.5"	20 mm / .75"	20 ml / 0.67 fl.oz	
P35020	Ø 50 mm / 2"	25 mm / 1"	35 ml / 1.17 fl.oz	
P35030	Ø 60 mm / 2.38"	30 mm / 1.15"	60 ml / 2.00 fl.oz	
P35040	Ø 70 mm / 2.76"	35 mm / 1.38"	85 ml / 2.83 fl.oz	
P35050	Ø 80 mm / 3.15"	40 mm / 1.55"	100 ml / 3.33 fl.oz	
P35060	Ø 100 mm / 4"	50 mm / 2"	250 ml / 8.33 fl.oz	
P35070	Ø 120 mm / 4.72"	60 mm / 2.36"	500 ml / 16.67 fl.oz	
P35080	Ø 160 mm / 6.3"	80 mm / 3.15"	1 L / 0.26 gal	
P35090	Ø 200 mm / 8"	100 mm / 4"	2 L / 0.53 gal	
P35100	Ø 250 mm / 10"	120 mm / 4.72"	5 L / 1.32 gal	

Bols inox / *Stainless steel bowls*


	Diamètre / Diameter	
O132030	Ø 300 mm / 11.8"	
O132035	Ø 350 mm / 13.7"	
O132040	Ø 400 mm / 15.7"	
O132045	Ø 450 mm / 17.7"	

Bocks à douche inox / *Stainless steel douche cans*


	Volume / Volume	
O133010	1 L / 0.26 gal	
O133015	1.5 L / 0.39 gal	
O133020	2 L / 0.52 gal	

4 Boîtes - plateaux et cuvettes
Surgical Trays

51

LANDANGER	FICHE TECHNIQUE PRODUIT <i>PRODUCT TECHNICAL SHEET</i>	FAQ 026 INFO Indice : 2 Date : 01/06/2018
-----------	---	---

Éditée le / *Publishing date*

2022-04-28

Page

2

REFERENCE	DESIGNATIONS - <i>DESCRIPTIONS</i>	CODE EAN
P35040	CUPULE A BEC FOND PLAT D:70MM H:35MM 85 ML	3661219117636
	LOTION BOWL WITH FLAT BOTTOM D:70MM HEIGHT 35MM CAPACITY 85ML	 3 661219 117636

INFORMATIONS REGLEMENTAIRES / *REGULATORY INFORMATION*

Classe CE :
CE Class :

I-MDR

Code GMDN :
GMDN code :

42893

PERFORMANCES REVENDIQUEES / *INTENDED PERFORMANCE*

Dispositif médical de bloc opératoire :
Operating room medical device :

REUTILISABLE/REUSABLE

CARACTERISTIQUES SPECIFIQUE / *SPECIFIC FEATURES*

Type matière :
Material type :

ACIER INOXYDABLE / STAINLESS STEEL

Nuance matière :
Material grade :

X2CrNi19-11(1.4306)

Dimensions : Voir désignations produits
Outer dimensions : See products description
Chapitre-pages

Sterilisation 2016

51

Nettoyage/Pré-désinfection : Standard pour instrumentation de bloc opératoire réutilisable.
Cleaning / Pre-disinfection : Standard conditions for reusable surgical instruments

Sterilisation :

La stérilisation doit être effectuée selon un procédé de stérilisation à la vapeur conformément à la norme ISO 17665-1 (procédé sous vide fractionné) avec un stérilisateur conformément à la norme EN 285.

Méthode de stérilisation à la vapeur saturée	Température	Durée	Temps de séchage	Pression absolue
Pré-vide	132 °C (270 °F)	4 minutes	30 minutes	2,9 bars, 2,8 atm, 41,6 psi
Pré-vide	134 °C (273 °F)	3 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	5 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi
Pré-vide	134 °C (273 °F)	18 minutes	35 minutes	3,0 bars, 3,0 atm, 44,1 psi

Sterilization process :

The Sterilization shall be performed according to ISO 17665-1 validated steam sterilisation procedure (fractional vacuum procedure) in an autoclave in compliance with EN 285.

Saturated steam sterilization method	Temperature	Time	Drying time	Absolute pressure
Pre-vacuum	132 °C (270 °F)	4 minutes	30 minutes	2.9 bar, 2.8 atm, 41.6 psi
Pre-vacuum	134 °C (273 °F)	3 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	5 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi
Pre-vacuum	134 °C (273 °F)	18 minutes	35 minutes	3.0 bar, 3.0 atm, 44.1 psi

EMBALLAGE / *PACKAGING*

Conditionnement : Pochette plastique en polyéthylène (PE).
Packaging : Polyethylene plastic pouch (PE).