NL

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EN

CureTape is an elastic, adhesive tape without medicines and is made of high-quality cotton with an acrylic adhesive layer. CureTape can be worn for several days, is free from latex and is water resistant.

CureTape may only be used by professional users who have had training in the special kinesiology-based taping technique.

Directions for use:

- In rare cases, the use of CureTape can cause skin irritations or allergic reactions. The treatment should be stopped immediately if adverse reactions occur and the tape should be removed from the skin.
- It is recommended that the user first applies a small piece of tape if he/she is over-sensitive to the use of plasters or tape. It must then be borne in mind that allergic reactions do not always occur immediately, but that a later reaction (after approximately 24-48 hours) can also occur. Remove the product immediately in case of adverse reactions.
- Do not use CureTape on wounds, burns and mucous membranes, nor on the mouth, nose or eyes.

- The skin should be cleaned before applying the CureTape.
- When removing the CureTape, the tape should be removed slowly in the direction of the hair growth. If the removal of the tape proves difficult, it is advised to use a tape removal agent.

Instructions

- Keep CureTape at room temperature (between 4 – 25 °C) and not in direct sunlight.
- Do not put CureTape on or near radiators or any other source of direct heat.
- Keep the tape out of reach of children.
- For single use only.
- Injuries can always occur, even when CureTape is used in the proper manner.

DJO Global, Inc. / THYSOL Group BV are not liable for any injuries and/or damage due to CureTape or the unprofessional use of the product.

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