EN - INSTRUCTIONS FOR USE

General Description

Manosplint® is a low temperature thermoplastic material used to fabricate splints, orthoses and rehabilitation aids.

Precautions

To be used under the guidance of a qualified medical professional.

Heating Methods, Temperature & Working Time

Hot Water Method: 140° - 162°F (60° - 72°C) for 1- 2 minutes with a forming working time of 2 - 4 minutes depending on material thickness and the degree in which the material was stretched. Manosplint® Ohio and Manosplint® Ohio S turn translucent when heated. Manosplint® Ohio F should be placed in a sealed plastic bag to prevent water absorption into the fabric material.

Dry Heat Oven Method: 176° - 194°F (80° - 90°C) for 3- 5 minutes with a forming working time of 2 - 4 minutes depending on material thickness and the degree in which the material was stretched. Manosplint® Ohio and Manosplint® Ohio S turn translucent when heated.

Patterns & Cutting

A pattern can be drawn directly on the sheet material with a scratch awl. Thicker material may require scoring first with a utility knife and then bending at the score. Cut the pattern with a heavy duty shears or scissors. Finish trimming with a scissors.

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Note, warming the material to soften it slightly (not heating all the way) will aid in the ease of cutting and form a soft sealed edge.

Forming the Splint

Utilize desired heating method. Note Moderate resistive materials require smooth strokes while maximum resistive material will require a firmer hand. Manosplint® Ohio S and Manosplint® Carolina are uncoated material "sticky" this will aid in application by keeping the material in place. It will also allow for a permanent bond. Coated materials Manosplint® Arizona Manosplint® Ohio and Manosplint® Wisconsin can form a temporary bond by pinching the material together or a permanent bond if the coating is removed by scraping or with a solvent.

Finishing & Adjusting

Manosplint® thermoplastic splints can be adjusted with a heat gun or dipping in hot water. Edges will remain smooth and closed. Splints maintain their shape while using low heat for fitting and adjustments. Manosplint® Ohio and Manosplint® Ohio S have 100% memory and if reheated can return to their original shape.

Care

Clean with a mild soap and lukewarm water. Do not immerse fabricated splints in water greater than 123°F (51°C) as the splint will lose its shape.

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