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Medes Limited

Item NA-N149XX - Ver. 1.0

Specifications are subject to change without notice

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Sensors

English

Thank you for purchasing a SleepSense® sensor.

Description

The Pediatric ThermoCan™ Interface Cable is used to connect the Pediatric ThermoCan™ disposable thermal + pressure and ETCO2 cannula to the PSG system. The Interface Cable provides clean and reliable respiration signals to the system. The output may be connected to any AC (low level) channel on the recorder.

Indications for use

SleepSense sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system. They are intended for use on children and adult patients who are screened during sleep disorder studies at a sleep laboratory or the patient's home.

Applying the Pediatric ThermoCan™ Interface Cable

1. Connect the Interface Cable to the PSG system.
2. Slide the Interface Cable's golden pins into the holes provided on the sides of the cannula nose piece. Push the pins all the way in. Avoid pushing the thermal flow sensor. The pins should be placed inside the white soft cushions, in parallel to the golden contacts of the thermal flow sensor.
3. Place the Pediatric ThermoCan™ cannula on the patient's face per the instructions included with the cannula.
4. Check that clear, strong signals are being transmitted before leaving the patient.

Cleaning the Interface Cable

- Wipe the cable and golden pins with a non-corrosive (to plastic) cleanser to clean before use.
- Make sure the complete sensor assembly is thoroughly dry before reusing it.
- Do not soak the sensor in disinfectants.

Technical Specifications:

Cable:	White insulation wire		
Sensor:	Thermocouple	Thermistor	Thermistor SD20
Signal Output:	2mV	1KΩ	20Ω
Recommended Filter Settings:	High Pass: 0.1 Hz/ Low Pass: 70 Hz or less		
Operating conditions:	5°C (40°F) - 40°C (104°F)		
Storage temperature:	-20°C (-4°F) - 60°C (140°F)		
Operating and storage humidity:	5% - 95% (Non-condensing		

!

Warnings and Precautions

- **Caution:** US Federal Law restricts this device to sale, distribution and use by or on the order of a physician.
- SleepSense sensors are for professional use only.
- SleepSense sensors may be used only in conjunction with an approved recording system.
- SleepSense sensors are only intended for use on healthy skin.
- Discontinue use if sensor shows signs of wear, damage, or exposed metal.

Limited Warranty
SLP warrants the ThermoCan™ Interface Cable to be free of defects in materials and workmanship for a period of d of twelve months from the date purchased. The sole liability of SLP and our distributor(s) is limited to replacement or repair of the product at the option of SLP with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall SLP or our distributor(s) be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This warranty is intended only for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect, or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

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