BRAEBON

For Today's Business of Sleep™

Canada:

1-100 Schneider Road
Kanata, ON, Canada, K2K 1Y2
01.613.831.6690
Toll-free (N.A.): 1.888.462.4841

CEpartner4U BV
Esdoorniaan 13
3951 DB Mearn
The Netherlands
Tel: +31.343.442.524

en PVDF Respiratory Effort Belt User Guide

Intended Use

The BRAEBON MEDICAL CORPORATION Ultima PVDF Respiratory Effort Sensor™ is intended for use during sleep disorder studies as a qualitative measure of respiratory effort for recording onto a data acquisition system. The sensor uses PVDF (piezo-electric) technology and plugs directly into the electrode headbox.

Warning: Respiratory effort sensors are for diagnostic purposes only and are not intended as apnea (apnoea) monitors.

Caution: US Federal law restricts this device to sale by or on the order of a physician. You may wash the belt, but not the PVDF (piezo-electric) sensor.

Note: Refer to www.braebon.com for more information. Product specifications are subject to change without notice.

How to Use

Refer to your sleep system user guide for manufacturer's recommended settings. Red 1.5 mm touchproof connector is positive; black is negative.

General recording system settings:

- Low frequency filter: 0.1 Hz (TC=1 sec or longer)
- High frequency filter: 5 Hz to 15 Hz
- 50 / 60 Hz notch filter: optional
- Sensitivity / Gain: 2 mV/cm or 4,000 gain (increase or decrease as needed)

Adjust the Ultima PVDF Respiratory Effort Sensor™ to fit the patient comfortably. Place the belt snugly around either the patient's thorax or abdomen. The thoracic belt should sit just below or above the nipple line and the abdominal belt should sit below the rib cage.

To use the loop connector, place one end of the belt through the loop and fold to fasten the Velcro onto the belt. Ensure that the belt is neither excessively tight nor loose.

To use the buckle connector, simply fasten one end of the belt into the other end of the belt. Ensure that the belt is neither excessively tight nor loose.

Specifications

Operating Temperature: 5°C (41°F)-32°C (90°F)

Storage Temperature: 0°C (32°F)-60°C (140°F)

Operating/Storage Humidity: 10-95% Non-condensing

Disinfection and Cleaning

Caution: You may wash the belt, but not the PVDF (piezo-electric) sensor. Do not steam autoclave this device.

The Ultima PVDF Respiratory Effort Sensor™ is classified as a non-critical item because it only contacts clothing and intact skin. You may machine wash the respiratory belt, but not the PVDF (piezo-electric) sensor. If necessary, you may use isopropyl alcohol pads to gently clean the sensor. For additional information, refer to the APIC guidelines for selection and use of disinfectants (*American Journal of Infection Control*, Vol. 18, number 2, April, 1990, p. 99-117).

Ensure that The Ultima PVDF Respiratory Effort Sensor™ is completely dry before using. Never immerse the PVDF (piezo-electric) sensor enclosure at any time. **Steam autoclaving is not recommended.**

Warranty

BRAEBON MEDICAL CORPORATION warrants to the first consumer that the respiratory effort sensor (the "Sensor"), when shipped in its original container, will be free from defective workmanship, performance and materials and agrees that it will, at its option, either repair the defect or replace the defective Sensor or part thereof at no charge to the purchaser for parts or labor for a time period of one year from the date of purchase. The warranty described herein shall be the sole and exclusive warranty granted by BRAEBON MEDICAL CORPORATION and shall be the sole and exclusive remedy available to the purchaser. Use of the Sensor constitutes total and complete acceptance of this warranty. Correction of defects, in the manner and for the time period described herein, shall constitute complete fulfillment of all liabilities and responsibilities of BRAEBON MEDICAL CORPORATION to the purchaser with respect to the Sensor and shall constitute full satisfaction of all claims, whether based on contract, negligence, strict liability or otherwise. In no event shall BRAEBON MEDICAL CORPORATION be liable, or in any way responsible, for any loss of revenues or damage, direct, incidental, or consequential, including property damage, loss of profit, or personal injury

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BRAEBON Medical Corp.