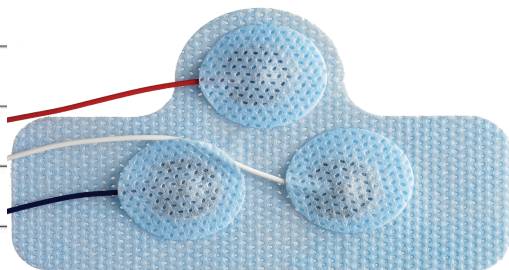
DYMEDIX DIAGNOSTICS

ADVANCING the SCIENCE of SLEEP™

Chin Electrode Patch

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Almofada do eletrodo para o queixo

Parche de electrodo de barbilla

Do Not Reuse

Latex Free

0010 Rev C

Ref 026489 - natus

ELECTRODE PLACEMENT DIAGRAM

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EN

CAUTION STATEMENTS

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

This product should be used only under the direct supervision of a trained medical professional. In the event of skin reaction, discontinue use.

This product is for diagnostic purposes only and is not to be used in life supporting or life sustaining situations.

FEATURES AND INTENDED USE:

Dymedix electrodes detect small electrical signals during sleep disorder studies. This electrode is intended to produce a small electrical signal to existing recording equipment. If there is no signal output from the chin electrode patch and it is properly placed on the patient or if the patch becomes damaged, discontinue use.

The Dymedix Chin Patch electrode contains 3 electrodes equally spaced at 1cm with color-coded wires.

INSTRUMENT CONNECTIONS & SETTINGS

Refer to the polysomnograph manual for proper instrument connection and settings. Response is affected by sensor placement, sleep position, ambient room temperature, sensitivity settings, frequency filter settings and system characteristics.

PRECAUTIONS FOR ELECTRODES

1. Store sealed electrodes at room temperature
2. Do not sterilize electrodes
3. Connect the electrode to electrically isolated equipment
4. Use all available recorded parameters in making diagnosis

PATIENT INSTALLATION

1. Clean the skin that electrode will be placed on with abrasive cleaner. Wipe off excess cleaner and allow to dry.
2. Remove electrode release liner and apply to cleaned unbroken area of patient's skin. Water or Saline gel may be added under the gel tab if needed to improve impedance.
3. Tape may be placed over the top of the electrodes for added security.
4. The blue and white wires must be connected to electrode inputs on the head box. The red wire must be connected to an unassigned input channel and used as a backup.
5. Proceed with polysomnography as outlined by clinician procedure.

ELECTRODE REMOVAL INSTRUCTIONS

1. At completion of the sleep study, remove electrodes from patient by pulling off skin.
2. Dispose of the electrode. Do not reuse.

WARRANTY

Dymedix warrants its electrodes to be free from defects in materials and workmanship and to perform satisfactorily for one year from the date of manufacture. Should the electrodes fail, the sole liability of Dymedix is limited to replacement of the product with no charge for parts or labor. Under no circumstances shall Dymedix be liable for any consequential damages arising from the use of, or the inability to use this product. This warranty is rendered void for conditions resulting from: damage, marginal performance or malfunctions caused by: reuse, misuse, abuse, neglect, improper amplification settings, improper line voltage, power fluctuations, or any adverse environmental conditions, tampering, unauthorized modifications, adjustments or repairs to the product or its accessories. This warranty is in lieu of all other warranties, expressed or implied and is extended only to the original purchaser.

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