EN

UTION STATEMENTS

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician. The product should be used only under the direct supervision of a trained medical professional. In event of skin reaction, discontinue use.

The product is for diagnostic purposes only and is not to be used in life supporting or life sustaining situations.

URES AND INTENDED USE:

Dymedix electrodes detect small electrical signals during sleep disorder studies. This electrode is needed to produce a small electrical signal to existing recording equipment. If there is no signal output from the electrode patch and it is properly placed on the patient or if the patch becomes damaged, discontinue use.

TRUMENT CONNECTIONS & SETTINGS

Refer to the polysomnograph manual for proper instrument connection and settings. Response affected by sensor placement, sleep position, ambient room temperature, sensitivity settings, amplifier filter settings and system characteristics.

CAUTIONS FOR ELECTRODES

Store sealed electrodes at room temperature.
Do not sterilize.
Connect the electrode to electrically isolated equipment.
Use all available recorded parameters in making diagnosis.

IENT INSTALLATION

Clean the area that electrode will be placed on with abrasive cleaner. Wipe off excess cleaner and allow to dry.
Remove electrode release liner and apply to cleaned unbroken area of patient's skin. Water or Saline gel may be added under the gel tab if needed to improve impedance.
Tape may be placed over the top of the electrodes for added security.
The electrodes must be connected to electrode inputs on the recording instrument.
Proceed with polysomnography as outlined by clinician procedure.

CTRODE REMOVAL INSTRUCTIONS

At completion of the sleep study, remove wires.
Remove electrodes from patient by pulling electrode off skin.
Dispose of the electrode. Do not reuse.

ARRANTY

Dymedix warrants its electrodes to be free from defects in materials and workmanship and to perform satisfactorily for one year from the date of manufacture. Should the electrodes fail, the sole liability of Dymedix is limited to replacement of the product with no charge for parts or labor. Under no circumstances shall Dymedix be liable for any consequential damages arising from the use of, or the inability to use this product. This warranty is rendered void for conditions resulting from: damage, physical performance or malfunctions caused by: reuse, misuse, abuse, neglect, improper amplifier settings, improper line voltage, power fluctuations, or any adverse environmental conditions, tampering, unauthorized modifications, adjustments or repairs to the product or its accessories. This warranty is in lieu of all other warranties, expressed or implied and is extended only to the original purchaser.

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Dymedix Diagnostics Inc