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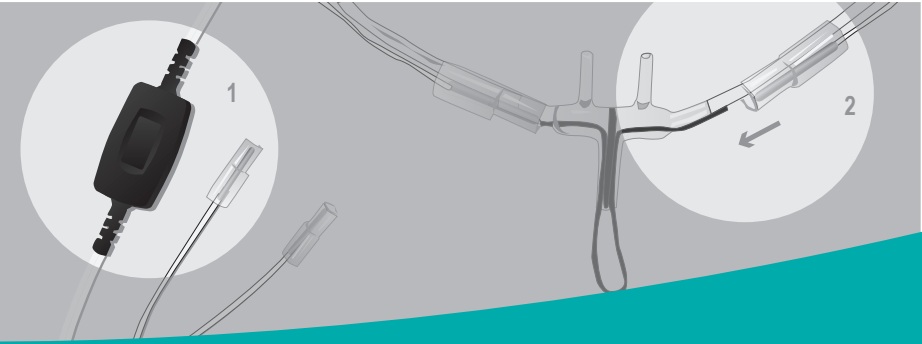
ThermoCan™ Cavo per Interfaccia

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ThermoCan™ schnittstellenkabel

To Reorder: Call: 1-800-356-0007 (in the USA) or your Natus Neurology Incorporated distributor
or email: supplies.nc@Natus.com

Manufactured by S.L.P., Ltd. for Natus Neurology Incorporated. Made in Israel.
S.L.P., Ltd.
62 Anilevitz St.
Tel Aviv, 6706016, ISRAEL
Tel: +972 3 6870640
Fax: +972 3 5371282
E-mail: info@sleepsense.com
Tel/Fax: +44 19238 59810
Herts WD7 7AR, England
Tel: +1 (630) 513 8478
Fax: +1 (630) 513 7479
Egin, IL 60124, USA
2511 Technology Dr., Suite 112
5 Beaumont Gate,
Shenley Hill, Radlett,
Medes Limited

Specifications are subject to change without notice
Les spécifications sont sujettes à modification sans préavis
Especificaciones sujetas a cambio sin previo aviso
Le specificazioni sono soggette a modifica senza preavviso
Die Spezifikationen sind ohne vorherige Meldung Änderungen unterworfen

natus
neurology®

Thank you for purchasing a SleepSense® sensor.

Description

The ThermoCan™ Interface Cable *figure 1* is used to connect the ThermoCan™ disposable thermal + pressure cannula *figure 2* to the PSG system. Internal battery (in some units) and filtering circuits provide clean and reliable respiration signals to the system. Battery should last at least two years of continuous use. The output may be connected to any AC (low level) channel on the recorder.

Indications for use

SleepSense sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system. They are intended for use on children and adult patients who are screened during sleep disorder studies at a sleep laboratory or the patient’s home.

Applying the ThermoCan™ Interface Cable

1. Connect the Interface Cable to the PGS system.
2. Slide the Interface Cable's golden pins into the holes provided on the sides of the cannula nose piece. Push the pins all the way in *figure 2*. Avoid pushing the black wire into the cannula body - the pin should rest parallel to the wire.
3. Place the ThermoCan™ cannula on the patient's face per the instructions included with the cannula.
4. Check that clear, strong signals are being transmitted before leaving the patient.

Cleaning the Interface Cable

- Wipe the cable and golden pins with a non-corrosive (to plastic) cleanser to clean before use.
- Make sure the complete sensor assembly is thoroughly dry before reusing it.
- Do not soak the sensor in disinfectants.

Technical Specifications:

Cable:	White PVC insulation lead wire
Sensor:	Disposable ThermoCan thermal cannula
Recommended Filter Settings:	High Pass: 0.1 Hz/ Low Pass: 70 Hz or less
Signal Output:	Approximately 2 mV for Normal Respiration
Operating conditions:	5°C (40°F) - 40°C (104°F)
Storage temperature:	-20°C (-4°F) - 60°C (140°F)
Operating and storage humidity:	5% - 95% (Non-condensing)

 Warnings and Precautions

- **Caution:** US Federal Law restricts this device to sale, distribution and use by or on the order of a physician.
- SleepSense sensors are for professional use only.
- SleepSense sensors may be used only in conjunction with an approved recording system.
- SleepSense sensors are only intended for use on healthy skin.
- Discontinue use if sensor shows signs of wear, damage, or exposed metal.

Attention: contains a lithium battery - dispose of properly.

Limited Warranty
SLP warrants the ThermoCan™ Interface Cable to be free of defects in materials and workmanship for a period of d of twelve months from the date purchased. The sole liability of SLP and our distributor(s) is limited to replacement or repair of the product at the option of SLP with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall SLP or our distributor(s) be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This warranty is intended only for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect, or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

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