Banda

Italiano

Bänder

Deutsch

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CE

or email: supplies.nc@Natus.com

To Reorder: Call: 1-800-356-0007 (in the USA) or your Natus Neurology Incorporated distributor

S.L.P. Ltd.

62 Mordechai Anilevitz St.

Tel-Aviv, 6706016

ISRAEL

Tel: +1 (630) 513 7479

Fax: +1 (630) 513 8478

E-mail: info@sleepsense.com

SleepSense.com

Tel: +423-663-169205

Tel/Fax: +44 19238 59810

S.L.P. Inc.

2511 Technology Dr., Suite 112

Elgin, IL 60124, USA

Shenley Hill, Radlett,

5 Beaumont Gate,

Medes Limited

EC REP

Manufactured by S.L.P. Ltd. for Natus Neurology Incorporated. Made in Israel.

To Reorder: Call: 1-800-356-0007 (in the USA) or your Natus Neurology Incorporated distributor

or email: supplies.nc@Natus.com

Specifications are subject to change without notice
Les spécifications sont sujettes à modification sans préavis
Especificaciones sujetas a cambio sin previo aviso
Le specificazioni sono soggette a modifica senza preavviso
Die Spezifikationen sind ohne vorherige Meldung Änderungen unterworfen

Belts

User Manual

Semi Reusable Inductive Belts (White/Blue)

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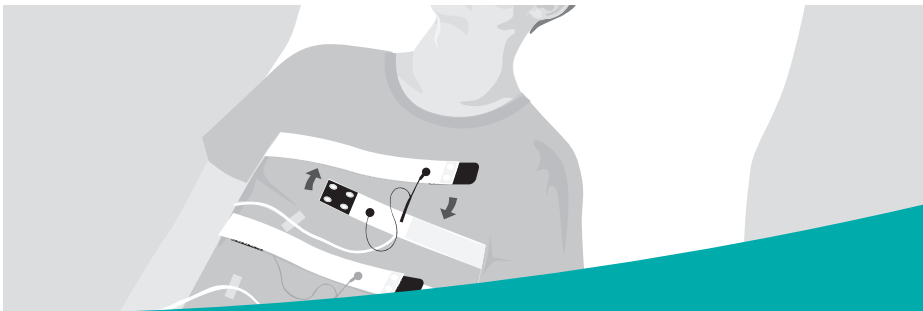
Belts

English

Thank you for purchasing a SleepSense® sensor.

Description

The inductance of the inductive belts is exactly proportional to the cross section area of the band's closed loop around the body. It therefore changes as the patient breathes. The electronics in the system or interface box measure these inductance changes to generate a signal that is the most accurate and reliable replication of respiration waveforms. Inductive belts easily connect to any system compatible with inductive effort sensors through a dedicated SleepSense interface cable.

Indications for use

SleepSense sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system. They are intended for use on children and adult patients who are screened during sleep disorder studies at a sleep laboratory or the patient's home.

Positioning and Connecting the Belt (Using a SleepSense Interface Cable)

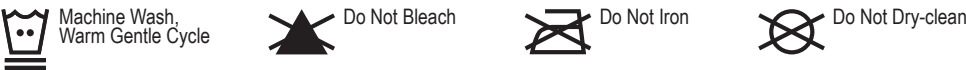
1. The patient should lie on his/her back for optimal positioning of the belt.
2. Select the proper size belt for the patient.
3. Strap the sensor around the patient's abdomen or chest over clothing.
4. Attach the first wire to the snap that is on the same side of the belt as the Velcro® patch.
5. While holding this end to the left of the patient's centerline, wrap the belt around the patient's chest or abdomen. For smaller patients move the starting point further to the left.
6. Pull the other end over the first end and secure to the belt on the right side of the centerline. The belt should be snug but not too tight, (see image).
7. Attach the second wire to the exposed snap. Secure both wires about 3" away from the connection.

- Check that clear, strong signals are being transmitted before leaving the patient.
- **When removing the interface cable do not pull on the wires. Remove from the snap.**

Attention! Using two chest or two abdomen interface cables on the same patient may cause interference.

Cleaning the Belt

- The belt are machine washable.
- Make sure the belt is thoroughly dry before re-use.
- Do not soak the belt in disinfectants.

Operating/Storage

Item #:	9003/9002 (different lengths)
Belt:	Synthetic inductive belt - non latex
Operating conditions:	5°C (40°F) - 40°C (104°F)
Storage temperature:	-20°C (-4°F) - 60°C (140°F)
Operating and storage humidity:	5% - 95% (Non-condensing)

!

Warnings and Precautions

- **Caution:** US Federal Law restricts this device to sale, distribution and use by or on the order of a physician.
- SleepSense sensors are for professional use only.
- SleepSense sensors may be used only in conjunction with an approved recording system.
- SleepSense sensors are only intended for use on healthy skin.
- Discontinue use if sensor shows signs of wear, damage, or exposed metal.

Limited Warranty
SLP warrants the Semi Reusable Inductive Belts to be free of defects in materials and workmanship for a period of 60 days from the date purchased. The sole liability of SLP and our Distributor(s) is limited to replacement or repair of the product at the option of SLP with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall SLP or our Distributor(s) be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This Warranty is intended only for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect, or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

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