Inductive

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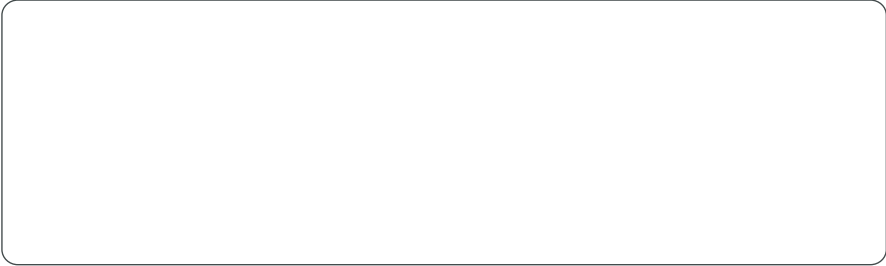
Item NA-N91XX – ver. 1.0

To Reorder: Call: **1-800-356-0007 (in the USA)** or your Natus Neurology Incorporated distributor
or email: **supplies.nc@Natus.com**

Manufactured by S.L.P., Ltd. for Natus Neurology Incorporated. Made in Israel.

S.L.P., Ltd.
62 Anilevitz St.
Tel Aviv, 6706016, ISRAEL
Tel: +972 3 6870640
Fax: +972 3 5371282
E-mail: info@sleepsense.com
Tel/Fax: +44 19238 59810

2511 Technology Dr., Suite 112
Eglin, IL 60124, USA
Shenley Hill, Radlett,
Herts WD7 7AR, England
Tel: +423-663-169205
Tel/Fax: +44 19238 59810

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Specifications are subject to change without notice

natus®
neurology

Inductive

Interface Cable

natus®
neurology

- User Manual
- Inductive Interface Cable**
- Manuel de l'utilisateur
- Câble d'interface inductive**
- Manual del Usuario
- Cable Interface inductivo**
- Manuale per l'Utente
- Induttivo cavo di interfaccia**
- Bedienungsanleitung
- Induktive Schnittstellenkabel**

Interface Cable

English

Thank you for purchasing a SleepSense® sensor.

Description

The SleepSense Interface Cable connects SleepSense inductive belts to any PSG system compatible with inductive effort sensors.

Indications for Use

SleepSense sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system. They are intended for use on children and adult patients who are screened during sleep disorder studies at a sleep laboratory or the patient's home.

Applying the Interface Cable

- 1.Connect the SleepSene Inductive Cable to the snaps on the SleepSense Inductive Belts (Reusable or Semi-Reusable Belts)
- 2.See the User Inductive supplied with the Inductive Belt for correct application.
3. Check that clear, strong signals are being transmitted before leaving the patient.
4. **When removing the sensor belt, do not pull on the wires. Remove from the snap.**

Attention! Using two chest or two abdomen interface cables on the same patient may cause interference.

Cleaning the Sensor

- Wipe the Interface Cable with a non-corrosive (to plastic) cleanser to clean before use.
- Make sure the Interface Cable is thoroughly dry before reusing it.
- Do not soak the Interface Cable in disinfectants.

Technical Specifications:

Name:	Interface Cables
Sensor:	Inductive Effort Sensor interface cable
Recommended filter settings:	High pass: 0.1 Hz / Low pass: 40 Hz or as recommended in the system's instructions
Operating conditions:	5°C (40°F) - 40°C (104°F)
Storage temperature:	-20°C (-4°F) - 60°C (140°F)
Operating and storage humidity:	5% - 95% (Non-condensing)

!

Warnings and Precautions

- **Caution:** US Federal Law restricts this device to sale, distribution and use by or on the order of a physician.
- SleepSense sensors are for professional use only.
- SleepSense sensors may be used only in conjunction with an approved recording system.
- SleepSense sensors are only intended for use on healthy skin.
- Discontinue use if sensor shows signs of wear, damage, or exposed metal.

Limited Warranty
SLP warrants the Inductive Interface Cable to be free of defects in materials and workmanship for a period of twelve months from the date purchased. The sole liability of SLP and our Distributor(s) is limited to replacement or repair of the product at the option of SLP with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall SLP or our Distributor(s) be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This Warranty is intended only for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect, or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

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