English

INSTRUCTIONS FOR USE
REUSABLE PRODUCTS GENERAL

Important product information:

Only to be used by trained and skilled personnel. Do not connect electrodes directly to a power source. Application of the electrodes should be supervised by a physician.

Cleaning and sterilization:

For details see Cleaning Recommendations.

Detailed product information:

See next page for detailed product information/instructions on:	Applicable instruction:
Concentric and Single Fiber needle electrodes	A, B
Nasopharyngeal electrodes	B
Monopolar and Subdermal needle electrodes	B
Cup- and Ear electrodes	B, C, D, E, F
Grounding and Grounding Plate electrodes	B, G, I, J
Surface electrodes	G, J
Bipolar Felt Pad and Bar electrodes	H, I, J, K
Small Stimulating and Ring electrodes	H, J, L

A000488 B

English

INSTRUCTIONS FOR USE
DETAILED PRODUCT INFORMATION

Detailed product information/instructions:

- A) Cable required.
- B) Do not use these electrodes for stimulation of any kind.
- C) The electrodes must be fixed and/or applied by usage of a conductive EEG paste or collodion in combination with conductive gel suitable for medical use.
- D) The conductive gel is inserted by a syringe, with a blunted needle, through the hole on top of the electrodes.
- E) Read manufacturers instructions on preparation and removal of the collodion, gel and paste before and after application.
- F) Contact with impure metals, some Skin Prep's and Conductive Gel's may affect the chlorinated surface of the electrode cups.
- G) Water and electrolyte gel between the felt strap/plate and the recording or stimulating electrodes should be avoided and removed before application.
- H) Water and electrolyte gel between the poles should be avoided and removed before application.
- I) The felt should be soaked in a normal saline solution (leaving dry a minimum space of 2cm of the felt strap towards the attachment of the cable) for a few minutes and then a small amount of electrolyte gel should be rubbed in.
- J) The steel surfaced electrodes should be moistened by an electrolyte gel before and during application.
- K) Notice electrodes coding for the polarity. A red dot, molded into the stimulator, indicates the anode.
- L) Notice cable color for the polarity, red indicates the anode.

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