Sensors

Italiano

Sensors

Deutsch

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Item NA-N13XX - ver. 2.0 (Side A)

CE

To Reorder: Call: **1-800-356-0007 (in the USA)** or your Natus Neurology Incorporated distributor
or email: **supplies.nc@Natus.com**

Manufactured by S.L.P., Ltd. for Natus Neurology Incorporated. Made in Israel.

Fax: +972 3 5371282
Tel: +972 3 6870640
E-mail: **info@sleepsense.com**

ISRAEL
Tel Aviv, 6706016,
62 Mordcheai Anilevitz St.
2511 Technology Dr., Suite 112
The Square 12, Am Flughafen,
60549 Frankfurt am Main,
Germany

Arazy Group GmbH
S.L.P., Inc.

EC REP

Specifications are subject to change without notice
Les spécifications sont sujettes à modification sans préavis
Especificaciones sujetas a cambio sin previo aviso
Le specificazioni sono soggette a modifica senza preavviso
Die Spezifikationen sind ohne vorherige Meldung Änderungen unterworfen

Sensors

User Manual

Piezo-Crystal Respiration Effort Sensors

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Sensors

English

Thank you for purchasing a SleepSense® sensor.

Description

Piezo crystal respiration effort sensors convert chest or abdominal respiration motions to a small analog voltage that provides a clear, reliable indication of respiration waveforms. These sensors do not require a battery or interface.

Indications for use

SleepSense® sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system. They are intended for use on children and adult patients who are screened during sleep disorder studies at a sleep laboratory or the patient’s home.

Applying the Sensor

The patient should lie on his/her back for optimal positioning of the sensor.

Positioning the Sensor

- Select the proper size band for the patient.
- 1. Strap the sensor bands around the patient’s abdomen or chest, over the nightclothes. Make sure the sensors do not touch the skin.
- 2. Select the area on the chest/abdomen where the bands will not slide out of place during the night. The sensor box should be slightly off the center line of the body.
- * **When using double tab sensors, make a small “wave” of loose band under the sensor.**
- 3. Gently tighten the bands to secure them in place. There will be interference, (noise) with the signal if the bands are too tight or too loose.
- Tape the cable approximately five inches away from the sensor to reduce strain and enhance reliability.
- Check that clear, strong signals are being transmitted before leaving the patient.

Cleaning the Sensor

- Wipe the sensor and cable with a non-corrosive (to plastic) cleanser to clean before use.
- Make sure the complete sensor assembly is thoroughly dry before reusing it.
- Do not soak the sensor in disinfectants.

Technical Specifications:

Part type:	Piezo Crystal Sensor
Cable:	White insulation wire
Sensor:	Sensor / Buckles/ Double Loop/Velcro® Tabs
Recommended filter settings:	High pass: 0.1 Hz / Low pass: 70 Hz, (or as recommended in the system’s instructions).
Operating conditions:	5°C (40°F) - 40°C (104°F)
Storage temperature:	-20°C (-4°F) - 60°C (140°F)
Operating and storage humidity:	5% - 95% (Non-condensing)

!

Warnings and Precautions

- **Caution:** US Federal Law restricts this device to sale, distribution and use by or on the order of a physician.
- SleepSense sensors are for professional use only.
- SleepSense sensors may be used only in conjunction with an approved recording system.
- SleepSense sensors are only intended for use on healthy skin.
- Discontinue use if sensor shows signs of wear, damage, or exposed metal.

Limited Warranty
SLP warrants the respiration effort sensor to be free of defects in materials and workmanship for a period of twelve months from the date purchased. The sole liability of SLP and our Distributor(s) is limited to replacement or repair of the product at the option of SLP with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall SLP or our Distributor(s) be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This Warranty is intended only for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect, or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

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