Applications (Client)

Module: iECG

Revision: 20170407

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Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Barcode Review Print Assign Confirm Help Logout

Quick Search Advanced Search Order Search

Search Type

- Unconfirmed ECGs
- Confirmed ECGs
- ECGs without matching order
- ECGs for a specific patient
- ECGs assigned to a specific inbox
- ECGs for a specific location
- Other Studies

Institution: All Facility: All Department: All

Acquired within the last: 30 Weeks

☐ STAT
☐ Include other studies
☐ Include order status information

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Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Barcode Review Print Assign Confirm Help Logout

Quick Search | Advanced Search | Order Search

Search Type

- Unconfirmed ECGs
- Confirmed ECGs**
- ECGs without matching order
- ECGs for a specific patient
- ECGs assigned to a specific inbox
- ECGs for a specific location
- Other Studies

Confirmed by: All

Institution: All

Facility: All

Department: All

Acquired within the last: 4 Months

☐ Include other studies

☐ Include order status information

Search

Search Results

Select All Clear

ID	Name	Date	Type	Source	State	Severity
<input type="checkbox"/> P38913	John, Schwarzer JOE	07-Feb-2014 4:26:37 PM	Standard 12	PageWriter TC	Confirmed	Defective
<input type="checkbox"/> P36479	JONARA, EUSABIO JOE	10-Feb-2014 7:04:22 AM	Standard 12	PageWriter TC	Confirmed	Defective
<input type="checkbox"/> T12345	Jones, Tim	10-Feb-2014 6:34:04 AM	Standard 12	PageWriter TC	Confirmed	Defective
<input type="checkbox"/> t332211	mister, smith	10-Feb-2014 8:29:35 AM	Standard 12	PageWriter TC	Confirmed	Defective
<input type="checkbox"/> 987123	nog, eenkeer	09-Feb-2014 9:00:06 PM	Standard 12	PageWriter TC	Confirmed	Defective
<input type="checkbox"/> 123321	Reinders, Bert	07-Feb-2014 4:08:09 PM	Standard 12	PageWriter TC	Confirmed	Defective
<input type="checkbox"/> 998877	reinders, gilian	08-Feb-2014 10:01:09 AM	Standard 12	PageWriter TC	Confirmed	Defective
<input type="checkbox"/> 1223345	shoe, bottom	09-Feb-2014 6:40:29 PM	Standard 12	PageWriter TC	Confirmed	Defective
<input type="checkbox"/> 332211	testsidra, ds	07-Feb-2014 3:48:51 PM	Standard 12	PageWriter TC	Confirmed	Defective
<input type="checkbox"/> 332211	testsidra, ds	07-Feb-2014 2:27:22 PM	Standard 12	PageWriter TC	Confirmed	Defective
<input type="checkbox"/> t112233	weer, een	10-Feb-2014 8:11:14 AM	Standard 12	PageWriter TC	Confirmed	Defective

Patient List

11 Studies 10 Patient(s) 0 Selected Philips

Application client IntelliSpace ECG EMS Recherche rapide – ECG sans ordonnance

Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Barcode Review Print Assign Confirm Help Logout

Quick Search Advanced Search Order Search

Search Type

- Unconfirmed ECGs
- Confirmed ECGs
- ECGs without matching order**
- ECGs for a specific patient
- ECGs assigned to a specific inbox
- ECGs for a specific location
- Other Studies

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Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Barcode Review Print Assign Confirm Help Logout

Quick Search Advanced Search Order Search

Search Type

- Unconfirmed ECGs
- Confirmed ECGs
- ECGs without matching order
- ECGs for a specific patient**
- ECGs assigned to a specific inbox
- ECGs for a specific location
- Other Studies

Institution: All

Facility: All

Patient ID:

Last Name: emden

First Name:

☒ Include other studies

☐ Include order status information

☐ Include previous last names

Search

Search Results

Select All Clear

Emden, John van ID: 111111111111 DOB: 1956-01-01

- ☐ 05-Feb-2014 4:22:11 PM Standard 12 Defective Confirmed: Philips
- ☐ 03-Feb-2014 12:50:09 PM Standard 12 Defective Confirmed: Philips
- ☐ 03-Feb-2014 12:00:33 PM Standard 12 Defective New
- ☐ 31-Jan-2014 4:27:43 PM Standard 12 Defective Confirmed: Philips
- ☐ 03-Sep-2005 5:45:45 PM Mason-Likar Abnormal New (BERNHARD, CARL)

Emden, John van ID: 1001001001 DOB: 1990-07-30

- ☐ 20-Feb-2014 8:12:33 AM Standard 12 Defective New
- ☐ 31-Jan-2014 3:26:42 PM Standard 12 Defective Awaiting Confirm

Patient List

7 Studies 2 Patient(s) 0 Selected Philips

Application client IntelliSpace ECG EMS

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Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Barcode Review Print Assign Confirm Help Logout

Quick Search | Advanced Search | Order Search

Search Type

- Unconfirmed ECGs
- Confirmed ECGs
- ECGs without matching order
- ECGs for a specific patient
- ECGs assigned to a specific inbox**
- ECGs for a specific location
- Other Studies

Assigned To: to be verified

☒ Include other studies
☐ Include order status information

Search

Search Results

hpsink, Ivo ID: 11223344
☐ 06-Feb-2014 8:45:12 AM Standard 12 Defective Awaiting Review

ROBERTS, WILLIAM ID: 20163
☐ 29-Jan-2009 1:29:58 PM Stress No Severity Awaiting Review

Ryder, June ID: 01609266
☐ 08-Jan-2008 9:26:00 AM Holter No Severity Awaiting Review

Select All Clear

Patient List

3 Studies 3 Patient(s) 0 Selected Philips

Application client IntelliSpace ECG EMS

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Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Barcode Review Print Assign Confirm Help Logout

Quick Search Advanced Search Order Search

Search Type

- Unconfirmed ECGs
- Confirmed ECGs
- ECGs without matching order
- ECGs for a specific patient
- ECGs assigned to a specific inbox
- ECGs for a specific location**
- Other Studies

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Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Barcode Review Print Assign Confirm Help Logout

Quick Search Advanced Search Order Search

Search Type

- Unconfirmed ECGs
- Confirmed ECGs
- ECGs without matching order
- ECGs for a specific patient
- ECGs assigned to a specific inbox
- ECGs for a specific location
- Other Studies

Institution: Unidentified Institution ☐ Include order status information

Facility: Unidentified Facility ☐ Include previous last names

Department: Unidentified Department ☐ Include ECGs

Patient ID: Q%

Last Name:

Acquired within the last: 10 Years

Search

Search Results Select All Clear

ID	Name	Date	Type	Source	State	Severity
<input type="checkbox"/> QMRN-042112490014	ABDO, EUGENE J	14-Nov-2006 10:17:19 AM	Stress	Stress	New	No Severity
<input type="checkbox"/> QMRN-04-26-2007 10:28:42	04-26-2007 10:28:42	26-Apr-2007 10:28:44 AM	Stress	Stress	New	No Severity
<input type="checkbox"/> QMRN-0988996500024	06-07-2005 11:28:09, MOORE GEORGE	07-Jun-2005 11:28:13 AM	Stress	Stress	Awaiting Confirm	No Severity
<input type="checkbox"/> QMRN-108391490005	ABENILLA, DENNIS	11-Apr-2007 4:08:29 PM	Stress	Stress	New	No Severity
<input type="checkbox"/> QMRN-11-20-2007 15:23:40	Demo Patient 11-20-2007 15:23:40	20-Nov-2007 3:23:52 PM	Stress	Stress	New	No Severity

Patient List

5 Studies 5 Patient(s) 0 Selected Philips

Application client IntelliSpace ECG EMS Recherche avancée

Philips IntelliSpace ECG Enterprise Edition - Not For Clinical Use

File View Tools Help

Search Barcode Review Print Assign Confirm Help Logout

Quick Search **Advanced Search** Order Search

Assigned To: All Institution: All State: All User Field 1: User Field 2: User Field 3: User Field 4: User Field 5: User Field 6:

Patient ID: Facility: All Severity: All

Last Name: Department: All Source: All

First Name: Start Date: 10-Aug-2015 Type: All

Include previous last names ☐ End Date: 13-Aug-2015 Studies without matching order ☐

STAT ☐ Date Of Birth: 01-Aug-1969 Include order status information ☐

Search

Assigned To: Dr Bob's - Emergency

Patient ID: All

Last Name: Dr Bob's - Emergency Cases to be verified

First Name:

Include previous last names ☐

STAT ☐

Include previous last names ☐

STAT ☒

New Patient Worklist 1 Find Patient Edit Patient ID

STAT Yes

Patient ID 100

Last Name Doe

DOB (dd/mm/yyyy)

Gender

Department Name

Include previous last names ☒

STAT ☐

Last Name: BERN% Depa

First Name: Sta

Include previous last names ☒

STAT ☐

Search Results

ID	Name
<input type="checkbox"/> 123456789	BERNHARD, Carl
<input type="checkbox"/> 11111111111	Emden, John van
<input type="checkbox"/> 11111111111	Emden, John van
<input type="checkbox"/> 11111111111	Emden, John van
<input checked="" type="checkbox"/> 11111111111	Emden, John van (BERNHARD, CARL)

Application client IntelliSpace ECG EMS Recherche avancée

Philips IntelliSpace ECG Enterprise Edition - Not For Clinical Use

File View Tools Help

Search Barcode Review Print Assign Confirm Help Logout

Quick Search Advanced Search Order Search

Assigned To: All Institution: All State: All User Field 1: User Field 2: User Field 3: User Field 4: User Field 5: User Field 6:

Patient ID: Facility: All Severity: All

Last Name: Department: All Source: All

First Name: Start Date: 10-Aug-2015 Type: All

Include previous last names End Date: 13-Aug-2015 Studies without matching order

STAT Date Of Birth: 01-Aug-1969 Include order status information

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Application client IntelliSpace ECG EMS Recherche avancée

Philips IntelliSpace ECG Enterprise Edition - Not For Clinical Use

File View Tools Help

Search Barcode Review Print Assign Confirm Help Logout

Quick Search Advanced Search Order Search

Assigned To: All Institution: All State: All User Field 1: User Field 2: User Field 3: User Field 4: User Field 5: User Field 6:

Patient ID: Facility: All Severity: All

Last Name: Department: All Source: All

First Name: Start Date: 10-Aug-2015 Type: All

Include previous last names ☐ End Date: 13-Aug-2015

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Philips IntelliSpace ECG Enterprise Edition - Not For Clinical Use

File View Tools Help

Search Barcode Review Print Assign Confirm Help Logout

Quick Search Advanced Search Order Search

Assigned To: All Institution: All State: All User Field 1:

Patient ID: Facility: All Severity: All User Field 2:

Last Name: Department: All Source: All User Field 3:

First Name: Start Date: 10-Aug-2015 Type: All User Field 4:

Include previous last names ☐ End Date: 13-Aug-2015 Studies without matching order ☐ User Field 5:

STAT ☐ Date Of Birth: ☒ 01-Aug-1969 Include order status information ☐ User Field 6:

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The screenshot displays the Philips IntelliSpace ECG Universal Edition software interface. The title bar reads "Philips IntelliSpace ECG Universal Edition - Not For Clinical Use". The menu bar includes "File", "View", "Tools", and "Help". The toolbar contains icons for Search, Barcode, Review, Print, Assign, Confirm, Help, and Logout. Below the toolbar, there are tabs for "Quick Search", "Advanced Search", and "ADT Search". The "ADT Search" tab is active, showing a form with the following fields:

- Patient ID:
- Account #:
- Last Name:
- First Name:
- Gender:
- DOB:
- Start Date:
- End Date:
- Outbox:

A "Search" button with a magnifying glass icon is located to the right of the form fields. Below the form, there is a "Search Results" section with a large empty box. To the right of the search results box are buttons for "Deselect All" and "Clear". At the bottom of the window, there is a status bar with a "Patient" tab, a "List" button, and a "Searching..." label. On the right side of the status bar, it shows "0 Studies 0 Patient(s) 0 Selected" and the Philips logo.

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Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Barcode Review Print Assign Confirm Help Logout

Quick Search Advanced Search ADT Search

Assigned To: All Institution: All State: Deleted Severity: All Source: All Type: All User Field 1: User Field 2: User Field 3: User Field 4: User Field 5: User Field 6:

Patient ID: Facility: All Last Name: Department: All Start Date: 08-Jun-1990 End Date: 11-Jun-2014 Include previous last names STAT Include ADT information

Search Results

ID	Name	Date	Type	Source	State	Severity
<input type="checkbox"/> 999999	<No name>	05-Jul-2007 12:00:00 PM	Holter	Holter	Deleted	No Severity
<input type="checkbox"/> P36367	Adsan, Anita JOE	11-Nov-2013 10:10:28 AM	Unknown	PageWriter TC	Deleted	No Severity
<input type="checkbox"/> 614461	Ariel, Justus	08-Nov-2013 4:08:54 PM	Unknown	PageWriter TC	Deleted	No Severity
<input type="checkbox"/> 673516	Arjun, Brandon	07-Nov-2013 12:10:46 PM	Unknown	PageWriter TC	Deleted	No Severity
<input type="checkbox"/> 154205	Caden, Andrew	05-Nov-2013 10:43:16 AM	Unknown	PageWriter TC	Deleted	No Severity

Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Open... Export... Import... Restore... Exit

Print Assign Verify Edit Compare Confirm Analyze Summary

Name: Arjun, Brandon

11-Jun-2014 1:04:31 PM

HR: 0 P: 0 RR: 0 QRS: 0 PR: T: QRSd:

INTERPRETATION (CRITERIA: 0B)

Age not entered, assumed to be 50 years old for purpose of ECG in

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File View Tools Help

Open... Export Import... Restore... Exit

Print Assign Verify Edit Compare Confirm Analyze Summary Stmt Waveform Report Help Logout

Name: BEESON, BARBARA Location: 00000 - Unidentified Facility / 00 - Unidentified Depart

S: 10-Feb-2012 10:56:29 AM

1 / 7 Find

Bookmarks

- Summary
- Tabular Summary
- Worst Case
- Average Beat Summary
- Average Beat Summary
- Average Beat Summary
- Peak Exercise

Q-Stress Final Report

Institution ID: 2/10/2012 10:56:29 AM

Attending: AZAN, KENNETH N MD Referring: KAHN

Patient: BEESON, BARBARA
MRN: M000112668
DOB: 1/17/1942 Age: 70 HT: --
Sex: FEMALE WT: --

Medication: Beta Blocker in last 24 hours: Unspecified

Resting HR	82	Target HR	140	Max HR	136	Protocol	Bruce
Resting SBP	134	Max Predicted HR	140	Max SBP	188	HRxBP	25944
Resting DBP	83	% Max HR	99	Max DBP	78	Billing Code	
Worst-case ST Level		-0.2 mV	Total Exercise Time	04:01			
Worst-case ST Slope		-1 mV	METs(a)	7.0			

Reason for Test: Angina
Reason for Ending Test:

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Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Save Print Assign Verify Edit Compare Confirm Analyze **Summary** Stmt Waveform Report Help Logout

ID: \$\$\$438566 Name: JAFFEE, DILLON Location: 99999 - Main Campus / 00 - Unidentified Department

22-Feb-1992 1:15:13 PM* 30-Aug-1991 7:31:07 AM 24-Aug-1991 1:34:43 PM 21-Aug-1991 4:57:28 AM

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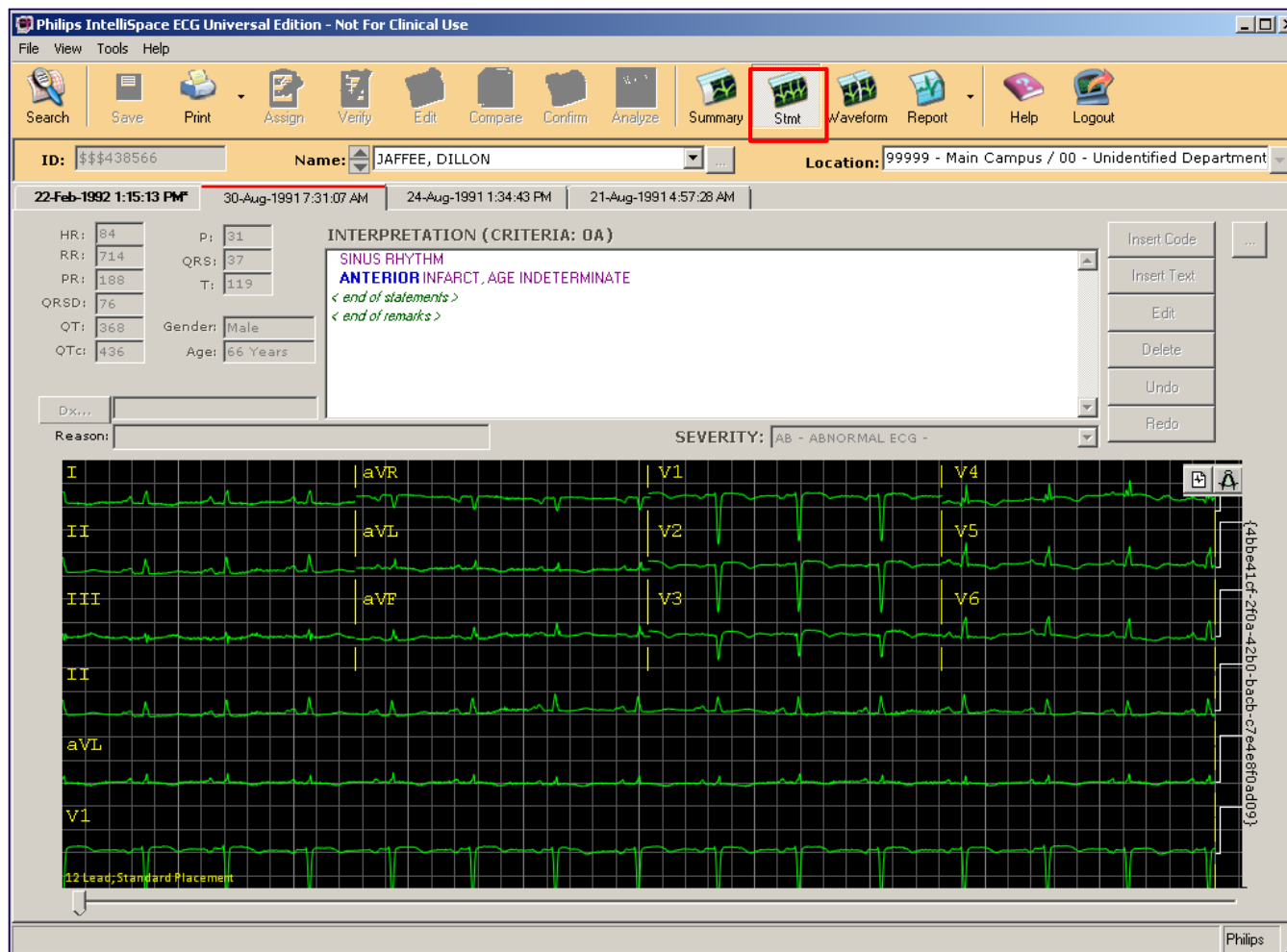
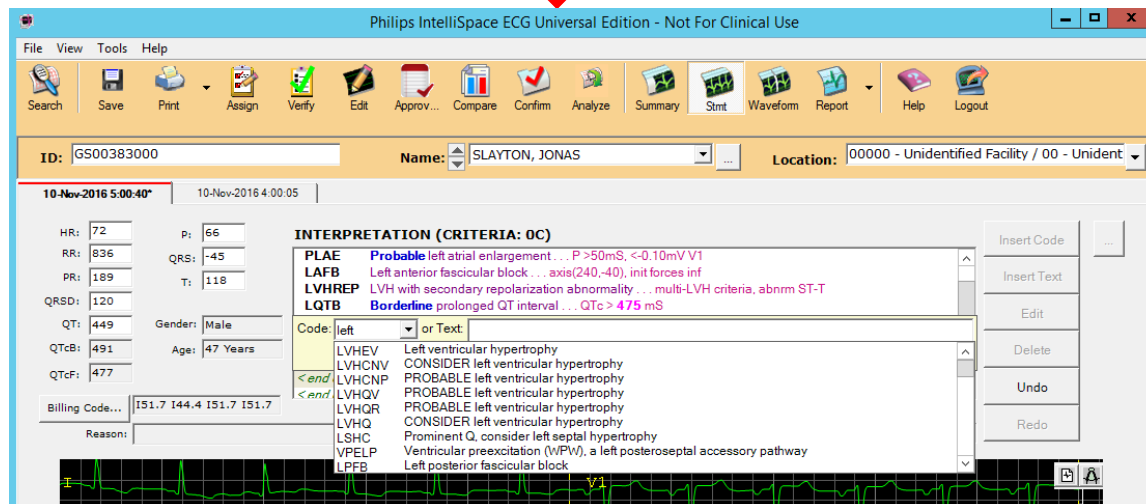
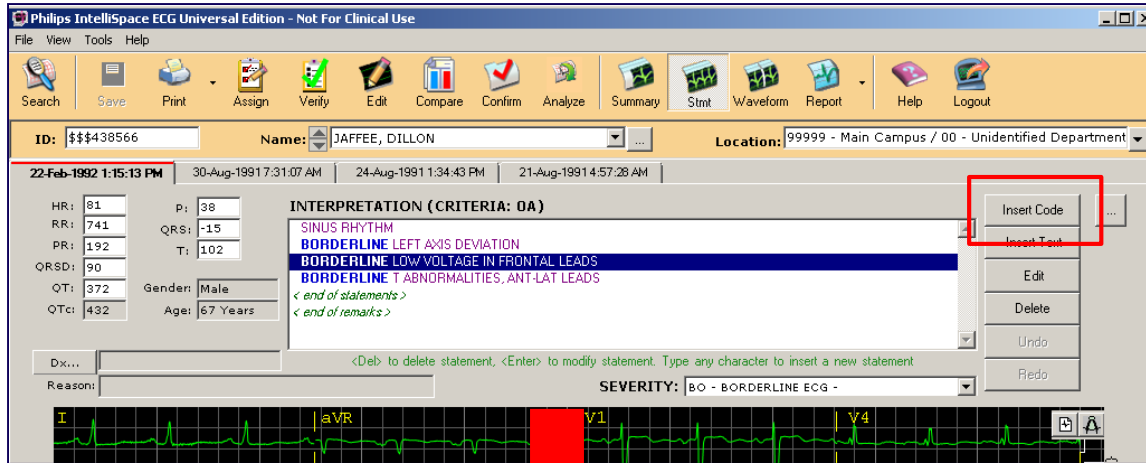
Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Save Print Assign Verify Edit Approv... Compare Confirm Analyze Summary Stmt Waveform Report Help Logout

ID: GS00383000 Name: SLAYTON, JONAS Location: 00000 - Unidentified Facility / 00 - Unident

10-Nov-2016 5:00:40* 10-Nov-2016 4:00:05

HR: 72 P: 66
RR: 836 QRS: -45
PR: 189 T: 118
QRS: 120
QT: 449 Gender: Male
QTcB: 491 Age: 47 Years
QTcF: 477

Billing Code... I51.7 I44.4 I51.7 I51.7 Reason:

INTERPRETATION (CRITERIA: 0C)

SR Sinus rhythm... normal P axis, V-rate 50 - 99
PLAE Probable left atrial enlargement... P >50ms, < <end of statements >
LAFB Left anterior fascicular block... axis(240,-40), i <end of remarks >
LVHREP LVH with secondary repolarization abnormality
LQTB Borderline prolonged QT interval... QTc > 47

Stmt Description

ANLER2 New conduction abnormality
VFIB Significant repolarization changes
UNKBIG 01234567891123456789212345678931234567894123456789512345678961
TWRV ** NO QRS COMPLEXES FOUND, NO ECG ANALYSIS POSSIBLE**
SINJ Ventricular fibrillation
REPT6D Bigeminal pattern, uncertain mechanism
QESPMI Sinus tachycardia with irregular rate
PLMP SEPTAL INJURY PATTERN
OSBW Abnormal T, suspect prox. LAD occlu. or CVA
NSTT EARLY TRANSITION INCREASED R/S RATIO IN V1, CONSIDER POSTERIOR
MFAT Misplaced precordial electrode(s)
LQT Osborn wave suggests hypothermia
JTRI NONSPECIFIC ST AND T-WAVE ABNORMALITY
ILMIQP Ectopic atrial tachycardia, multifocal
HPOCA Prolonged QT interval
AGMUNK Junctional rhythm with VPCs in a trigeminal pattern
FASCR PROBABLE inferolateral infarct, age indeterm
FASCR Prolonged QT interval suggests hypocalcemia
FASCR Age and gender not entered, assume ****yo male for purpose of ECG interpretation
FASCR Fascicular rhythm

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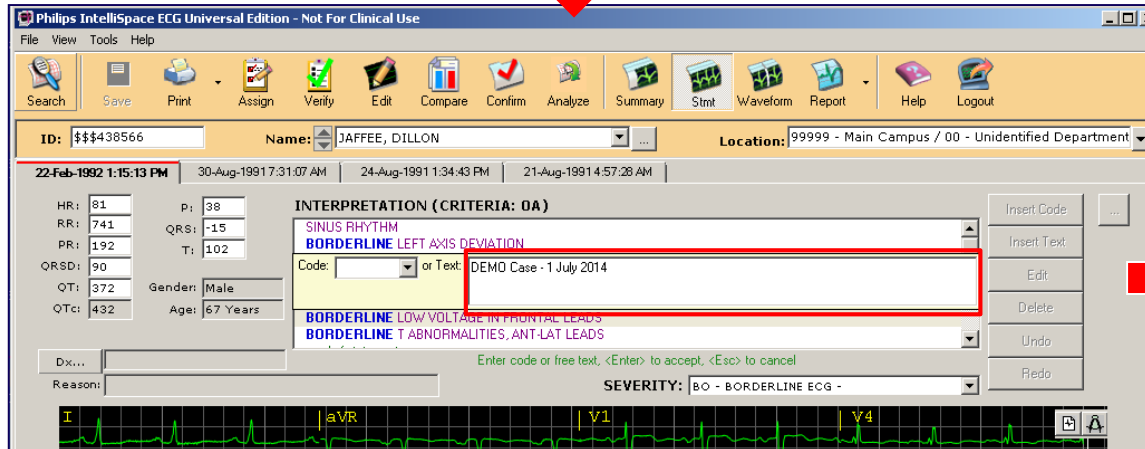
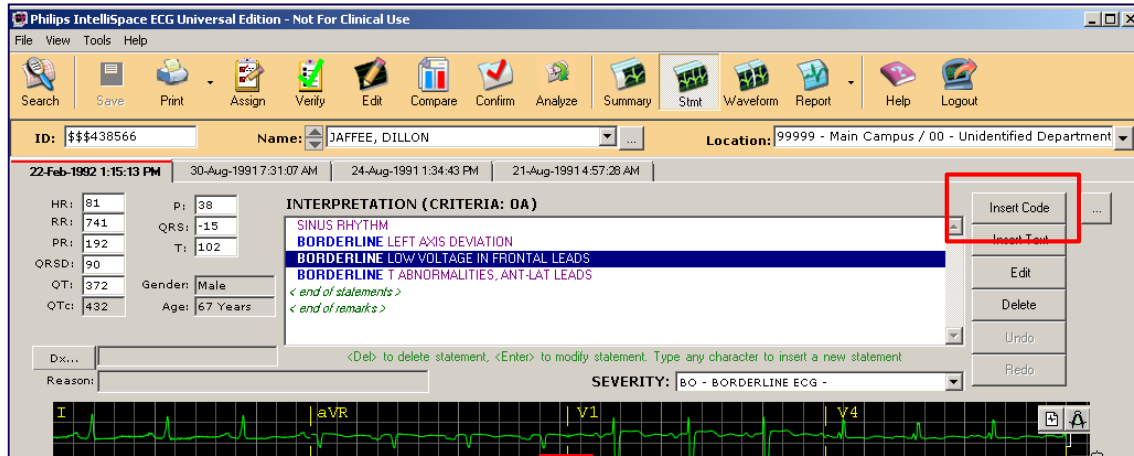
BORDERLINE LEFT AXIS DEVIATION
 DEMO Case - 1 July 2014
 BORDERLINE LOW VOLTAGE IN FRONTAL LEADS
 BORDERLINE T ABNORMALITIES, ANT-LAT LEADS
 < end of statements >

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Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Save Print Assign Verify Edit Compare Confirm Analyze Summary Stmt Waveform Report Help Logout

ID: \$\$\$438566 Name: JAFFEE, DILLON Location: 99999 - Main Campus / 00 - Unidentified Department

22-Feb-1992 1:15:13 PM 30-Aug-1991 7:31:07 AM 24-Aug-1991 1:34:43 PM 21-Aug-1991 4:57:28 AM

HR: 81 P: 38
RR: 741 QRS: -15
PR: 192 T: 102
QRSd: 90
QT: 372 Gender: Male
QTc: 432 Age: 67 Years

INTERPRETATION (CRITERIA: 0A)

SINUS RHYTHM
BORDERLINE LEFT AXIS DEVIATION
BORDERLINE LOW VOLTAGE IN FRONTAL LEADS
BORDERLINE T ABNORMALITIES, ANT-LAT LEADS
< end of statements >
< end of remarks >

Dx... to delete statement, <Enter> to modify statement. Type any character to insert a new statement.

Reason:

SEVERITY: BO - BORDERLINE ECG -

Insert Code
Insert Text
Edit
Delete
Undo
Redo

Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Save Print Assign Verify Edit Compare Confirm Analyze Summary Stmt Waveform Report Help Logout

ID: \$\$\$438566 Name: JAFFEE, DILLON Location: 99999 - Main Campus / 00 - Unidentified Department

22-Feb-1992 1:15:13 PM 30-Aug-1991 7:31:07 AM 24-Aug-1991 1:34:43 PM 21-Aug-1991 4:57:28 AM

HR: 81 P: 38
RR: 741 QRS: -15
PR: 192 T: 102
QRSd: 90
QT: 372 Gender: Male
QTc: 432 Age: 67 Years

INTERPRETATION (CRITERIA: 0A)

BORDERLINE LEFT AXIS DEVIATION
BORDERLINE LOW VOLTAGE IN FRONTAL LEADS
BORDERLINE T ABNORMALITIES, ANT-LAT LEADS
< end of statements >
< end of remarks >

Remark: Demo Remark

Dx... Edit free text, <Enter> to accept, <Esc> to cancel

Reason:

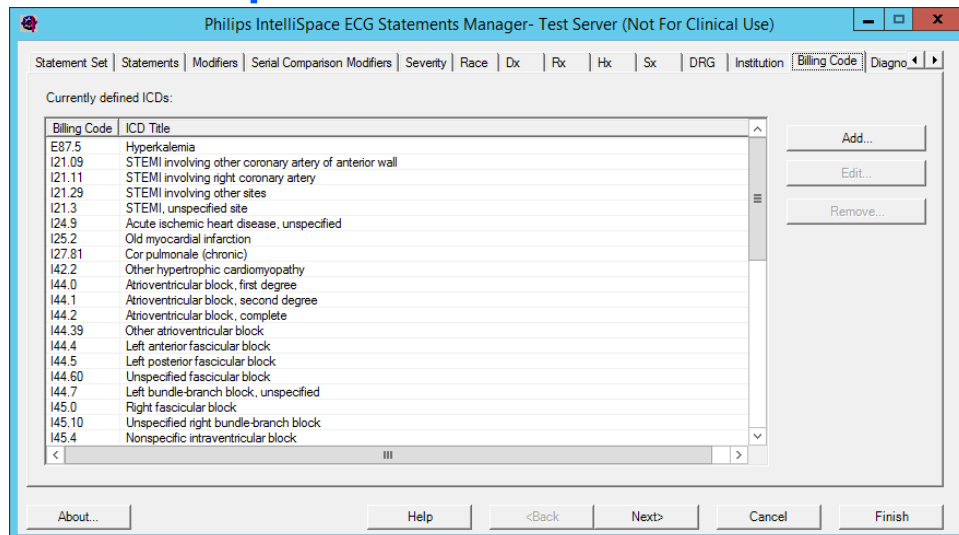
SEVERITY: BO - BORDERLINE ECG -

Insert Code
Insert Text
Edit
Delete
Undo
Redo


BORDERLINE LEFT AXIS DEVIATION
* DEMO Case - 1 July 2014
BORDERLINE LOW VOLTAGE IN FRONTAL LEADS
BORDERLINE T ABNORMALITIES, ANT-LAT LEADS
< end of statements >
* Demo Remark
< end of remarks >

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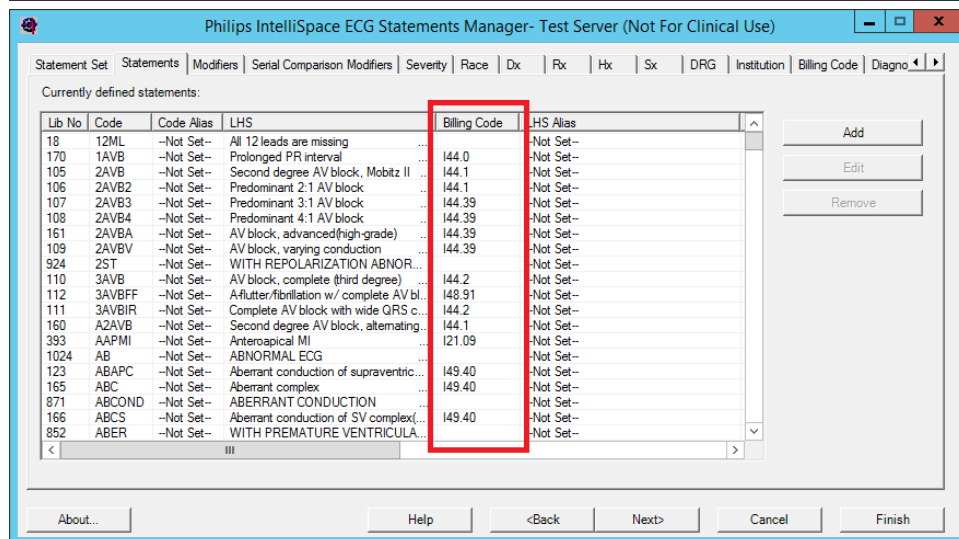
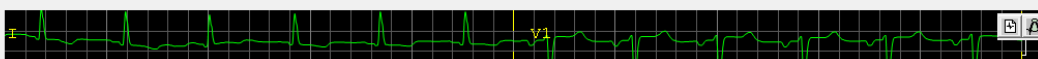
GS00383000 SLAYTON, JONAS 10-Nov-2016 5:00:40
Age: 47 Years Male Unidentified Institution (000)
Unidentified Facility (00000)
Unidentified Department (00)

Billing Code: I51.7, I44.4, I51.7, I51.7

HR 72 . Sinus rhythm
RR 836 . Probable left atrial enlargement
PR 189 . Left anterior fascicular block
QRSd 120 . LVH with secondary repolarization abnormality
QT 449 . Borderline prolonged QT interval
QTcB 491
QTcF 477
-- AXIS --
P 66
QRS -45
T 118

- ABNORMAL ECG -

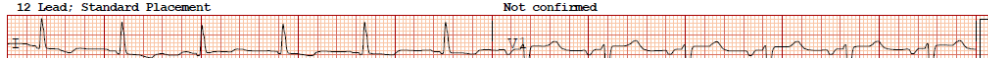Previous Study: 10-Nov-2016 04:00:05 - Abnormal Confirmed
12 Lead: Standard Placement Not confirmed

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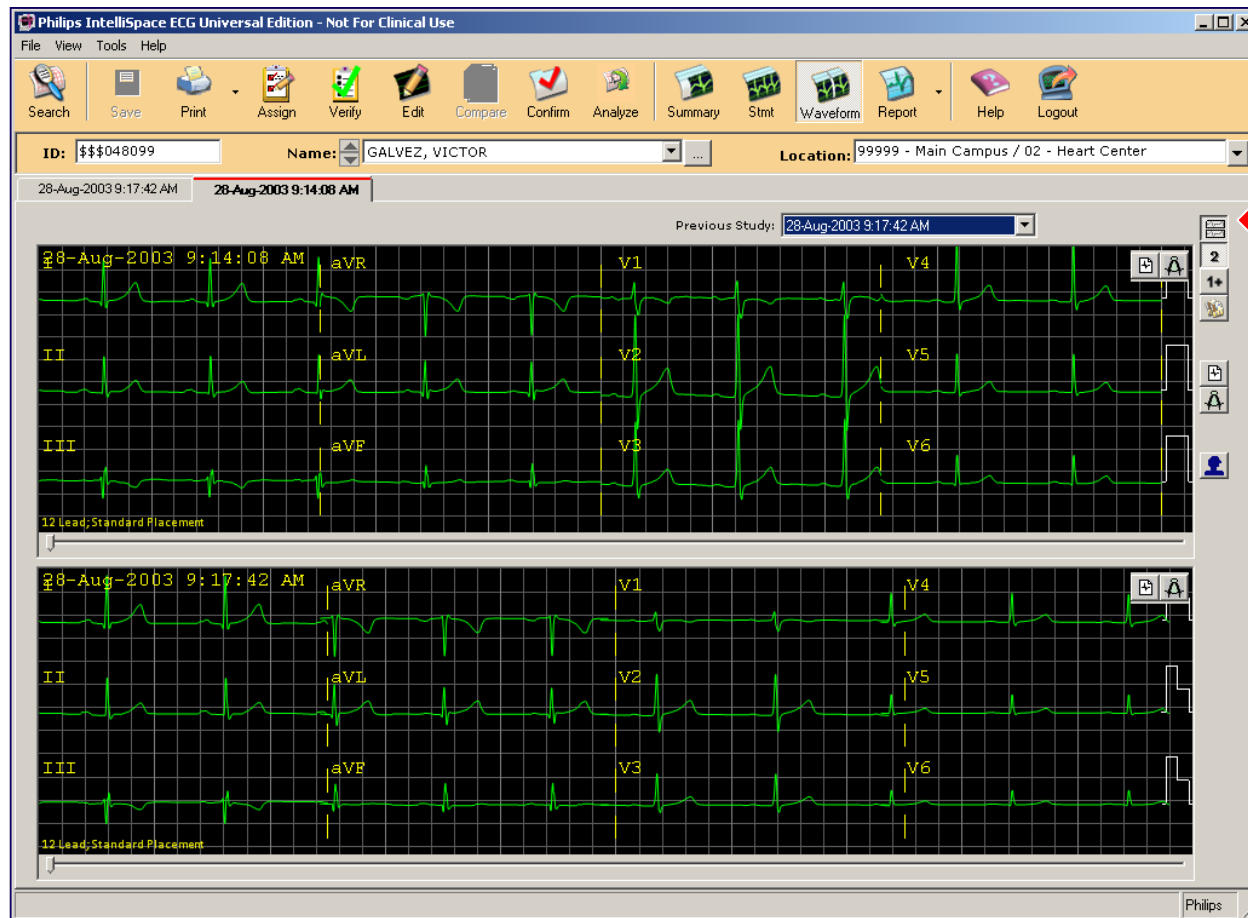
Philips IntelliSpace ECG Universal Edition - Not For Clinical Use

File View Tools Help

Search Save Print Assign Verify Edit Compare Confirm Analyze Summary Stmt Waveform **Report** Help Logout

ID: \$\$\$1234 Name: HILDEBRAND, BEN Location: Heart Center

04-Feb-2004 10:11:32 AM 04-Feb-2004 10:01:21 AM 04-Feb-2004 9:56:14 AM

1 / 1 Find

\$\$\$1234 HILDEBRAND, BEN 04-Feb-2004 10:01:21 AM

Tutorial Database - Main Campus - Heart Center (999-99999-02)

Rhythm Analysis

GROUP MEASUREMENTS		Group
Member Count	9	1
Member %	100	
Longest Run	9	
Mean QRS Duration	76	
Low Ventr Rate	60	
Mean Ventr Rate	62	
High Ventr Rate	64	
V-Rate Std. Dev.	1	
Mean RR Interval	968	
Mean Atrial Rate	60	
A-Rate Std. Dev.	0	
Avg P Count	1	
# Not Avg P Beats	0	
Low PR Interval	164	
Mean PR Interval	168	
High PR Interval	168	
PR Int. Std. Dev	4	
Mean PR Segment	68	
Mean QT Interval	392	
Mean QT Segment	316	
Comp. Pause Count	0	

GLOBAL RHYTHM PARAMETERS	
Atrial Rate	60
Low Ventr Rate	60
Mean Ventr Rate	62
High Ventr Rate	64
Flut-Fib Indicator	0
Fixed Mult P Morph	Yes
Mult P Test Valid	Yes
Paced Beats Measrd	No
Delta Wave Count	0 (0 4)
Bigeminy Count	0 (0)
Trigeminy Count	0 (0)
Wenckebach Count	0 (0)

RHYTHM GROUPING OF BEATS

1 1 1 1 1 1 1 1

ECGOTIC RHYTHM

PACEMAKER:

Mode:

Malfunction:

MISC:

GROUP FLAGS

Atrial Pace	No
Ventr. Pace	No
Interpolated Beat	No
Sinus Arrest	No
PR Progress Longer	No
Wenckebach	No
Bigeminy	No
Trigeminy	No
Aberrant Shape	No
Multifocal	No
Mult. P Test Done	Yes
QRS Measured	Yes

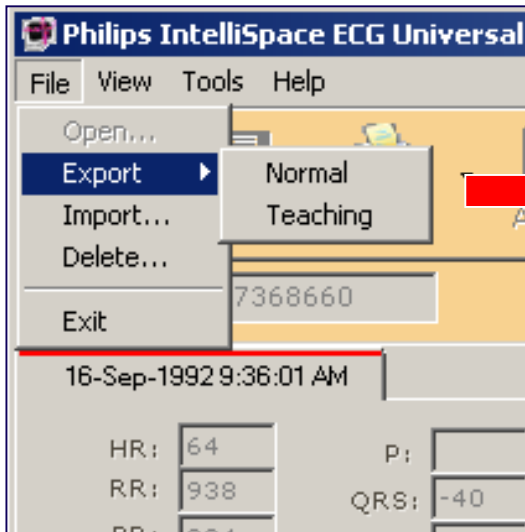
Device: Speed: 25 mm/sec Lead: 10 mm/mV Chest: 10 mm/mV 60- 0.15-150 Hz P0000A P7

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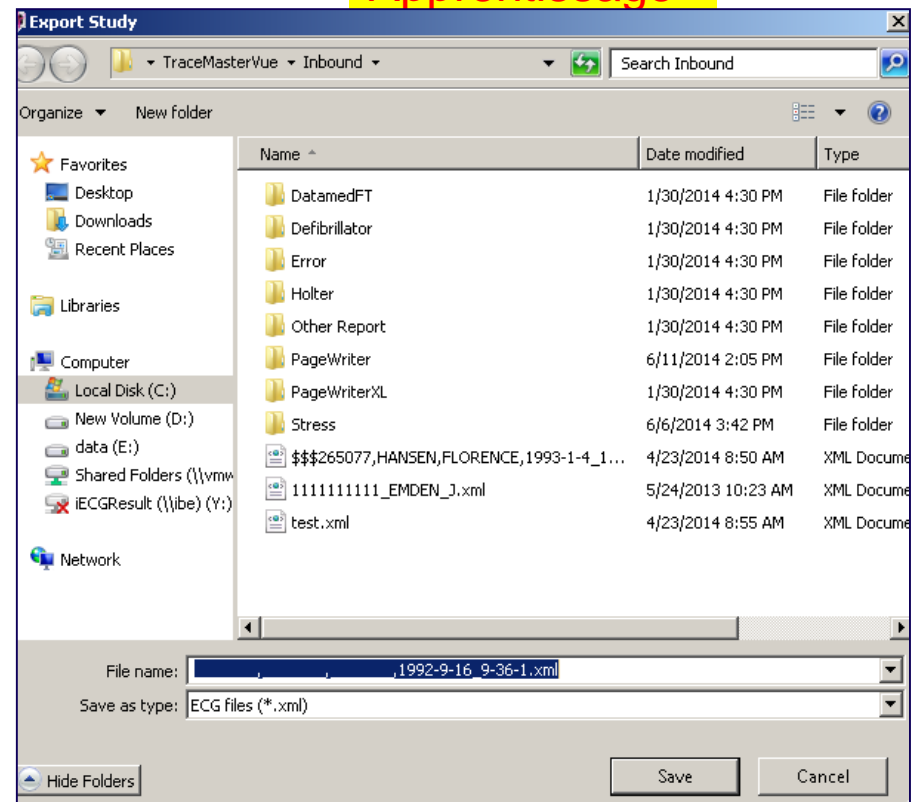
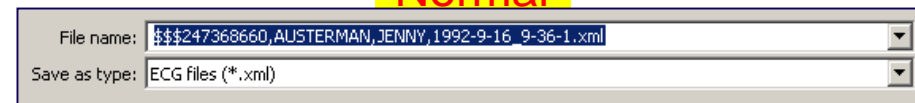
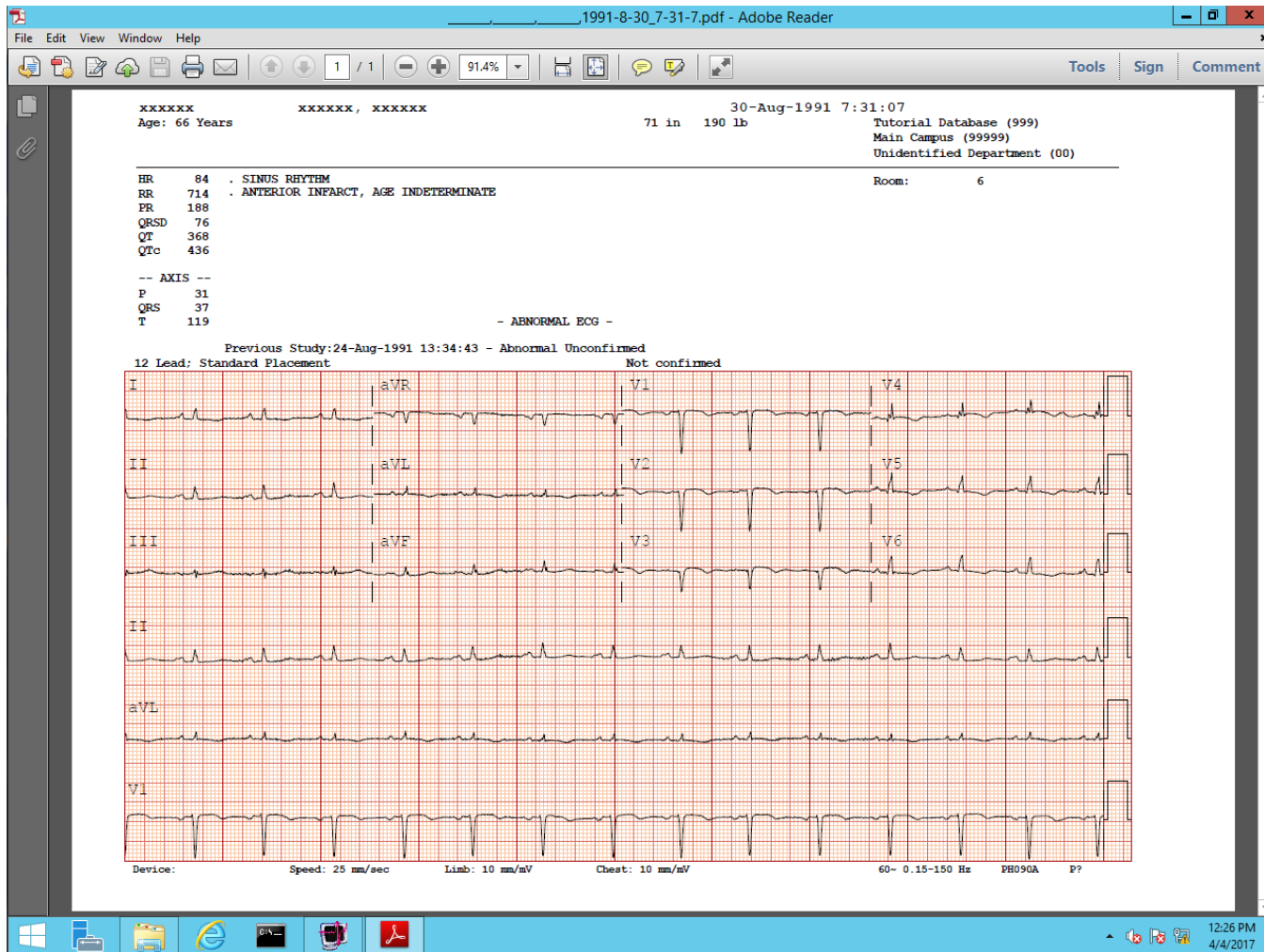
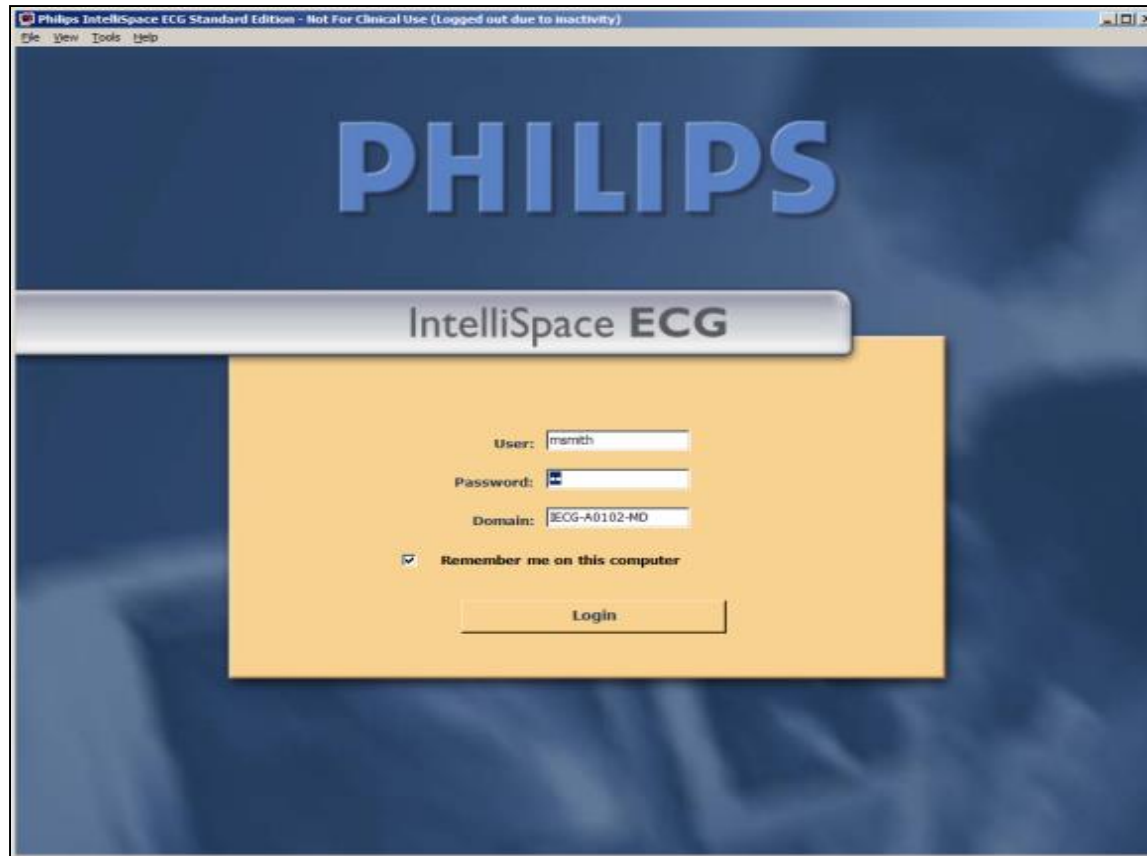
Client Web IntelliSpace ECG Anywhere

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The screenshot shows the 'IntelliSpace ECG Anywhere' web application. The 'Report Search' tab is active. The search filters on the left include 'Assigned To' (ESmith), 'Approved by' (<All>), 'Institution' (<All>), 'Facility' (<All>), 'Department' (<All>), and 'Acquired within the last:' (30 Years). The search results table shows the following data:

Patient Name	ID	Date/Time	Status	Department	Assigned To	Gender	DOB
ADDISON, LORENZO	ID:SS56756645	23-May-1989 6:05:35	Abnormal	Internal Medicine	Assigned To ESmith	Unknown	
ALLEN, EDWARD	ID:GS00314041	08-Nov-2016 22:00:30	Abnormal	Unidentified Department	Assigned To ESmith	Male	1928-06-25
		23-Jun-2006 11:11:28	Abnormal	Unidentified Department	Assigned To ESmith		
		18-Jun-2006 11:40:30	Abnormal	Unidentified Department	Assigned To ESmith		
BARRYMORE, DAVID	ID:GS00409661	08-Jun-2006 10:45:11	Abnormal	Unidentified Department	Assigned To ESmith	Male	
		08-Jun-2006 7:04:31	Abnormal	Unidentified Department	Assigned To ESmith		
		07-Jun-2006 7:13:06	Abnormal	Unidentified Department	Assigned To ESmith		
		06-Jun-2006 7:20:37	Abnormal	Unidentified Department	Assigned To ESmith		
BARTS, TERRY	ID:GS00188567	26-May-2006 7:40:16	Abnormal	Unidentified Department	Assigned To ESmith	Female	1935-06-04
		19-May-2006 7:40:39	Abnormal	Unidentified Department	Assigned To ESmith		

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
The screenshot displays the 'IntelliSpace ECG Anywhere' web application. The 'Patient Search' tab is active. On the left, there are input fields for 'Institution' (set to '<All>'), 'Facility' (set to '<All>'), 'MRN' (empty), 'Last Name' (filled with 'jaffee'), 'First Name' (empty), and 'Date of Birth' (placeholder 'mm/dd/yyyy'). A green 'Search' button is at the bottom of these fields.

The top right of the interface shows a summary bar with the following counts: 'All reports' (4), 'Stat' (0), 'Abnormal' (4), 'Normal' (0), and 'Approved' (0). There are 'Review' and 'Confirm' buttons next to these counts.

The search results are displayed in a table. The first result is for 'JAFEE, DILLON' with ID 'SSS438566'. The table lists four reports for this patient, all with a status of 'Abnormal' and assigned to 'ESmith'. The last report is marked as 'Confirmed'.

Report ID	Status	Department	Action
22-Feb-1992 13:15:13	Borderline	Unidentified Department	Assigned To ESmith
30-Aug-1991 7:31:07	Abnormal	Unidentified Department	Assigned To ESmith
24-Aug-1991 13:34:43	Abnormal	Unidentified Department	Assigned To ESmith
21-Aug-1991 4:57:28	Abnormal	Unidentified Department	Confirmed

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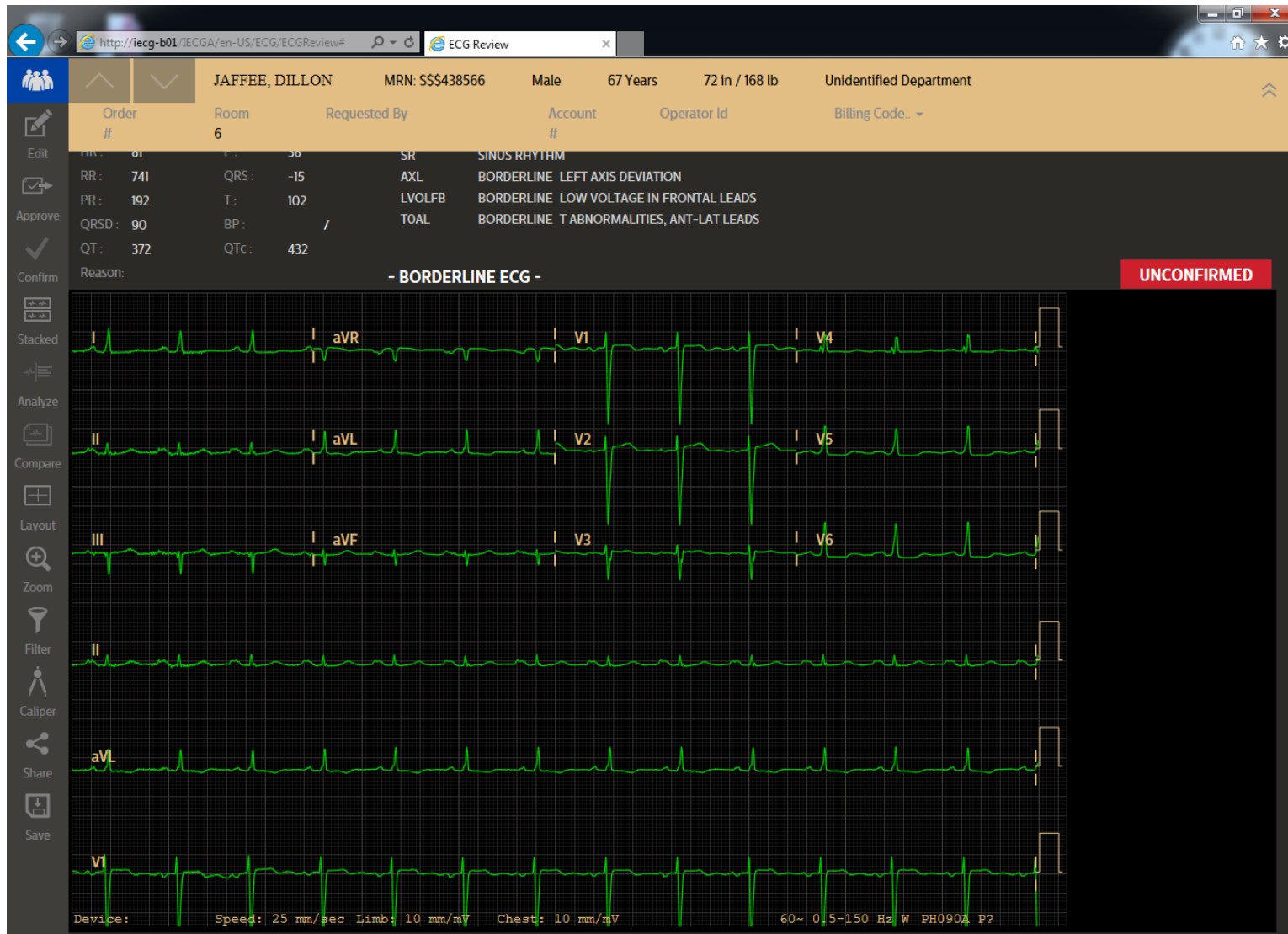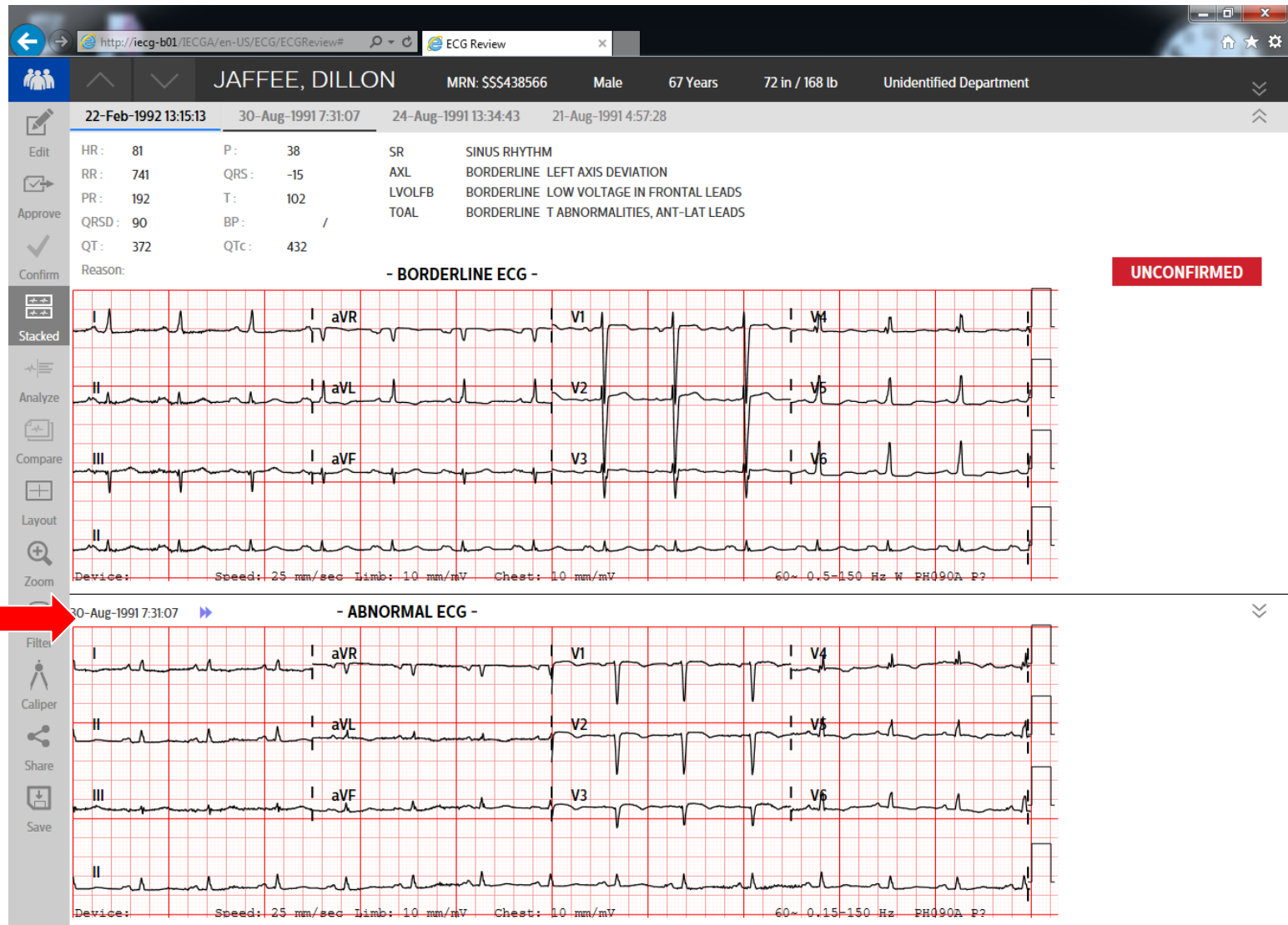
16-May-2006 10:55:49 11-May-2006 9:47:04

HR: 120 P: 50 ST Sinus tachycardia
RR: 500 QRS: 50 MVSPC Multiple premature complexes, vent & supraventricular
PR: 160 T: LVHSR Consider left ventricular hypertrophy
QRSD: 84 BP: / STELVH Anterior ST elevation, probably due to LVH
QT: 311 QTc: 440 COMPG When compared with ECG of 11-May-2006 9:47:04,
SCRHY Significant change in rhythm
Reason: - ABNORMAL ECG -

UNCONFIRMED

Layout

3 X 4 3 X 4 1R 3 X 4 3R
6 X 2 6 X 2 1R 12 X 1
Pan 12
Lead Sequence
Standard
Time Sequential
Lead Groups
II aVF V5
Rhythm
II
I
I

Speed(mm/sec):
25 50
Limb Scale(mm/mV):
2.5 5 10 20
Chest Scale:
Full Half

Cancel Done

G Review

MRN: GS0022977 Male 88 Years Unidentified Department

Sinus tachycardia
Multiple premature complexes, vent & supraventricular
Consider left ventricular hypertrophy
Anterior ST elevation, probably due to LVH
When compared with ECG of 11-May-2006 9:47:04,
Significant change in rhythm

UNCONFIRMED

IAL ECG -

11-May-2006 9:47:04 - NORMAL ECG -

Device: Speed: 25 mm/sec Limb: 10 mm/mV Chest: 5 mm/mV

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The screenshot displays the iECG Anywhere software interface. At the top, patient information is shown: JAFFE, DILLON, MRN: 555438566, 66 Years, 71 in / 190 lb, Unidentified Department. Below this, a timeline of dates and times is visible: 22-Feb-1992 13:15:13, 30-Aug-1991 13:07, 4-Aug-1991 13:34, and 21-Aug-1991 04:57:28. On the left, vital signs are listed: HR: 84, P: 31, RR: 714, QRS: 37, PR: 188, T: 119, QRSD: 76, BP: 0 / 0, QT: 368, QTc: 436. The main area contains a list of ECG statements with edit controls (X, ^, M) next to them. The statements are: SINUS RHYTHM, ANTERIOR INFARCT, AGE INDETERMINATE, and <end of statements>. At the bottom right, there is a SEVERITY dropdown menu set to - ABNORMAL ECG - and buttons for Undo and Redo.

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Code: left v or Text:

LVHQP PROBABLE left ventricular hypertrophy

LVHRS CONSIDER left ventricular hypertrophy

<end> LVHQ CONSIDER left ventricular hypertrophy

<end> LVHQR PROBABLE left ventricular hypertrophy

LVHVA PROBABLE left ventricular hypertrophy

LVHVAQ Left ventricular hypertrophy

LVHSR CONSIDER left ventricular hypertrophy

LVHTA CONSIDER left ventricular hypertrophy

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☐ Confirm once ☒ Enable batch confirm

☒ Enable 'Remember Me' option

☐ Re-authentication required for ECG confirmation

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☐ Confirm once ☒ Enable batch confirm

☒ Enable 'Remember Me' option

☒ Re-authentication required for ECG confirmation

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
The screenshot displays the Philips iECG Anywhere software interface. At the top, patient information is shown: JAFFEE, DILLON, MRN: SSS438566, Male, 66 Years, 71 in / 190 lb, Unidentified Department. Below this, a timeline of ECG recordings is visible: 22-Feb-1992 13:15:13, 30-Aug-1991 07:31:07 (selected), 24-Aug-1991 13:34:43, and 21-Aug-1991 04:57:28. The left sidebar contains buttons for Edit, Confirm, and A/B. The main area shows ECG parameters: HR: 84, P: 31, RR: 714, QRS: 37, PR: 188, T: 119, QRSD: 76, BP: 0 / 0, QT: 368, QTc: 436. A text entry field on the right contains the following text: SINUS RHYTHM, ANTERIOR INFARCT, AGE INDETERMINATE, Adding in a free form text statement, and <end of statements>. The SEVERITY dropdown is set to ABNORMAL ECG. Buttons for Undo and Redo are also present.

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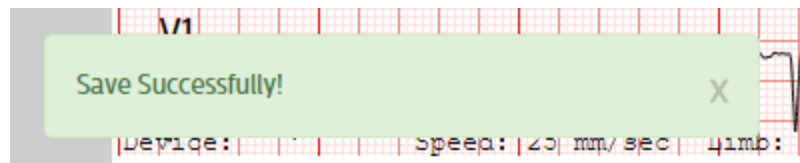Search Type

- Unconfirmed ECGs
- Approved ECGs
- Confirmed ECGs
- ECGs without matching order
- ECGs for a specific patient
- ECGs assigned to a specific inbox**
- ECGs for a specific location
- Other Studies

Assigned To: ESmith

☐ Only approved ECG

☐ Include other studies

☐ Include order status information

Search

Search Results Select All Clear

BRUGER, RICHARD ID: GS00110864

- ☐ 09-Nov-2016 6:00:12 Standard 12 Abnormal Awaiting Review
- ☐ 08-Nov-2016 23:00:33 Standard 12 Abnormal Awaiting Review

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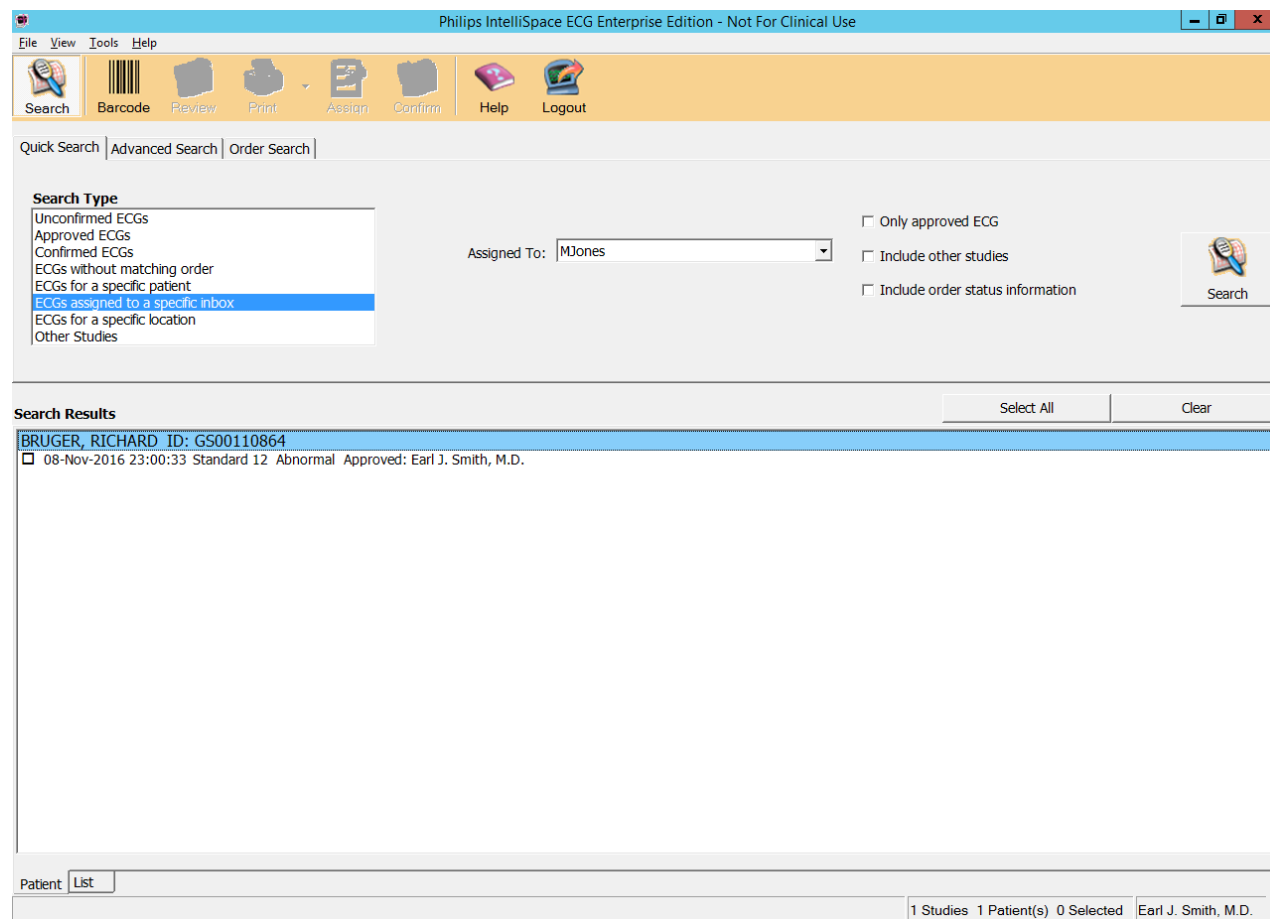
The screenshot displays the Philips IntelliSpace ECG Enterprise Edition software interface. The window title is "Philips IntelliSpace ECG Enterprise Edition - Not For Clinical Use". The interface includes a menu bar (File, View, Tools, Help) and a toolbar with various icons for search, save, print, assign, verify, edit, unapprove, uncompress, confirm, analyze, summary, stmt, waveform, report, help, and logout.

Key information displayed includes:

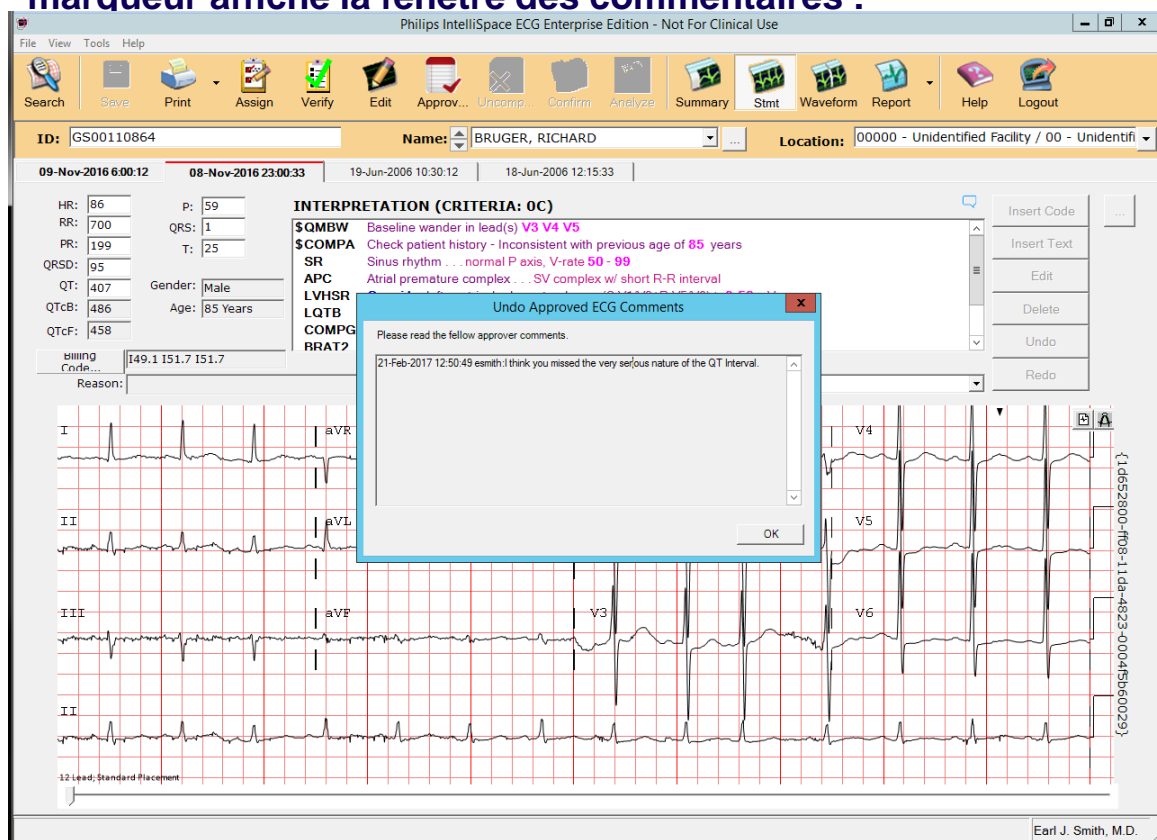
- ID:** GS00110864
- Name:** BRUGER, RICHARD
- Location:** 00000 - Unidentified Facility / 00 - Unidenti
- Interpretation (Criteria: 0C):**
 - \$QMBW Baseline wander in lead(s) V3 V4 V5
 - \$COMPA Check patient history - Inconsistent with previous age of 85 years
 - SR Sinus rhythm . . . normal P axis, V-rate 50 - 99
 - APC Atrial premature complex . . . SV complex w/ short R-R interval
 - LVHRS Consider left ventricular hypertrophy . . . (S V1/V2+R V5/V6) > 3.50 mV
 - LQTB Borderline prolonged QT interval . . . QTc > 475 mS
 - COMPG When compared with ECG of 19-Jun-2006 10:30:12 ,
 - BRAT2 Significant rate decrease . . . rate less than previous (>20 bpm)
- ECG Waveform:** A 12-lead ECG waveform is displayed at the bottom, showing leads I, II, III, aVR, aVL, aVF, and V1 through V6.

A dialog box titled "Undo Approved ECG Comments" is open in the foreground. It contains the text: "Please enter a reason or comments to undo approved ECG." and a text area with the input: "I think you missed the very serious nature of the QT interval". The dialog box has "OK" and "Cancel" buttons.

The bottom right corner of the software window shows the name "Earl J. Smith, M.D.".

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The screenshot shows the IntelliSpace ECG Anywhere web application. The browser address bar displays `http://localhost/IECGA/en-US/Search/Home`. The application header includes the title "IntelliSpace ECG Anywhere" and a "Logout" button. The main interface is divided into a left sidebar with search filters and a main content area displaying search results.

Search Filters (Left Sidebar):

- Assigned To: ESmith
- Approved by: <All>
- Institution: <All>
- Facility: <All>
- Department: <All>
- Acquired within the last: 30 Years
- Search button
- Save as default link

Search Results (Main Content Area):

Report Search | Patient Search <<

Summary: All reports 1 | Stat 0 | Abnormal 1 | Normal 0 | Approved 0

Buttons: Review | Confirm

1 Patients / 1 Reports

Patient ID	Date	Status	Department	Assigned To	Gender
BRUGER, RICHARD ID: GS00110864	09-Nov-2016 6:00:12	Abnormal	Unidentified Department	Assigned To ESmith	Male

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