Entropy EasyFit Sensor

Disposable, pkg of 25 pcs

REF M1174413

Simboly

Refer to the separate leaflet 2087478-001 located within the product package for an explanation of symbols.

Intended use


Entropy EasyFit Sensor is intended to be used for adults and pediatric patients aged from 2 years with GE Entropy measurement devices to enable recording of physiological signals such as EEG. To connect the sensor to the measurement device, use the GE Entropy Cable.

Expected service life

Expected service life is the time period during which the sensor or cable is expected to remain safe to use. The following criteria can be used to determine the end of the expected service life:

- The sensor has been used on a single patient during a single procedure.
- 24 hours has elapsed after opening of the sensor pouch.
- The sensor has passed its expiration date that is on the pouch.

Instructions for use (see Figures 1 to 6)

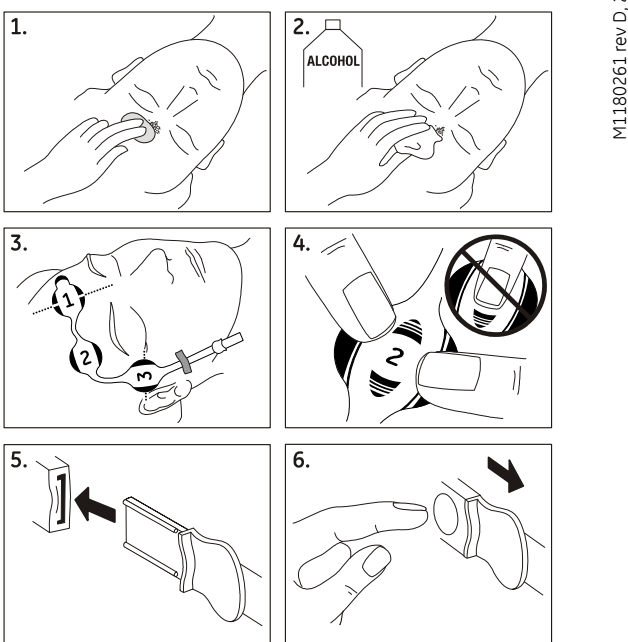1. Prepare the skin by rubbing it lightly with the preparation pad included in the sensor package.
2. Wipe the skin with alcohol and let dry.
3. Apply the sensor on the forehead. Position electrode number 1 in the center of the forehead, approximately 2 cm above the hairline.
4. Insert the sensor into the GE Entropy Cable (REF M1050784) until fully engaged.
5. Do not press the electrodes at the center, but on the edges to ensure adhesion. Insert the sensor tab into the GE Entropy Cable (REF M1050784) until fully engaged.
6. After disconnecting the sensor, press the release button on the GE Entropy Cable while pulling the sensor from the tab.
Please refer to the user documentation of the Entropy measurement monitor for details information.

Maintenance
For single use only – do not reuse. Reuse may pose a risk of infection and/or might result in measurement error.
Dispose of this sensor and its packaging in accordance with the local environmental and waste disposal regulations.

Contraindications
This sensor should never be applied to a skin surface that has been damaged by friction, laceration, incision, or burning. The use of the sensor on irritated, chapped, or inflamed skin is contraindicated. This sensor is not MRI compatible.

Warnings
Use only supplies and accessories approved by GE for proper and accurate function of the measurement. Using components other than the ones approved in the device manual might result in a measurement error.
If a skin rash or other unusual symptoms develop, stop use and remove the sensor. Continued use might cause skin damage.
Make sure that the electrodes, sensor and connectors do not touch any electrically conductive material, including earth. Patient grounding may disturb other monitored parameters such as EEG or might result in a risk when defibrillating the patient.
Do not touch the electrodes, table, bed, instruments, modules or the monitor during defibrillation. Touching the mentioned parts of the system might predispose the operator to defibrillation energy.

Ensure that the sensor is placed on the patient's forehead as instructed. Placing sensor otherwise might result in a risk when defibrillating patient.
Carefully remove patient cabling to avoid patient discomfort and entanglement.

Cautions
For single use only – do not reuse. Reuse may pose a risk of infection and/or might result in measurement error.
Ensure that there is no direct pressure on the sensor, especially when the patient is in the prone position. Continuous pressure on the sensor may cause skin damage.
Limit to short-term use (maximum of 24 hours). Longer use of the sensor might result in measurement error and/or skin damage especially on a patient with liver disease.
Do not use if the electrode gel has dried out. To avoid dry-out, do not open the cap until ready for use. Do not use an expired sensor. Using an expired unit or expired sensor might result in a measurement error.
Do not cut sensor components. Cutting sensor components can result in improper operation.
Do not reposition the sensor. Repositioning the sensor might cause poor contact.

Note
Constant attention by a qualified professional is needed whenever these accessories are used.

Technical information
Storage temperature: 10 to 30°C (50 to 86°F).
Operating temperature: 10 to 40°C (50 to 104°F).
Transport temperature: 10 to 50°C (50 to 122°F).

This sensor is safe to use during electrostimulation and defibrillation.

Trademarks
GE, GE Monogram, and Entropy are trademarks of General Electric Company or one of its subsidiaries.

Ordering information
For more information please visit www.gehealthcare.com or contact your local sales company.

Copyright © 2009-2015 General Electric Company. All rights reserved.

GE Healthcare Finland Oy
Kustannuskatu 2
FI-00501 Helsinki, Finland
+358 9 23394 11
www.gehealthcare.com

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Copyright © 2009-2015 General Electric Company. Всички права запазени.
GE, GE Monogram and Entropy are trademarks of General Electric Company or one of its subsidiaries.
Copyright © 2009-2015 General Electric Company. Alle rettigheder forbeholdes.