

# NMT Sensor Cables

**REF** 888415 1.5 m/5 ft.  
888414 3.3 m/11 ft.

## Explanation of symbols


**LOT** Batch code       Caution, consult accompanying documents  
**REF** Catalog number      **LATEX FREE** Does not contain latex  
 This package can be recycled

## Only (USA)

U.S. Federal law restricts this device to sale by or on the order of a physician.

## Intended use

Use NMT Sensor Cable only with NMT Sensors on GE Healthcare Monitors.

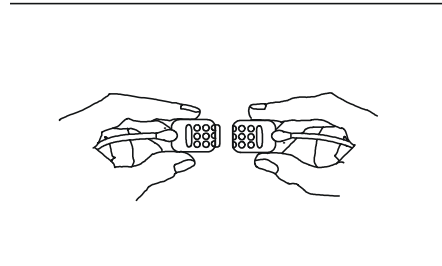

## Instructions for use

Connect a NMT Sensor with the NMT Sensor Cable. Plug the NMT Sensor Cable to the NMT measuring device.

## Maintenance

Before use, check that the cable is intact and clean.

## Cleaning

Gently clean the product with a detergent using a pad. Follow the cleaning instructions of the detergent, e.g. in case you need to rinse. Allow the product to dry completely after cleaning.

## Disinfection

Disinfect the cleaned product with a disinfectant such as ethanol, isopropyl alcohol or a chlorite compound. Do not use hypochlorite as a disinfectant. If the product is immersed into the disinfectant (e.g. iodex), the product must be rinsed and dried well.

## Ordering information

For further information please consult your GE Healthcare Supplies & Accessories catalogue or your local sales company.

Copyright © 2006 General Electric Company. All rights reserved.

GE Healthcare Finland Oy  
Helsinki, Finland  
358 10 394 11

