GE Healthcare

Reusable Temperature Probes

Contains: 1 Piece

en	Instructions For Use
cs	Návod k použití
da	Instruktioner
de	Gebrauchsanweisung
et	Kasutusjuhend
el	Οδηγίες χρήσης
es	Instrucciones de uso
fr	Mode d'emploi
ko	사용 지침
hr	Upute za upotrebu
it	Istruzioni per l'uso
lv	Lietošanas instrukcija
lt	Naudojimo instrukcija
hu	Használati útmutató
nl	Gebruiksaanwijzing
jp	取扱説明書
no	Bruksanvisning
pl	Instrukcja użycia
pt	Instruções de utilização
pt-br	Instruções de uso
ro	Instructiuni de utilizare
ru	Инструкция по применению
sk	Návod na použitie
sr	Uputstvo za upotrebu
fi	Käyttöohjeet
sv	Bruksanvisning
tr	Kullanım Talimatları
zh	使用说明

Type: Esophageal/Rectal, Pediatric

REF 1010132010GE

Type: Esophageal/Rectal, Adult

REF 1010132011GE

Type: Skin Application, Adult

REF TRRE 1010132023GE

Indication for Use

The series 400 reusable probes are intended for the continuous measurement and monitoring of patient temperature using compatible GE patient monitors. They are available in various designs.

The temperature sensors are approved for use by professional medical staff and for professional health care facility application only.

The materials that have contact to the patient during application comply with the requirements of biocompatibility according to ISO 10993-1.

Instruction for Use

1. Temperatures can be taken e.g. in the oesophagus, on the skin surface or rectally. Please use water-based lubricants only.
2. **Skin:** place the probe on the skin using plasters or in special circumstances on the extremities such as fingers or toes. Make sure that the skin contact is not interrupted by movement.
3. **Oesophagus:** the probe should be placed in the lower third of the oesophagus.
4. **Rectally:** The probe is in correct position when it has passed the sphincter.

Environmental Specification

Reusable and autoclavable (hotsteam sterilization)

Overall length: 280 cm (nominal)

Temperature range:	Storage/Transport:	-30 °C to +70 °C
	Operation:	+0 °C to +50 °C

Humidity range:	Storage/Transport:	10 to 95 % r. h. non-condensing
	Operation:	10 to 100 % r. h.

Ambient pressure:	Storage/Transport:	800 to 1060 hPa
	Operation:	800 to 1060 hPa
	Accuracy:	25 °C to 45 °C ± 0.1 °C

Transient response time (2 K transient):	Adult Skin types:	< 45 s
	Adult R/E Types:	< 20 s
	Pediatric R/E types:	< 25 s

Disposal

Before disposing of the product it may be necessary to clean and disinfect it. Local regulations may apply for disposal of used medical devices.

Cleaning and disinfecting the probe (reprocessing)

1. Before re-use of the probe it must be cleaned, disinfected and/or sterilized.
2. Before cleaning or disinfecting the sensor, it has to be detached from the monitor.

3. Temperature probes can be cleaned with a cleaning agent, by wiping the probe with a soft cloth or by washing in a machine at up to 95 °C. Avoid high mechanical force to the probe during cleaning process.
4. To disinfect the probe, wipe the probe and the contact surfaces with disinfectant or immerse the probe in disinfectant solution. We recommend the use of Cidex® OPA or Incidin® Plus. In any case refer to the IfU supplied by the disinfectant manufacturer. Avoid the use of disinfectants containing ketones.

Steam sterilisation

Steam autoclave at 132 °C/4min or 134 °C/5min using standard cycles. Follow the generally applicable instructions and regulations! Clean any residues from the temperature probe before sterilisation and disinfect the probe according to applicable reprocessing rules.

Warnings

1. Use only with compatible GE patient monitors. Please refer to the GE monitor/module user documentation for detailed compatibility information.
2. Strong sources of electromagnetic fields, e.g. electro-surgical devices, can adversely affect the functioning of the equipment. If at all possible, the probe should be completely removed from the patient before a surgical device or other HF source is activated! Check the protective earthing if you intend to use HF devices and temperature measuring equipment in parallel. Minimise the potential danger by selecting a temperature measurement point far away from the HF current path.
3. Do not use the product within range of magnetic resonance tomography (MRT) or computed tomography (CT) devices.
4. Stop using the probe immediately if it becomes damaged.
5. For further information and warnings please read instructions for use accompanying the temperature measuring device.

Caution

Improper use of probes can effect damages of the internal wiring and injure electrical insulation as well as incorrect temperature readings. In case of sensor damage the sensor must not be used and has to be exchanged.

Maintenance/Service

The temperature probes do not contain serviceable or replaceable components. Modification or replacement of components is generally not allowed. It is recommended to run an inspection of accuracy (MTK) at intervals of two years.

Returning the product

Every product sent to us for inspection must have been cleaned and disinfected beforehand.

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GE and GE Monogram are trademarks of General Electric Company. All other brands are trademarks of bluepoint medical GmbH & Co. KG.
GE Healthcare Finland Oy
bluepoint medical GmbH & Co. KG