Patient Spirometry Kit

REF	8004381	D-lite+ Spirometry Kit, 2m/7ft
	8004382	D-lite+ Spirometry Kit, 3m/10ft
	8002718	Pedi-lite Spirometry Kit, 2m/7ft
	M1032634	Pedi-lite Spirometry Kit, 3m/10ft
	889560	D-lite Spirometry Kit, 3m/10ft

Symbols

Refer to the separate leaflet 2087478-001 located within the product package for an explanation of symbols.


Intended use

For GE monitors with Patient Spirometry function to enable Spirometry measurement.
D-lite+ is intended for patients with tidal volumes 150 to 2000 ml and Pedi-lite+ is intended for patients with tidal volumes 5 to 300 ml.
D-lite- and Pedi-lite- are intended for use in humid conditions.

Expected service life

Expected service life is the time period during which the patient spirometry kit is expected to remain safe for use. The following criteria can be used to determine the end of the expected service life.

- The patient spirometry kit sensor becomes foggy or dirty.
- The patient spirometry kit has been used on a single patient during a single procedure.
- 24 hours has elapsed after spirometry sensor is taken into use.

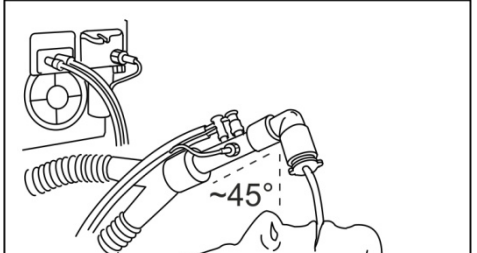2076382-001 Rev. C 2016-02-09

Material

PS, PVC, TPE
This product is not made with natural rubber latex.

Instructions for use

- Confirm that you have selected the correct D-lite type in the Patient Spirometry menu.
- The monitor does not detect the type of the used sensor. Check the lower tidal volume limit from the module and the monitor manual.
- Connect the Patient Spirometry tube to the module and the gas sampling line to the water trap. Ensure that all connections are secure.
- Check that all parts of D-lite are upwards.
- Patient D-lite is inclined in 45° angle and the patient side higher than the machine side to prevent water flow to the patient.

It is crucial that the D-lite is never positioned horizontally, incline position increases water outflow from the sensor and thus improves the measurement accuracy.

Maintenance

The patient spirometry kit is disposable.
This product and package should be disposed of according to local environmental and waste disposal regulations.

Warnings

Single-use products are not designed to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy and/or system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization and/or reuse.
Constant attention by a qualified professional is needed whenever these accessories are used.
This product should not be used if the integrity of the packaging has been compromised or there are visible cracks on the product.
Do not wash or disinfect the product.
No modification is allowed.

Cautions

Disconnect the D-lite, D-lite-, Pedi-lite or Pedi-lite- while a nebulizer is in use. Any type of cleaning or disinfection method will affect the characteristics of the surface of the spirometry sensor and lead to erroneous readings of the monitored values.

Technical information

Storage temperature: -50°C to +50°C (23°F to 122°F)
D-lite+ dead space is 9.5 ml.
Pedi-lite+ dead space is 2.5 ml.
For technical description, see the host device user guide or manual, or contact your local sales representative.

Trademarks

D-lite is a trademark of GE Healthcare Finland Oy. GE and GE Monogram are trademarks of General Electric Company.

Ordering information

For further information please consult your GE Supplies & Accessories catalog or your local sales contact.

Copyright © 2015 General Electric Company. All rights reserved.

English	Български
GE	БЕЛГАС
GE Healthcare Finland Oy Kuortaneentie 2 FI-00510 Helsinki, Finland Tel: +358 10 394 11 www.gehealthcare.com	

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Copyright © 2015 General Electric Company. Всички права запазени.
Copyright © 2015 General Electric Company. Reservados todos los derechos.
Copyright © 2015 General Electric Company. Všechna práva vyhrazena.
Copyright © 2015 General Electric Company. Tous droits réservés.