Invasive Blood Pressure Cables	English
Reusable	
REF 2104158-001, -002, -003, -004	
Invasive Blood Pressure Cables, ICU Medical Transpac-IV	
REF 2104166-001, -002, -003, -004	
Invasive Blood Pressure Cables, Argon	
REF 2104162-001, -002, -003, -004	
Invasive Blood Pressure Cables, Edwards LS TruWave	
REF 2104107-001, -002, -003, -004	
Invasive Blood Pressure Cables, Utah/Biotrans	
Note: -001 Single 3.6m/12ft, -002 Single 1.2m/4 ft, -003 Dual 3.6m/12 ft, -004 Dual 1.2m/4 ft.	
REF 2103966-001	
Invasive Blood Pressure Dual Adapter Cable 30 cm/1 ft U	

Intended use

The invasive blood pressure cables are intended to be used for connecting one or two invasive blood pressure transducers to GE monitoring equipment requiring rectangular 1 pin connector cable. These accessories are indicated for use by qualified medical personnel only.

Instructions for use

- Before use, check that the product is dry, intact, and clean.
- Connect the cable single or dual to the host monitor/module and to a compatible transducer. Note 2104158-001, -002, -003, -004 cables are compatible with ICU Medical Transpac-IV transducers, 2104166-001, -002, -003, -004 cables are compatible with Argon transducers, 2104162-001, -002, -003, -004 cables are compatible with Edwards LS TruWave transducers, and 2104107-001, -002, -003, -004 cables are compatible with Utah/Biotrans transducers)
- Please refer to the monitor/module user information for detailed compatibility information.
- When the invasive blood pressure monitoring is completed, disconnect the cable from the transducer and monitor/module.

Symbols

The symbols in Figure 1 apply to this product.

Figure 1

Refer to the separate leaflet 2106936-001 located within the product package for an explanation of symbols.

Warnings

- Use only supplies and accessories approved by GE for proper and accurate functioning of measurement. Using components other than the ones approved and listed in the device manual might result in measurement errors and hazardous connection.
- Do not use the cables if you see any visual signs of damage. A damaged product should be discarded immediately. Never try to repair a damaged product or use a product repaired by others.
- Check that the product is dry, intact, and clean to reduce the risk of cross-contamination, possible effects to the measurement accuracy and/or system performance, the risk of malfunction, and compromised electrical safety.
- Carefully route patient cables to avoid patient discomfort and entanglement.
- Mating connectors with force without proper alignment may damage the connectors and disrupt the electrical circuitry.

Cautions

- To avoid personal injury to users or to other persons moving in the vicinity of the cables or tubing, route all cables and tubing in such a way that they do not present a tripping hazard.
- To avoid skin damage, ensure that the accessory is not pressing against the patient and check especially that the patient is not lying on the connector.

Maintenance

- Do not treat the cables with oil/s, grease, household or laboratory or aggressive fluids or solvents (e.g. alcohol). Do not sterilize or immerse the cables in liquids. Protect from strong UV radiation.

Cleaning

- Wipe the product with a cloth moistened with mild detergent or soap solution until it is visually clean. Allow the product to dry completely after cleaning.

Disinfection

Disinfect the product according to your general hospital cleaning and disinfection practices. Following disinfectant agents can be used on this product without compromising the product safety: Ethyl Alcohol (90 %), Isopropyl Alcohol (60 %), Glutaraldehyde (2 %), and Phenol (2 %).

Expected service life

Expected service life is the time period during which the sensor or cable is expected to remain safe for use. The following criteria can be used to determine the end of the expected service life:

- The sensor or cable shows signs of physical damage.
- The sensor or cable is malfunctioning or the monitor shows unusual data.

Disposal

This product and package should be disposed of according to local environmental and waste disposal regulations.

Technical information

Non-operating temperature: -30 to 60°C (-22 to 140°F)

Operating temperature: 10 to 40°C (50 to 104°F)

Operating and non-operating humidity: 10 to 90% RH non-condensing

Trademarks

GE and GE Monogram are trademarks of General Electric Company.

Ordering information

For further information, please visit www.gehealthcare.com or contact your local sales company.

Copyright © 2017 General Electric Company. All rights reserved.

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Copyright © 2017 General Electric Company. Reservados todos los derechos. Fabricado en Taiwan.