English

Supplement for CARESCAPE Respiratory Modules, N-CAiO and E-miniC modules

5851452-99

1st edition

2020-07-15

© 2020 General Electric Company. All rights reserved.

All specifications are subject to change without notice.

GE Healthcare Finland Oy

Kuortaneenkatu 2

FI-00510 Helsinki, Finland

Tel: +358 10 39411

www.gehealthcare.com

Intended use of this supplement

This supplement covers necessary updates to the host device's and modules' user and service documentation. New gas exhaust lines have been introduced due to changes in international standards related to gas exhaust connector design requirements.

This supplement is to be used with the following products:

- CARESCAPE Respiratory Modules
- Airway Gas Option, N-CAiO
- Single-width Airway Module, E-miniC

Choosing the correct gas exhaust line

Make sure that you are using a gas exhaust line that is compatible with the module's gas exhaust connector. Prefixes SGV, SUA and SUB listed in this supplement refer to the first three characters in the serial number on the module device label. The following figures show the compatible gas exhaust lines for each connector.

Luer exhaust connector:

1. Module with Luer exhaust connector (prefix SGV in the serial number with CARESCAPE Respiratory Modules and N-CAiO module, and E-miniC module without prefix in the serial number). Note that this connector has a distinct collar around the male pin.
2. M1031274 Exhaust line for gas return or scavenging
3. With compatible GE Anesthesia Carestations: 8004462 Exhaust line with Colder fitting 1 m (41 in) or 8004463 Exhaust line with Colder fitting, 18 cm (7 in)

GE custom exhaust connector:

1. Module with GE custom exhaust connector (prefix SUA in the serial number with CARESCAPE Respiratory Modules and N-CAiO module, and E-miniC module with prefix SUB in the serial number). Note that:
 - the connector does not have a collar around the male pin.
 - the diameter of the connector's male pin is larger than with the Luer exhaust connector.
2. 5514184 Gas exhaust line, white conical, disposable, 2 m/6.6 ft for gas scavenging
3. With compatible GE Anesthesia Carestations: 5514182 Gas exhaust line, white conical and Colder, disposable, 1 m/3.3 ft or 5514183 Gas exhaust line, white conical and Colder, disposable, 0.18 m/7 in

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