尺寸:W137mm x H95mm

CO M100 Y100 K20 CO M0 Y0 K80 CO M0 Y0 K50 C85 M50 Y0 K0


Unimed Medical Supplies, Inc
BldK8, Nangang 3rd Industrial Park, Tangtou, Shijian, 518108
Shenzhen, PEOPLE'S REPUBLIC OF China
Tel:+86 755 26695137 ext.860 Fax:+86 755 26697984
Web:www.unimed.cn Email:info@unimed.cn
Obelits s.a.
Boulevard Général Wahis 53
1030 Brussels, BELGIUM

REF: Y07-A055
Rev: A4

**Reusable SpO2
Adaptor/Extension Cable
User's Manual**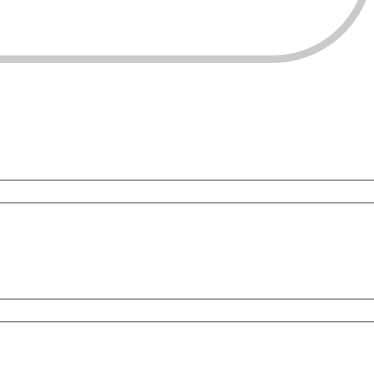

Contents	
1.....	English
4.....	French
8.....	Spanish
10.....	German
13.....	Polish
16.....	Portuguese
19.....	Russian
22.....	Korean

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Reusable SpO2 Adaptor/Extension Cable User's Manual

1 Indications
SpO2 adaptor/extension cables are designed for the electrical connection between patient monitor & pulse oximetry and SpO2 sensors, indicated for use in continuous non-invasive monitoring of arterial oxygen saturation and pulse rate. They are fully compatible sensors with major brands of patient monitor & pulse oximetry.

2 Packaging-Storage
The cables are individually packaged.
Cables must be stored in its original packaging and within the storage conditions to maximize the storage life of the sensor.
Storage conditions are as follows:

- Ambient temperature: -10 to +55°C
- Relative humidity: 10 to 85%

3 Cleaning
The cable must be cleaned using a soft cloth moistened with soapy water or 70% isopropyl alcohol. Allow the cable to dry thoroughly before use. Do not use any abrasive agents or chemical product except 70% isopropyl alcohol. Do not irradiate, autoclave, soak or immerse the cable in any kind of solution.

4 Warranty/Liability
The manufacturer offers a one-year warranty against defects in material and /or workmanship from the date of purchase. But it does not cover the damage or breakage due to the abusive use or negligent care of the cable. We guarantee that the equipment conforms to the specifications of the safety and performance standards currently in force and applicable to it.

5 Warning:

- Ensure the cable is physically intact, with no broken or frayed wires or damaged parts. Visually inspect the cable and discard if cracks or discoloration are found.
- Do not attempt to reprocess, recondition or recycle the cables as these processes may damage the electrical components, potentially leading to patient harm.
- Failure to properly connect the sensor or the oximeter module to the sensor will result in intermittent readings, inaccurate results, or no reading.
- To avoid damage to the cable, always hold it by the connector rather than the cable when connecting or disconnecting either end.
- Always refer to the oximeter module operator's manual for complete instructions or additional instructions.
- Carefully route cable and patient cable to reduce the possibility of patient entanglement or strangulation.

6 CAUTION:
Federal (U.S.) Law restricts this device to sale by or on the order of a physician.

Symbol explanation

Caution	Serial number	Manufacturer	Non-sterilization	Latex free	See the instructions	Not for reuse and disinfection

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