YI Attachment Wraps

For use with the YI Reusable Sensor

DIRECTIONS FOR USE
 Single patient use only  Not made with natural rubber latex  Non-sterile

Prior to using Masimo Attachment Wraps, the user should read and understand the Operator's Manual for the device and/or monitor, the Directions for Use for the YI Sensor, and this Directions for Use.

CONTRAINDICATIONS

YI Attachment Wraps are contraindicated for patients who exhibit allergic reactions to foam urethane products and/or adhesive tape.

DESCRIPTION

Masimo® YI attachment wraps are used to attach the YI sensor to the application site. The attachment wraps are for single-patient use only.

INSTRUCTIONS

A) Attachment Wrap Selection by Body Weight

Body Weight	Cleanshield® Multisite Wrap	Standard Petite Wrap	Standard Wrap	Foam Wrap
 1 kg – 3 kg				
3 kg – 10 kg				
10 kg – 30 kg				
10 kg – 50 kg				
> 30 kg				

B) Attaching the Wrap

- Refer to **Fig. 1**. The foam wrap can be trimmed to the shorter length for smaller site applications such as a child's finger or toe or a pre-term infant's foot or hand.
- Remove the backing from the Standard Wrap / Cleanshield Wrap / Standard Petite Wrap, leaving the backing on the tab end.
- Refer to **Fig. 2a**. Push the "button" on the emitter sensor pad (cable is marked with a red indicator) through the hole on the left **Fig. 2b**, and the remaining button on the detector sensor pad through the hole on the right **Fig. 2c**.

NOTE: Refer to **Fig. 2d**. Use the center hole of the Petite Wrap for patients 10 – 30 kg.

- For the Cleanshield Wrap only**, refer to **Fig. 3**. Fold the upper portion of the attachment tape down over the sensor pads and remove the release liner from the folded over section of the adhesive wrap prior to applying the sensor to the site.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Consult instructions for use		Mark of Conformity to European Medical Device Directive 93/42/EEC
	Follow instructions for use		Authorized representative in the European community
	Manufacturer		Lot code
	Use-by date YYYY-MM-DD		Catalogue number (model number)
	Do not re-use/Single patient use only		Storage-humidity limitation
	Non-Sterile		Storage-temperature range
	Not made with natural rubber latex		Keep dry
	Less than		Greater than
	Do not use if package is damaged		Body weight
Rx ONLY	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.		Instructions/Directions for Use/Manuals are available in electronic format at http://www.masimo.com/techdocs Note: eRFU is not available for CE mark countries.

Patents: <http://www.masimo.com/patents.htm>

Masimo, is a federally registered trademark of Masimo Corporation.

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Masimo este o marcă comercială înregistrată la Masimo Corporation.
Masimo je federálne registrovaná ochranná známka spoločnosti Masimo Corporation.