Masimo® Forehead Sensor Headband

DIRECTIONS FOR USE

en

- DESCRIPTION**
The Masimo Forehead Sensor Headband is indicated for use with the TFA-I disposable transreflectance sensors and the TF-I reusable transreflectance sensors.
- WARNINGS**
- The site must be checked frequently or per clinical protocol to ensure adequate adhesion, circulation, skin integrity and correct optical alignment.
 - Exercise extreme caution with poorly perfused patients; skin erosion and pressure necrosis can be caused when the sensor is not frequently moved. Assess site as frequently as every (1) hour with poorly perfused patients and move the sensor if there are signs of tissue ischemia.
 - During low perfusion, the sensor site needs to be assessed frequently for signs of tissue ischemia, which can lead to pressure necrosis.
 - Do not use tape to secure the sensor to the site; this can restrict blood flow and cause inaccurate readings. Use of additional tape can cause skin damage, and/or pressure necrosis or damage the sensor.
 - Sensors applied too tightly or that become tight due to edema will cause inaccurate readings and can cause pressure necrosis
 - Misapplied sensors or sensors that become partially dislodged may cause incorrect measurements.

- INSTRUCTIONS**
1. Following the instructions that are included with the sensor, apply the sensor to the forehead. Sensor should be just above the eyebrow in line with the center of the eye (pupil).
 2. Apply the headband. Headband should be secure enough to apply slight pressure to the sensor and should completely cover the sensor (Fig. 1).
 3. The cable should be routed up and over the headband so the cable does not apply pressure to the skin and is not pulling on the sensor.
- The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Mark of conformity to European medical device directive 93/42/EEC		Separate collection for electrical and electronic equipment (WEEE).	Rx ONLY	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Consult instructions for use		Atmospheric pressure limitation		Follow instructions for use (blue background)
	Manufacturer	REF	Catalogue number (model number)	EC/REP	Authorized representative in the European community
	Date of manufacture YYYY-MM-DD	###	Masimo reference number		Fragile, handle with care
	Single patient use only		Storage humidity limitation		Storage temperature range
	Non-Sterile		Do not use if package is damaged		Keep dry
	Not made with natural rubber latex		Instructions/Directions for Use/Manuels are available in electronic format @ http://www.Masimo.com/TechDocs Note: eIFU is not available in all countries		

Patents: <http://www.masimo.com/patents.htm> Masimo, TF-I and are federally registered trademarks of Masimo Corporation.

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© 2019 Masimo Corporation.